
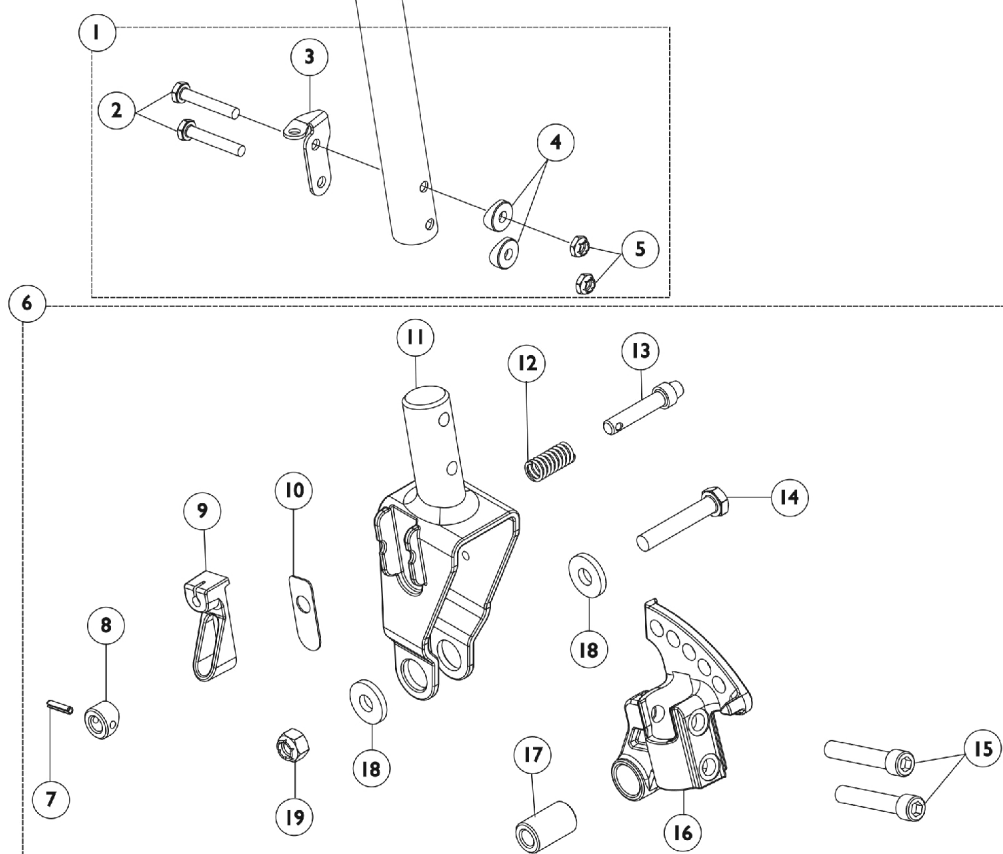


Tiller Angle Adjustment Hardware

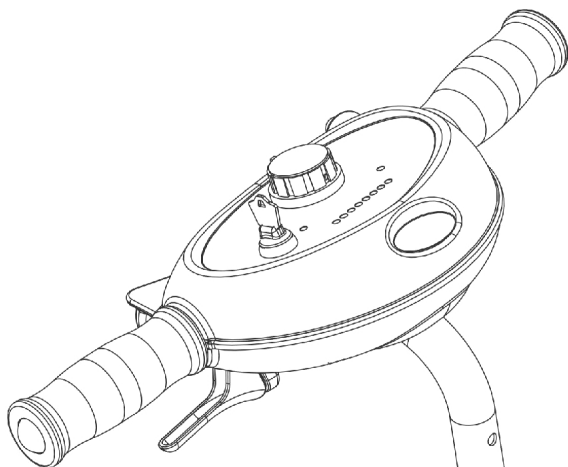

Tiller Angle Adjustment Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B,C	1120475-NLA	NLA - Kit, Tiller Mounting Hardware (Invacare 300 Scooter)	1	1
2			Screw, Hex Head (1/4-20 x 1-1/2")	1	2
3			NLA - Bracket, Cable Mounting, Black	1	1
4			Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
5			NLA - Locknut (1/4-20)	1	2
1	B,D	1125361-NLA	NLA - Kit, Tiller Mounting Hardware (Invacare 400 Scooter)	1	1
2			Screw, Hex Head (1/4-20 x 1-5/8")	1	2
3			NLA - Bracket, Cable Mounting, Black	1	1
4			Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
5			NLA - Locknut (1/4-20)	1	2
6	A	1120471-NLA	NLA - Kit, Adjustable Angle Tiller	1	1
7			NLA - Pin, Locking	1	1
8			NLA - Spring, Compression (7/8" L)	1	1
9			NLA - Pivot, Adjustable Angle Upper	1	1
10			NLA - Spacer	1	1
11			NLA - Wedge, Actuator	1	1
12			NLA - Glide, Adjustable Angle	1	1
13			NLA - Pin, Roll (1/8" x 9/16")	1	1
14			Locknut (5/16-18)	1	1
15			Washer, Flat (21/64 x 53/64 x 7/64")	1	2
16			NLA - Bushing (5/16 x 5/8 x 1")	1	1
17			NLA - Pivot, Adjustable Angle Lower	1	1
18			Screw, Socket Head w/ Patch (5/16-18 x 1-3/4")	1	2
19			NLA - Screw, Hex Head (5/16-18 x 1-3/4")	1	1
NOTE: A - Includes: Items 7-19. B - Includes: Items 2-5. C - Used on the Zoom 300 Compact Scooter. D - Used on the zoom 400 Midsize Scooter.					