
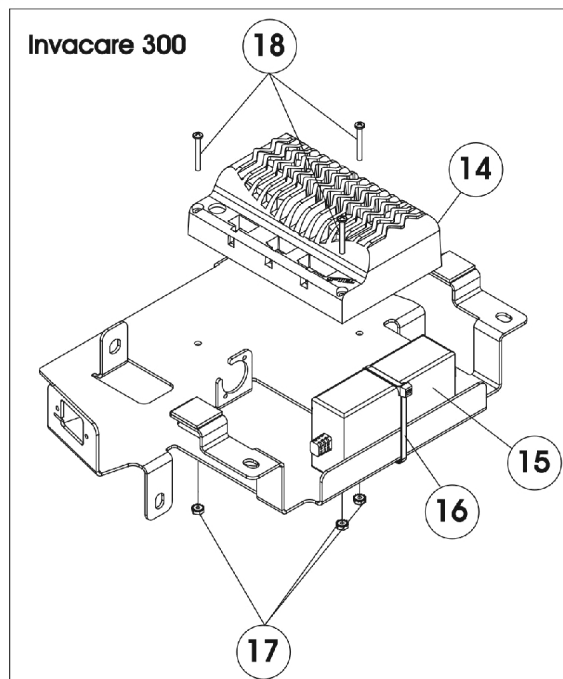


Charger and Controller

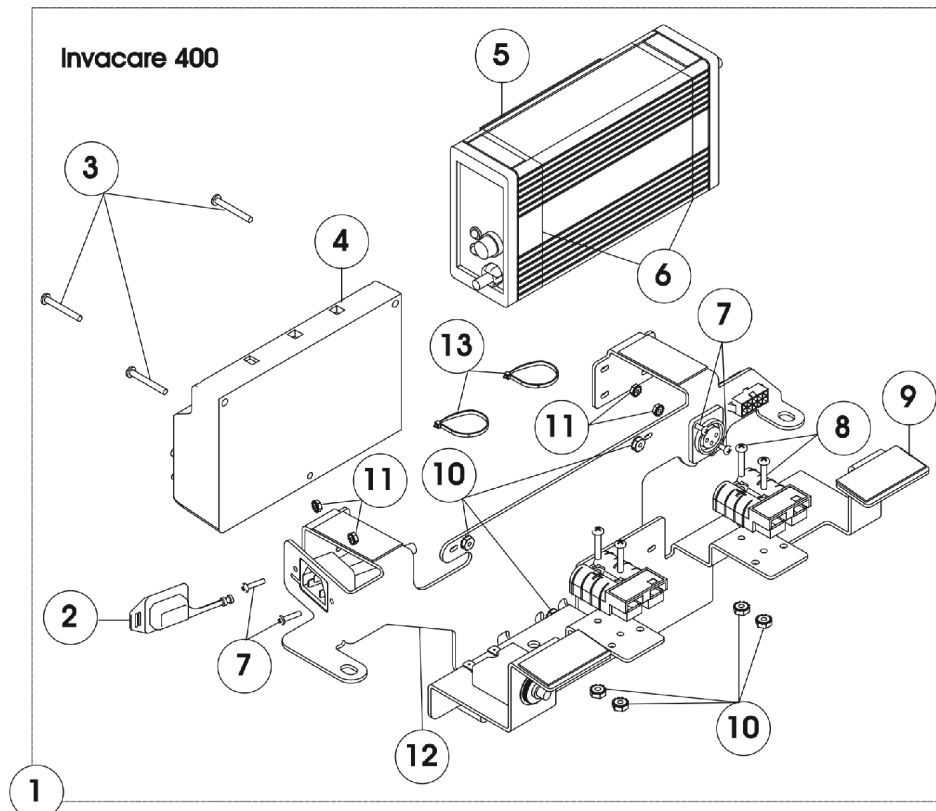

Charger and Controller

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B,F,G	1125393-NLA	NLA - Kit, Controller & Battery Charger w/ Tray Assembly (Invacare 400 Scooter)	1	1
2			Cover, Connector	1	1
3			NLA - Screw, Phillips Pan Head (#6-32 x 1-1/8")	1	3
4			NLA - Controller (Invacare 400 Scooter)	1	1
5			NLA - Charger, Battery 3 Amp. (Invacare 400 Scooter & LX3)	1	1
6			Tie Wrap (17-1/2" L)	1	2
7			Screw, Phillips Pan Head (#4-40 x 1/2")	1	3
8			NLA - Screw, Phillips Pan Head (#6-32 x 3/4")	1	4
9			NLA - Fastener, Hook with Adhesive Back (1" W x 1" L)	1	1
10			NLA - Nut, Keps (#6-32)	1	7
11			Nut, Hex Keps (#4-40)	1	4
12			NLA - Tray, Controller & Battery Charger (Invacare 400 Scooter)	1	1
13			Tie Wrap (5-5/8" L)	1	6
5	F,G	1114176-NLA	NLA - Charger, Battery 3 Amp. (Invacare 400 Scooter & LX3)	1	1
6		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	1
14		1119183-NLA	NLA - Controller (Invacare 300 Scooter)	1	1
15	E	1116914-NLA	NLA - Charger, Battery 2 Amp. (Invacare 300 Scooter & SX3)	1	1
16	E	1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	1
17		P0870X050-NLA	NLA - Nut, Keps (#6-32)	1	3
18		1039773-NLA	NLA - Screw, Phillips Pan Head (#6-32 x 1-1/8")	1	3
19		1125394	NLA - Kit, Controller & Battery Charger Mounting Tray Hardware	1	1
	D,F		NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	4
			NLA - Locknut (1/4-20)	1	4
			Washer (1/4 x 23/32 x 1/16")	1	8
			Cord, AC Power, 115VAC, 3 Prong On-Board (6' 6-1/2" L)	1	1
			NLA - Package, Tie Wrap (5-5/8" L)	10	1
	A	1121086	Batteries	1	1
	A,C	1095503-NLA			
	A	See Chart			

NOTE: A - Not Shown.
 B - Includes: Items 2-12 and Rear Wire Harness (found on "Rear Wiring Harness" page).
 C - These Cable Ties bind the wires to each other.
 D - Includes: (4) 1/4-20 x 1-5/8 Hex Head Cap Screws, (8) Washers, and (4) 1/4-20 Hex Lock Nuts.
 E - Used on the Zoom 300.
 F - Used on the Zoom 400.
 G - Item #5 Battery Charger is no longer available for sale individually. Must order item #1 "Kit, Controller & Battery Charger w/ Tray Assembly" complete.

Battery

Equipment Number	Description
------------------	-------------

Charger and Controller

Battery

Equipment Number	Description
12AH-2 See Note(s) A, B	Batteries, 12V (pair)
U1BH-2 See Note(s) C, A	Batteries, U1 with Handle (pair)
NOTE: A - Batteries are an equipment item and numbers are shown for reference only. B - Used on the Zoom 300. C - Used on the Zoom 400.	