



Back Canes, Backrest Crossmember, and Pelvic Positioning Strap

Transport Ready Chairs requiring Backrest Hardware parts must be returned to Invacare for service.

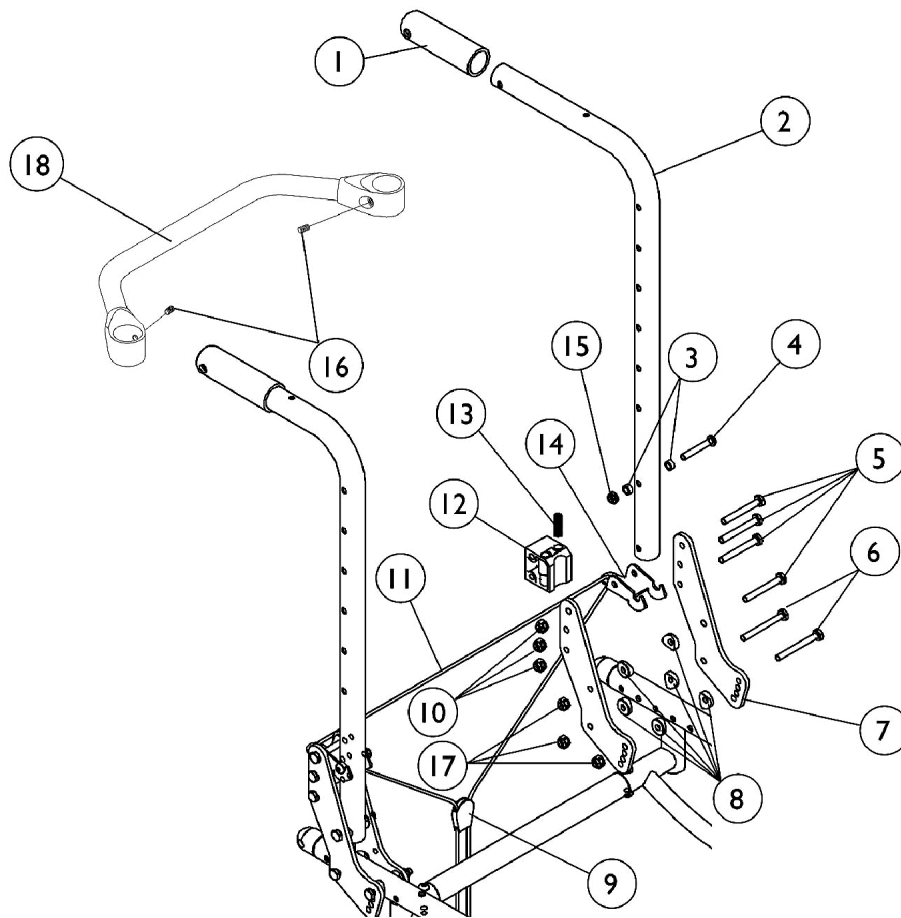

Non Transport Ready Chairs

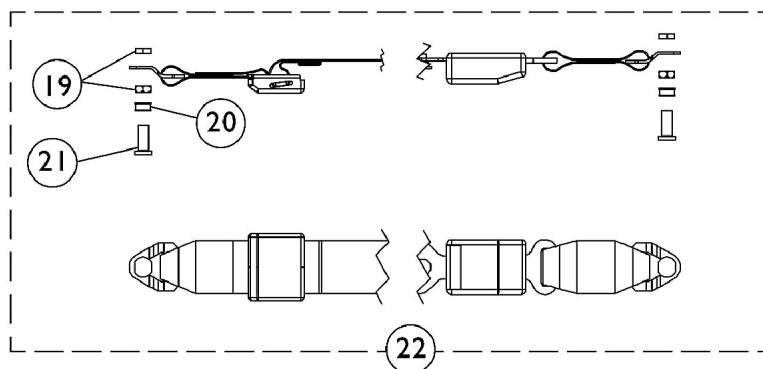

Pelvic Positioning Strap

Back Canes, Backrest Crossmember, and Pelvic Positioning Strap

Transport Ready Chairs requiring Backrest Hardware parts must be returned to Invacare for service.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1055398	Grip, Hand, Foam (4")	1	2
2	A	1121600	Back Cane Assembly, Black - Right/Left (20" H)	1	2
2	B	1120035-NLA	NLA - Back Cane Assembly, Black - Right/Left (20" H)	1	2
2	A,D	1120038-NLA	NLA - Back Cane Assembly, Chrome - Right/Left (20" H)	1	2
2	A	1121601	Back Cane Assembly, Black - Right/Left (24" H)	1	2
2	B	1120036-NLA	NLA - Back Cane Assembly, Black - Right/Left (24" H)	1	2
2	A,D	1120040-NLA	NLA - Back Cane Assembly, Chrome - Right/Left (24" H)	1	2
3		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
4		1019590	Screw, Socket Button Head (1/4-20 x 1-5/8")	1	2
5		1055349	Screw, Hex Head (1/4-28 x 2")	1	8
6		1055350	Screw, Hex Head (1/4-28 x 2-1/4")	1	4
7		1079636-NLA	NLA - Plate, Back	1	4
8		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	12
9		1029412	Lock, Cord, Black	1	1
10		1065362	Locknut (1/4-28)	1	12
11		1141633	Cord, Nylon (6-1/2 ft.)	1	1
12		1046001-NLA	NLA - Block, Stop	1	2
13		1048802-NLA	NLA - Spring, Compression (5/16 x 1-3/8 x 1/32")	1	2
14		1046000-NLA	NLA - Latch, Seat, Black	1	2
15		1025195	Locknut (1/4-20)	1	2
16		1045495-NLA	NLA - Screw, Set Socket (1/4-20 x 1/4")	1	2
17		1085111	Nut, Hex (1/4-28)	1	6
18		See Chart	#18 Backrest Crossmember	1	1
19		1118579-NLA	NLA - Nut, Hex Jam (3/8-16)	1	4
20		1118568-NLA	NLA - Bushing, Nylon (5/16 x 7/16 x 5/32")	1	2
21		1118569-NLA	NLA - Pin, Threaded	1	2
22	E,F	1118580-NLA	NLA - Pelvic Positioning Strap w/ Hardware (37" Long)	1	1
22	C,E	1119980	Pelvic Positioning Strap w/ Hardware (55" Long)	1	1

NOTE: A - Use on chairs with Fixed Height, Adjustable Height, or "T" Arms
 B - Use on chairs with Cantilever Arms (AT99N, AT55D, & AT55F)
 C - 17-22" wide chairs
 D - Chrome chairs only. (Chrome is no longer available on chairs built After 5/13/03)
 E - Includes Positioning Strap plus items 19-21
 F - 12-16" wide chairs

Back Canes, Backrest Crossmember, and Pelvic Positioning Strap

Transport Ready Chairs requiring Backrest Hardware parts must be returned to Invacare for service.

#18 Backrest Crossmember

12" Wide	13" Wide	14" Wide	15" Wide	16" Wide	17" Wide
1042508-NLA	1042509-NLA	1042510	1042511	1042512	1050083
18" Wide	19" Width	20" Width	21" Width	22" Width	
1050080	1050077	1050074	1085292-NLA	1085293-NLA	