
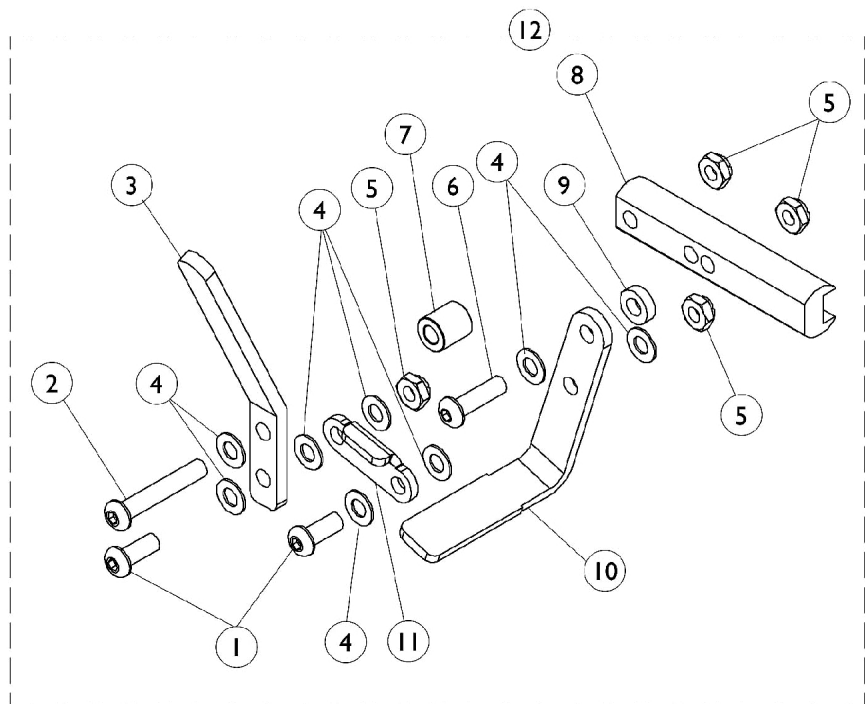
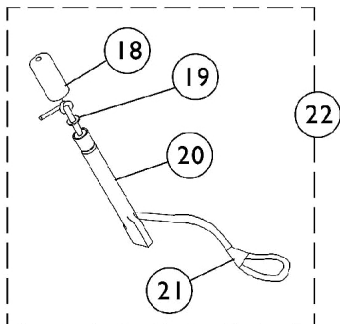


Wheel Locks

Chairs with 12" Rear Wheels and Vent Tray

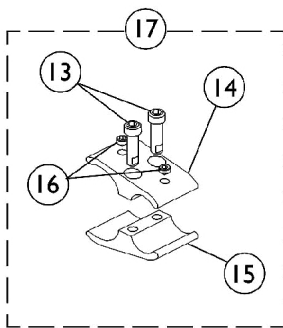

Wheel Locks

Chairs with 12" Rear Wheels and Vent Tray

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	2
2		1083099	Screw, Socket Button Head (1/4-28 x 1-1/2")	1	1
3		1028414	Handle, Wheel Lock, Black - Right/ Left	1	1
4		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	8
5		1025765	Locknut (1/4-28)	1	4
6		1055306-NLA	NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
7		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
8		1080551-NLA	NLA - Strap, Wheel Lock	1	1
9		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
10		1021405-NLA	NLA - Shoe, Wheel Lock	1	1
11		1018733-NLA	NLA - Link, Stop, Black	1	1
12	A	1094542-NLA	NLA - Wheel Lock Assembly, Right	1	1
12	A	1094543-NLA	NLA - Wheel Lock Assembly, Left	1	1
13		90036X000-NLA	NLA - Screw, Allen Head (1/4-28 x 1")	1	2
14		1079550-NLA	NLA - Clamp, Non-Threaded	1	1
15		1079551-NLA	NLA - Clamp, Threaded	1	1
16		1101207-NLA	NLA - Screw, Set Socket Cone Point (1/4-20 x 1/4")	1	2
17	B	1085266-NLA	NLA - Wheel Lock Clamp Hardware	1	2
18		1128817	Package, Tip, Rubber, Black (Pkg. of 10)	1	1
19		1013218	Washer (#10 Flat)	1	1
20		1038597-NLA	NLA - Handle, Extension, Black (6")	1	1
21		1038662-NLA	NLA - Cord, Elastic	1	1
22	C	1063125	Wheel Lock Extension Handle Assembly	1	1
22	C	1038599	Wheel Lock Extension Handle Assembly, Black (6") (Pair)	2	1
	D	1039644-NLA	NLA - Instruction Sheet, Wheel Lock Extensions	1	1
NOTE: A - Includes items 1-11 B - Includes items 13-16 C - Includes items 18-21 D - Not Shown					