


Vent Tray Assembly

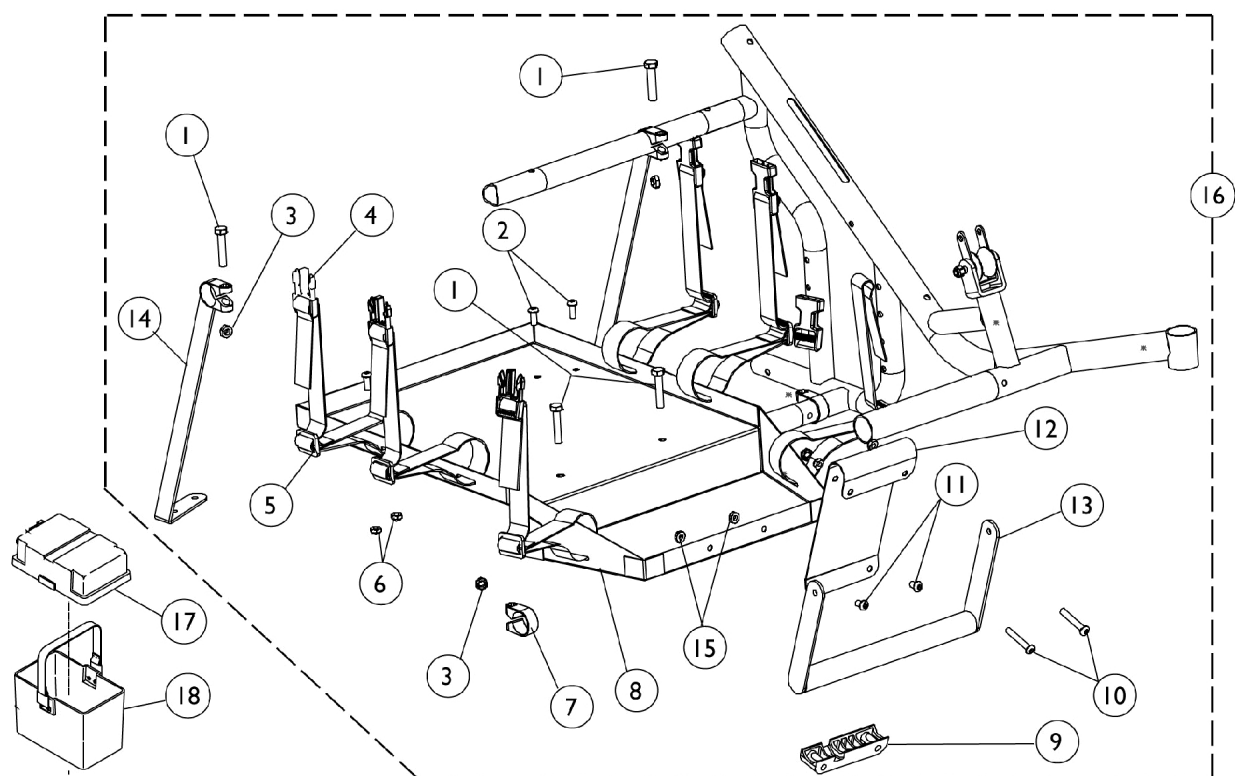

Vent Tray Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1044939-NLA	NLA - Screw, Hex Head (5/16-18 x 1-3/4")	1	4
2		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	4
3		1026158	Locknut (5/16-18)	1	4
4		1052426-NLA	NLA - Buckle, Slide Release	1	3
5		1052424-NLA	NLA - Strap with Slide Release Buckle (22" Long)	1	6
6		1025765	Locknut (1/4-28)	1	4
7		1093511-NLA	NLA - Clamp (1")	1	2
8		1093385-NLA	NLA - Tray, Ventilator	1	1
9		1012172-NLA	NLA - Clamp, Saddle	1	1
10		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
11		1088932	Screw, Socket Button Head (1/4-20 x 3/8")	1	2
12		1093386-NLA	NLA - Hanger, Battery Tray	1	1
13		See Chart	#13 Vent Tray Hanger	1	1
14		1093382-NLA	NLA - Hanger, Vent Tray - Right	1	1
14		1093383-NLA	NLA - Hanger, Vent Tray - Left	1	1
15		1025195	Locknut (1/4-20)	1	4
16		See Chart	#16 Vent Tray Assembly	1	1
17		1095747	Top, Battery Box	1	1
18	1057944	Battery Box Bottom with Strap, 22NF	1	1	
		1093582-NLA	NLA - Instruction Sheet, Vent Tray	1	1

NOTE: A - Not Shown
B - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.

#13 Vent Tray Hanger

16" Wide	17" Wide	18" Wide	19" Wide	20" Wide	21" Wide	22" Wide
1093496-NLA	1093497-NLA	1093498-NLA	1093499-NLA	1093500-NLA	1093501-NLA	1093502-NLA

#16 Vent Tray Assembly

16" Wide	17" Wide	18" Wide	19" Wide	20" Wide	21" Wide	22" Wide
1093637-NLA	1093638-NLA	1093639-NLA	1093640-NLA	1093641-NLA	1093642-NLA	1093643-NLA
Includes items 1-15						