
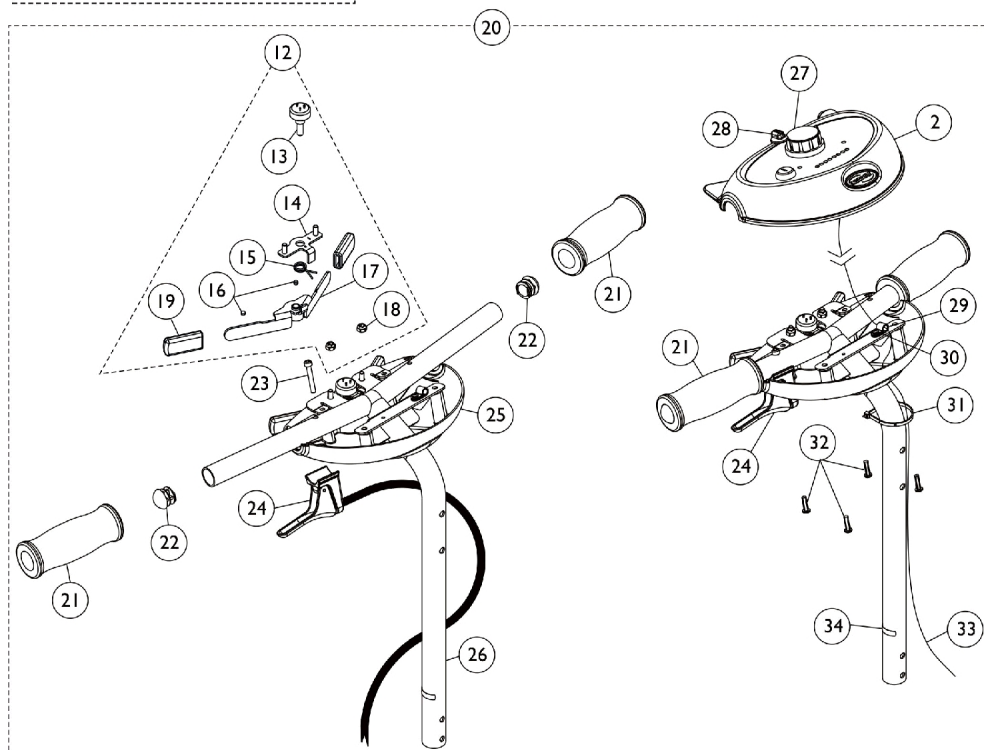


Tiller Assembly with Main Wiring Harness - Invacare 300 Scooter

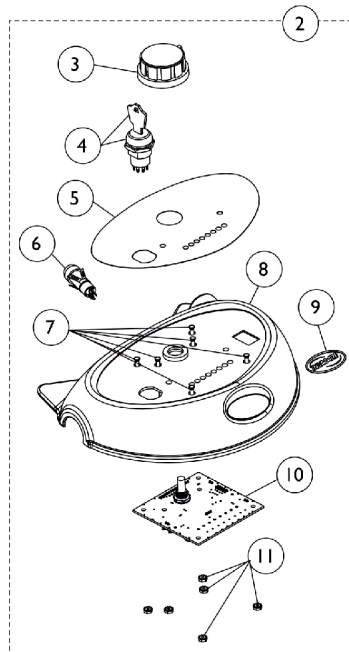

Tiller Assembly with Main Wiring Harness - Invacare 300 Scooter

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1126100-NLA	NLA - Key, Ignition	1	1
2		1118267-NLA	NLA - Shroud, Tiller with Circuit Board - Candy Red Pearl	1	1
3			Knob, Control	1	1
4			NLA - Key Switch with Key Assembly	1	1
5			NLA - Decal, Faceplate	1	1
6			Switch, Horn	1	1
7			NLA - Stud (#4-40 x 3/8")	1	6
8			NLA - Shroud, Tiller, Upper, Candy Red Pearl	1	1
9			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
10			NLA - Circuit Board with Speed Pot	1	1
11			Nut, Hex Keps (#4-40)	1	6
	A		NLA - Label, Patent Pending	1	1
2		1118268-NLA	NLA - Shroud, Tiller with Circuit Board - Deep Blue Pearl	1	1
3			Knob, Control	1	1
4			NLA - Key Switch with Key Assembly	1	1
5			NLA - Decal, Faceplate	1	1
6			Switch, Horn	1	1
7			NLA - Stud (#4-40 x 3/8")	1	6
8			NLA - Shroud, Tiller, Upper, Deep Blue Pearl	1	1
9			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
10			NLA - Circuit Board with Speed Pot	1	1
11			Nut, Hex Keps (#4-40)	1	6
	C,D		NLA - Label, Patent Pending	1	1
12		1119473-NLA	NLA - Kit, Throttle Lever with Instruction Sheet	1	1
13			Wig Wag (Throttle Pot)	1	1
14			Bracket, Support	1	1
15			Spring, Torsion	1	1
16			Screw, Set Socket Cup Point (#8-32 x 3/16")	1	2
17			Tiller Lever Assembly, Black	1	1
18			Locknut (#10-32)	1	2
19			NLA - Cap, End, Throttle, Black	1	2
			Instruction Sheet, Throttle Lever	1	1
			NLA - Bag, Poly Zip-Lip (8-3/4" L x 6" W)	1	1
20	B	1118430-NLA	NLA - Tiller Assembly - Deep Blue Pearl	1	1
2			Switch, Horn	1	1
2			NLA - Decal, Faceplate	1	1
2			NLA - Shroud, Tiller, Upper, Deep Blue Pearl	1	1
2			Nut, Hex Keps (#4-40)	1	6
4			NLA - Key Switch with Key Assembly	1	1
7			NLA - Stud (#4-40 x 3/8")	1	6
9			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
10			NLA - Circuit Board with Speed Pot	1	1
12			NLA - Throttle Lever Assembly	1	1
21			NLA - Grip, Hand, Black Foam	1	2

Tiller Assembly with Main Wiring Harness - Invacare 300 Scooter

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
22			Plug Button, Black (5/8")	1	2
23			NLA - Screw, Socket Head (#10-24 x 1-1/4")	1	1
24			NLA - Handle, Tiller Angle Adjustable with Cable (Invacare 300 Scooter)	1	1
25			NLA - Shroud, Tiller, Lower	1	1
26			NLA - Weldment, Tiller (Compact)	1	1
27			Knob, Control	1	1
28			NLA - Boot, Key Switch	1	2
29			NLA - Clamp, Cable	1	1
30			NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	1
31			Tie Wrap (5-5/8" L)	1	1
32			Screw, Phillips Pan Head (#8-16 x 5/8")	1	4
33			NLA - Harness, Wiring, Main Tiller	1	1
34			NLA - Label, Serial Number	1	1
			NLA - Label, Patent Pending	1	1
20	B	1118429	NLA - Tiller Assembly - Candy Red Pearl	1	1
2			Switch, Horn	1	1
2			NLA - Decal, Faceplate	1	1
2			NLA - Shroud, Tiller, Upper, Candy Red Pearl	1	1
2			Nut, Hex Keps (#4-40)	1	6
4			NLA - Key Switch with Key Assembly	1	1
7			NLA - Stud (#4-40 x 3/8")	1	6
9			NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	1
10			NLA - Circuit Board with Speed Pot	1	1
12			NLA - Throttle Lever Assembly	1	1
21			NLA - Grip, Hand, Black Foam	1	2
22			Plug Button, Black (5/8")	1	2
23			NLA - Screw, Socket Head (#10-24 x 1-1/4")	1	1
24			NLA - Handle, Tiller Angle Adjustable with Cable (Invacare 300 Scooter)	1	1
25			NLA - Shroud, Tiller, Lower	1	1
26			NLA - Weldment, Tiller (Compact)	1	1
27			Knob, Control	1	1
28			NLA - Boot, Key Switch	1	2
29			NLA - Clamp, Cable	1	1
30			NLA - Screw, Phillips Pan Head (#6-20 x 3/8")	1	1
31			Tie Wrap (5-5/8" L)	1	1
32			Screw, Phillips Pan Head (#8-16 x 5/8")	1	4
33			NLA - Harness, Wiring, Main Tiller	1	1
34			NLA - Label, Serial Number	1	1
			NLA - Label, Patent Pending	1	1

NOTE: A - Includes: Items 3-11 and Patent Label (not shown).
 B - Includes: Items 2, 4, 7, 9-10, 12, 21-34, Patent Label (not shown).
 C - Includes: Items 13-19, Instruction Sheet and Bag (not shown).
 D - Throttle Pot is included with this assembly.