


Power Articulating Legrest ELRPW/ELRPW22 Harnesses and Hardware For ASBA Seat (Before 7/5/05)

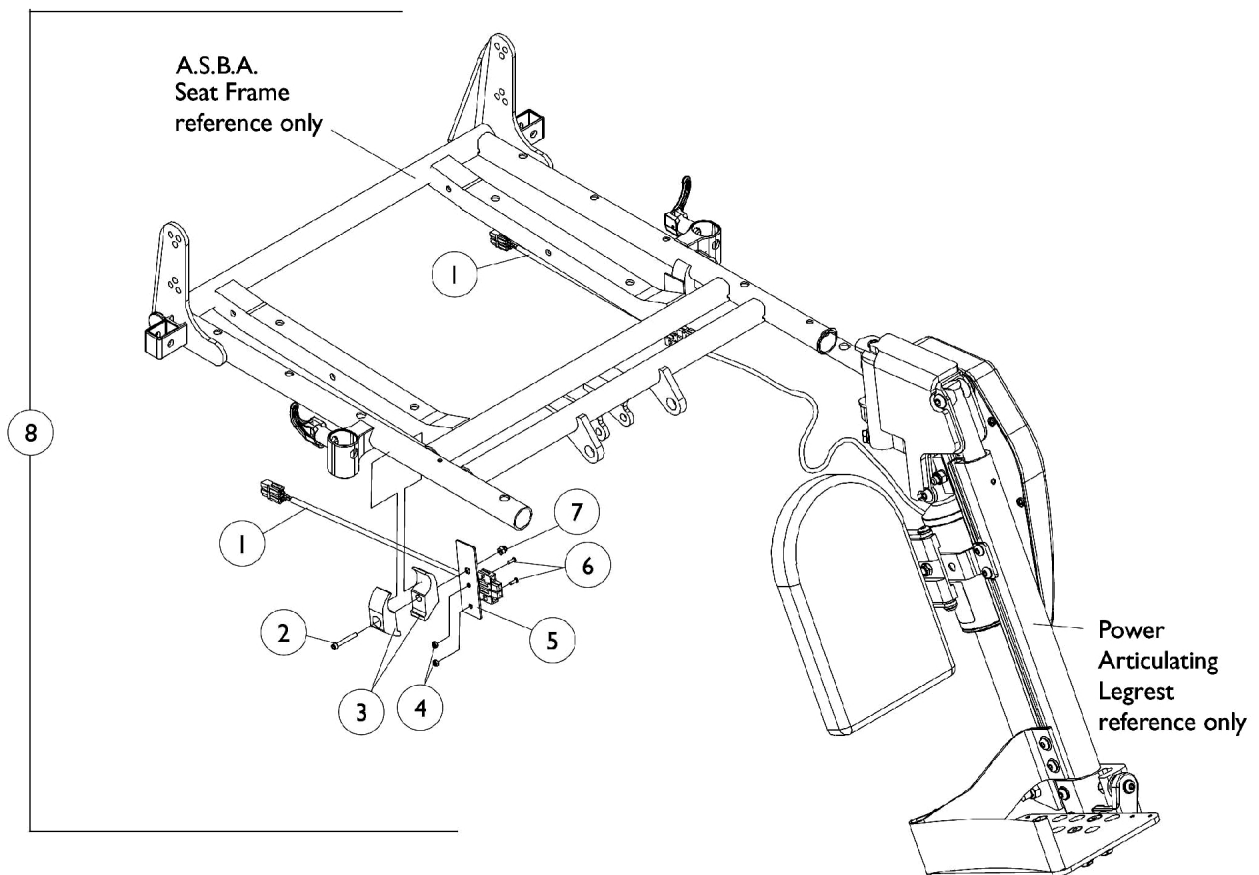

Power Articulating Legrest ELRPW/ELRPW22 Harnesses and Hardware For ASBA Seat (Before 7/5/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1101400	Harness, ELRPW Power Legrest - Right	1	1
1		1101401	Harness, ELRPW Power Legrest - Left	1	1
2		1003424	Screw, Socket Head (1/4-20 x 1-3/4")	1	2
3		1018692-NLA	NLA - Clamp, Half, Non-Threaded	1	4
4		1056098	Locknut (#8-32)	1	4
5		1102022-NLA	NLA - Bracket, Harness Wire, Black	1	2
6		1091871-NLA	NLA - Screw, Socket Head (#8-32 x 5/8")	1	4
7		7930338-NLA	NLA - Nut, Acorn (1/4-20)	25	2
8	A	1101520-NLA	NLA - Kit, Power Leg Harness and Mounting Hardware	2	1
	B	1104253-NLA	NLA - Tie Wrap (8" L)	10	1
NOTE: A - Includes items 1-7 B - Not Shown					