
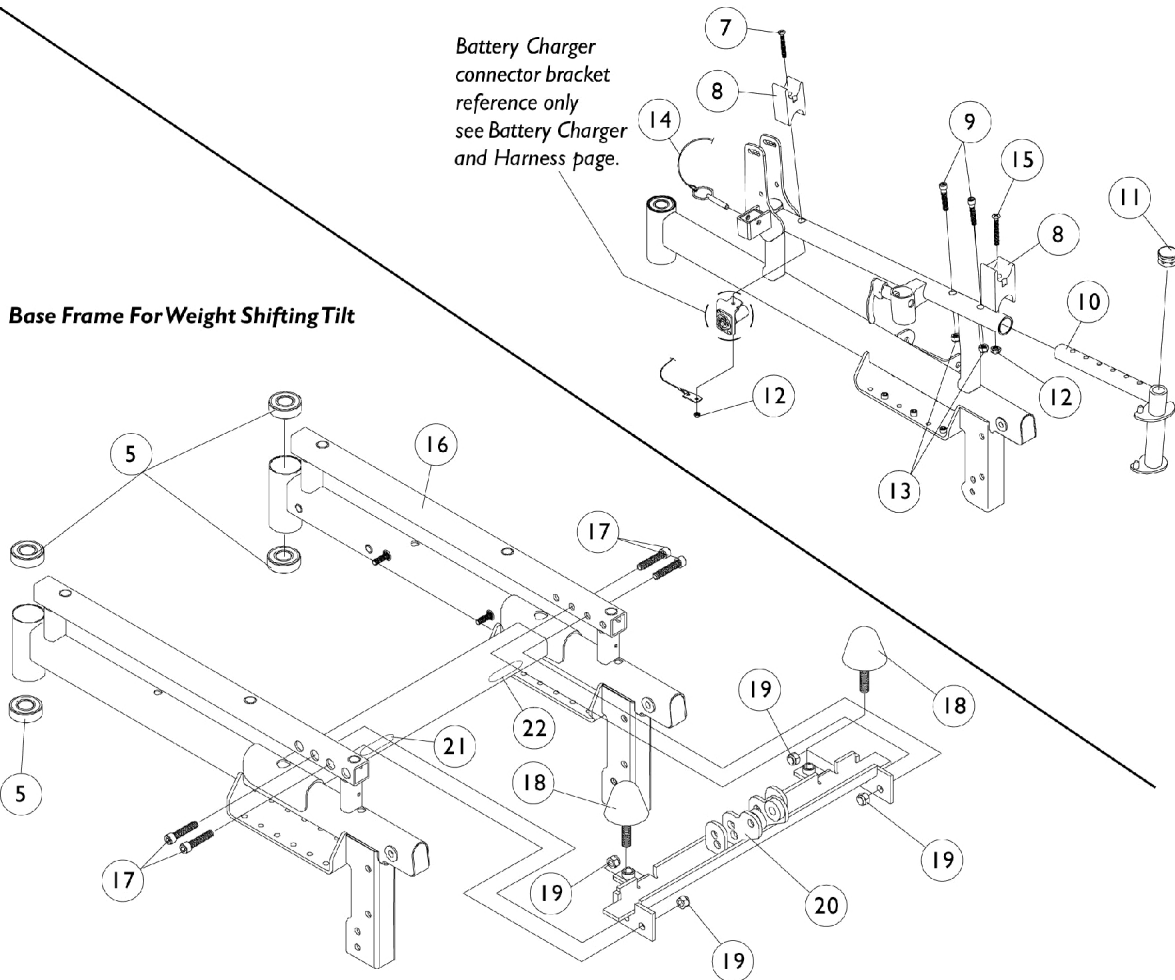


# Frames and Hardware

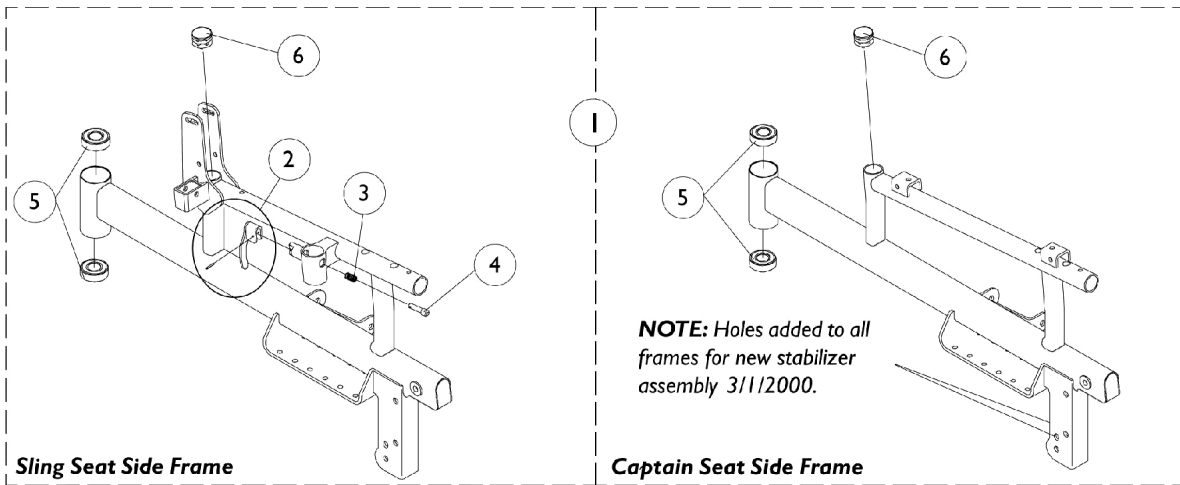

## Frames and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881071541-NL A	NLA - Frame, Side with Bearings, Sling Seat, Color - Right	1	1
1	A,B	8881071542-NL A	NLA - Frame, Side with Bearings, Sling Seat, Color - Left	1	1
1	A,C	8881074147-NL A	NLA - Frame, Side with Bearings, Captain Seat, Color - Right	1	1
1	A,C	8881074148-NL A	NLA - Frame, Side with Bearings, Captain Seat, Color - Left	1	1
1	C,E	1081016-NLA	NLA - Side Frame Assembly, Captain Seat - Right	1	1
1	C,E	1081017-NLA	NLA - Side Frame Assembly, Captain Seat - Left	1	1
2		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
3		47471X074	Spring, Arm Lock	1	1
4		1028594	Pin, Arm Lock, Black	1	1
5		40117X006	Bearing (5/8")	1	2
6		1029501	Plug Button, Black (1")	1	1
7	D	1085124-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1-3/4")	1	2
8		1007541	Clamp, Seat	1	2
9		7930304-NLA	NLA - Screw, Socket Head (1/4-20 x 1-1/4")	25	1
10		1060099-NLA	NLA - Hanger, Front Rigging, Black - Right	1	1
10		1060100-NLA	NLA - Hanger, Front Rigging, Black - Left	1	1
11		1027097	Plug Button, Black (3/4")	1	1
12		1025731	Locknut (#10-32)	1	2
13		7930333	Locknut (1/4-20)	25	1
14	D	1068039	Pin, Pull	1	1
15		1085134-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1-5/8")	1	1
16	A,B,F	1079192-NLA	NLA - Base Frame with Bearings	1	1
17		1021344	Screw, Socket Head (5/16-18 x 1-1/2")	1	4
18		0225100-NLA	NLA - Bumper, Tilt	1	2
19		1026158	Locknut (5/16-18)	1	4
20		1098615-NLA	NLA - Bracket, Link w/ Caution Decal, Black	1	1
21		1035900	Decal, Transportation	1	1
22		60106X013-NLA	NLA - Plate, Name, Patent Numbers	1	1

NOTE: A - Decals not included with frames. See page titled, "Packaging, Literature and Decals."  
 B - Specify frame color when ordering. See page titled, "Frame Finishes, Touch-Up Paint and Upholstery Options."  
 C - Captain seat side frames come in 24P wet black only.  
 D - Screws changed 9/16/99 and serve as a direct replacement for chairs manufactured before 9/16/99.  
 E - Captain Seat Side Frames with serial numbers 98F39748 - 98F39877 must use #1081016 and/or #1081017.  
 F - Weight Shifting Tilt.

NOTE: Sling seats with shrouds and Weight Shift Tilt seat frame with shrouds come in 24P wet black only.