


Crossbraces and Hardware - Before 7/1/99 Captain Seat

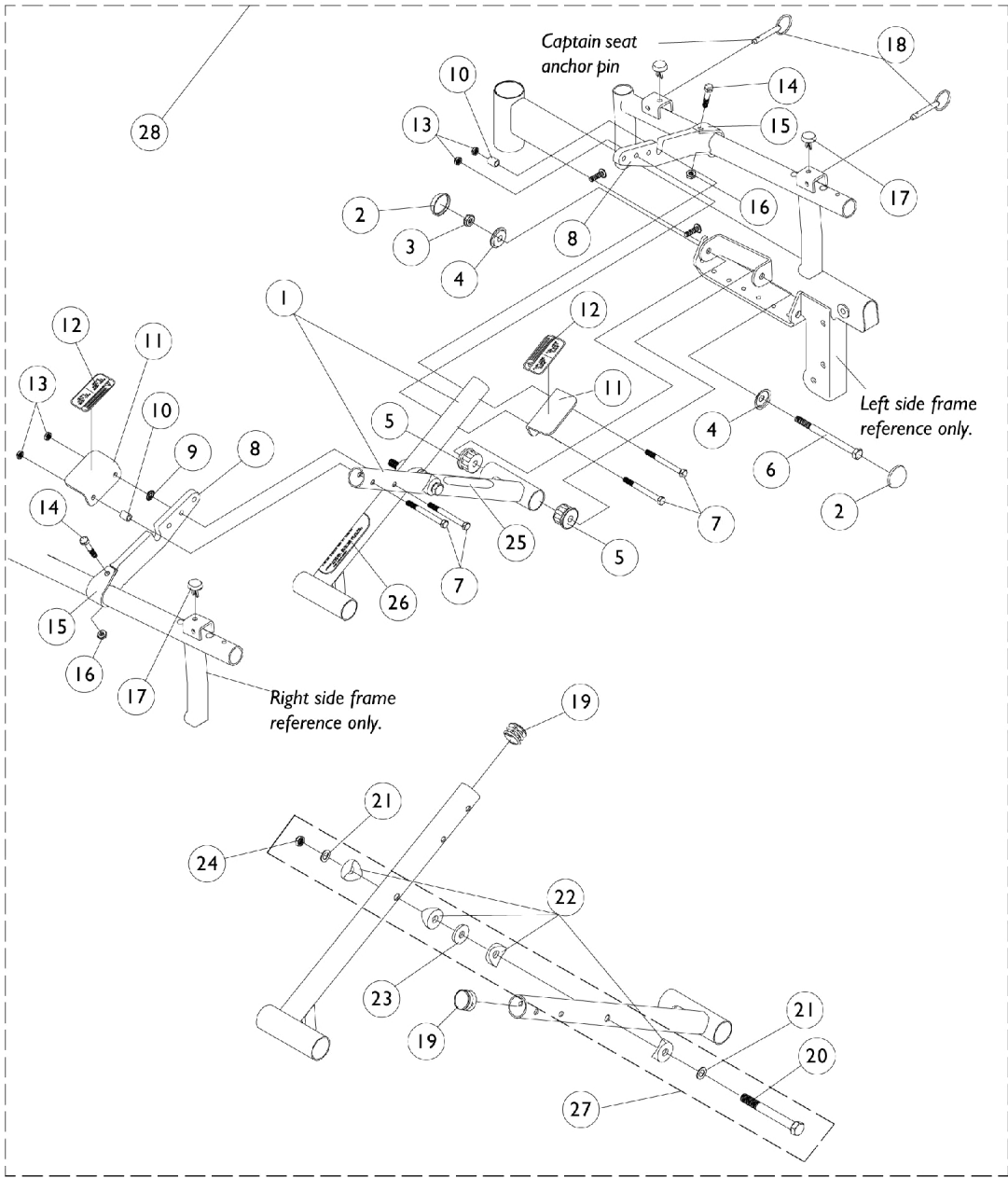

Crossbraces and Hardware - Before 7/1/99 Captain Seat

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1071586-NLA	NLA - Crossbrace, Captain Seat (18" W)	1	2
2		1028835-NLA	NLA - Cap, Hub, Black	1	4
3		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
4		1028491-NLA	NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	4
5		1056708-NLA	NLA - Plug, Pivot Tube	1	4
6		1038179-NLA	NLA - Screw, Hex Head (5/16-24 x 4-1/4")	1	2
2-6	B,C,F	1057830-NLA	NLA - Hardware, Crossbrace Pivot Tube	2	1
7		1024796-NLA	NLA - Screw, Hex Head (1/4-20 x 2")	1	4
8	A	1074109-NLA	NLA - Link, Pivot	1	2
9		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
10	A	1071583-NLA	NLA - Spacer (1/4 x 3/8 x 15/32")	1	2
11	A	1074141-NLA	NLA - Bracket, Push, Black	1	2
12		1074142-NLA	NLA - Decal, Crossbrace	1	2
13		7930333	Locknut (1/4-20)	25	1
14		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	2
15		1057828-NLA	NLA - Strap, Pivot Link	1	2
16		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
17		1071604-NLA	NLA - Bumper, Nylon	1	4
18		1071602-NLA	NLA - Pin, Anchor (Captain Seat)	1	4
19		1029501	Plug Button, Black (1")	1	4
20		1025705-NLA	NLA - Screw, Hex Head (5/16-24 x 3-1/4")	1	2
21		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
22		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
23		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
24		7930330-NLA	NLA - Locknut (5/16-24)	25	1
25		1035900	Decal, Transportation	1	1
26		1119281-NLA	NLA - Plate, Name, Patent Numbers	1	1
27	E	1086180-NLA	NLA - Hardware, Crossbrace Center Bolt Attaching	1	1
28	A,D	1074118-NLA	NLA - Hardware, Crossbrace	1	1

NOTE: A - Chairs manufactured before 7/1/99 need to be assembled with the crossbrace upgrade kit #1086801. Upgrade kit components can be found on Crossbraces and Hardware page dated after 6/30/99
 B - Not Shown.
 C - Includes items 2 through 6.
 D - Less items 1 and 8. Includes items 2-7 and 9-26.
 E - Includes items 20-24.
 F - Pair