



Crossbraces and Hardware Less Seat Rail

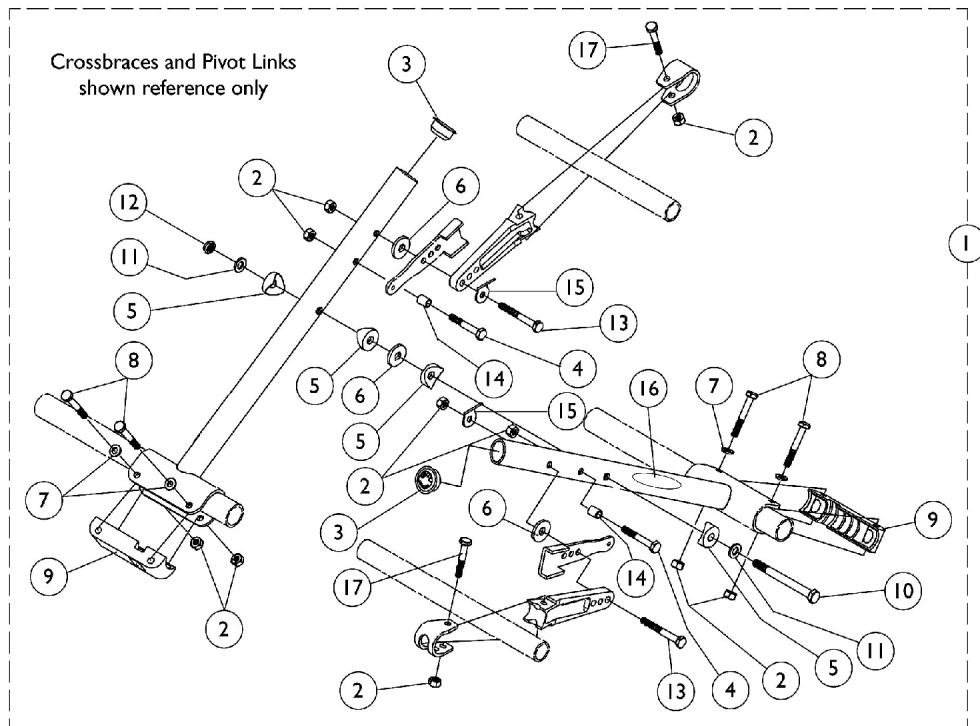

Crossbrace less Seat Rail Hardware

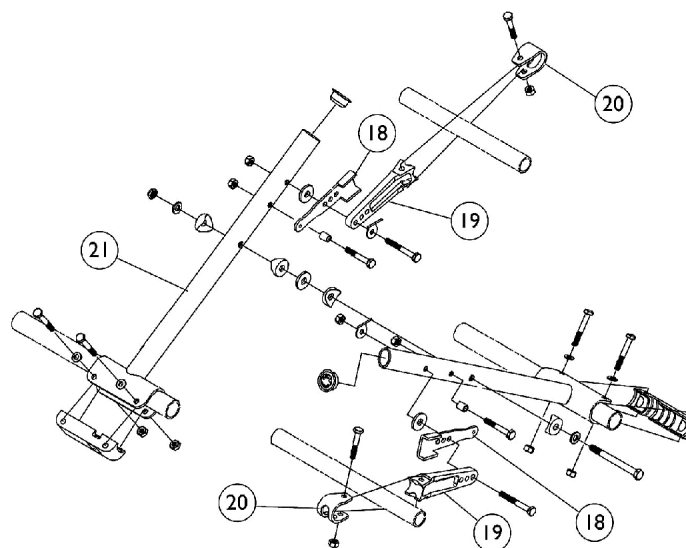

Crossbraces and Pivot Links

Crossbraces and Hardware Less Seat Rail

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1063207-NLA	NLA - Kit, Crossbrace Less Seat Rail Hardware	1	1
2		1065362	Locknut (1/4-28)	1	8
3		1056328	Plug Button, Black (1")	1	4
4		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	2
5		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
6		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	3
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
8		1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	4
9		1012172-NLA	NLA - Clamp, Saddle	1	2
10		1025705-NLA	NLA - Screw, Hex Head (5/16-24 x 3-1/4")	1	1
11		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
12		1025730	Locknut (5/16-24)	1	1
13		1055349	Screw, Hex Head (1/4-28 x 2")	1	2
14		1026162	Spacer (1/4 x 3/8 x 11/32")	1	2
15		1026903-NLA	NLA - Tab, Tie Down	1	2
16		1035900	Decal, Transportation	1	1
17		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
	B	1141633	Cord, Nylon (6-1/2 ft.)	1	1
	B	1029412	Lock, Cord, Black	1	1
18	G	1063668-NLA	NLA - Stop, Pivot Link, Black	1	2
18	C	1063666-NLA	NLA - Stop, Pivot Link, Black	1	2
18	F	1063667-NLA	NLA - Stop, Pivot Link, Black	1	2
19	G	1128752-NLA	NLA - Link, Pivot, Black (12 - 13" W)	1	2
19	C	1126169	NLA - Link, Pivot, Black (14 - 16" W)	1	2
19	D	1124938-NLA	NLA - Link, Pivot, Black (17 - 19" W)	1	2
19	E	1126170	Link, Pivot, Black (20 - 22" W)	1	2
20	G	1012180	Bracket, Pivot, Black	1	2
20	H	1124937-NLA	NLA - Bracket, Pivot, Black	1	2
21		See Chart	#21 Crossbraces Less Seat Rail	1	1
NOTE: A - Includes items 2-17 plus not shown items Nylon Cord and Cord Lock. B - Not Shown C - 14-16" wide D - 17-19" wide E - 20" wide F - 17-20" wide G - 12-13" wide H - 14-20" wide					

Crossbraces and Hardware

Less Seat Rail

#21 Crossbraces Less Seat Rail

Chair Width	12"	13"	14"	15"	16"
Right/Left	1063200-NLA	1063201-NLA	1063202-NLA	1063203-NLA	1063204
Chair Width	17"	18"	19"	20"	
Right/Left	1063205	1063206	1127126-NLA	1127127-NLA	