
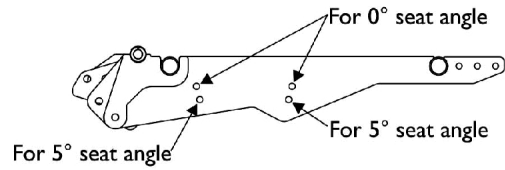




# Seat Frame Mounting Hardware

## Recline Only

### For Conventional and Contoura Seating

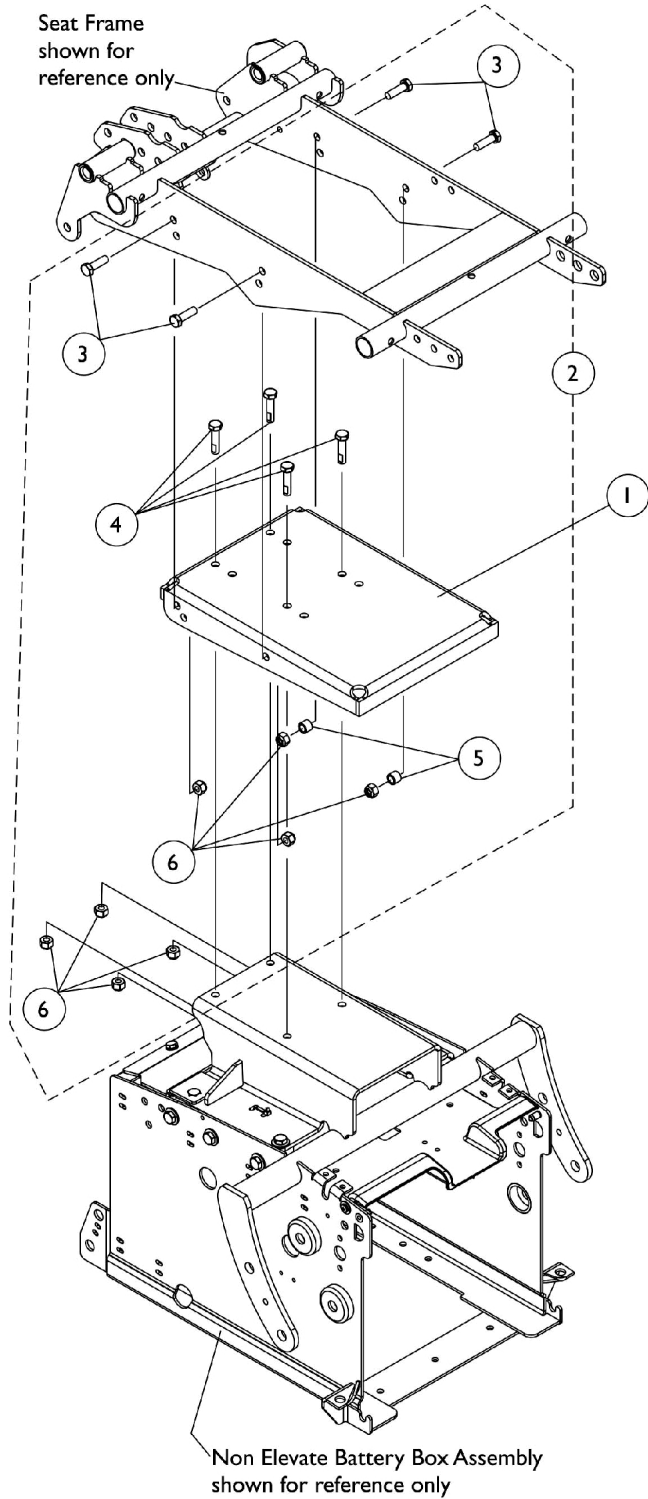

**Seat Frame Mounting Detail**

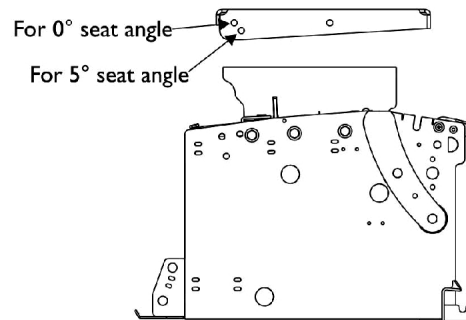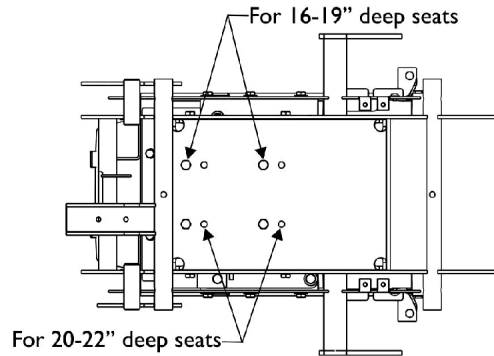

**Recline Only Mounting Bracket Detail**

## Seat Frame Mounting Hardware Recline Only For Conventional and Contoura Seating

| Item Number                  | Note | Part Number | Description   | Quantity Per |          |
|------------------------------|------|-------------|---|--------------|----------|
|                              |      |             |   | Package      | Assembly |
| 1                            | A    | 1125646-NLA | NLA - Plate, Recline Only Seat Frame Mounting               | 1            | 1        |
| 2                            |      | 1125751     | Kit, Seat Frame Mounting Hardware, Recline Only Non-Elevate | 1            | 1        |
| 3                            |      |             | Screw, Hex Head (5/16-18 x 1")                              | 1            | 4        |
| 4                            |      |             | Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")                 | 1            | 4        |
| 5                            |      |             | Spacer (5/16 x 7/16 x 3/8")                                 | 1            | 2        |
| 6                            |      |             | Locknut (5/16-18)   | 1            | 8        |
| NOTE: A - Includes items 3-6 |      |             |   |              |          |