


Shroud Covers - After 2/29/00

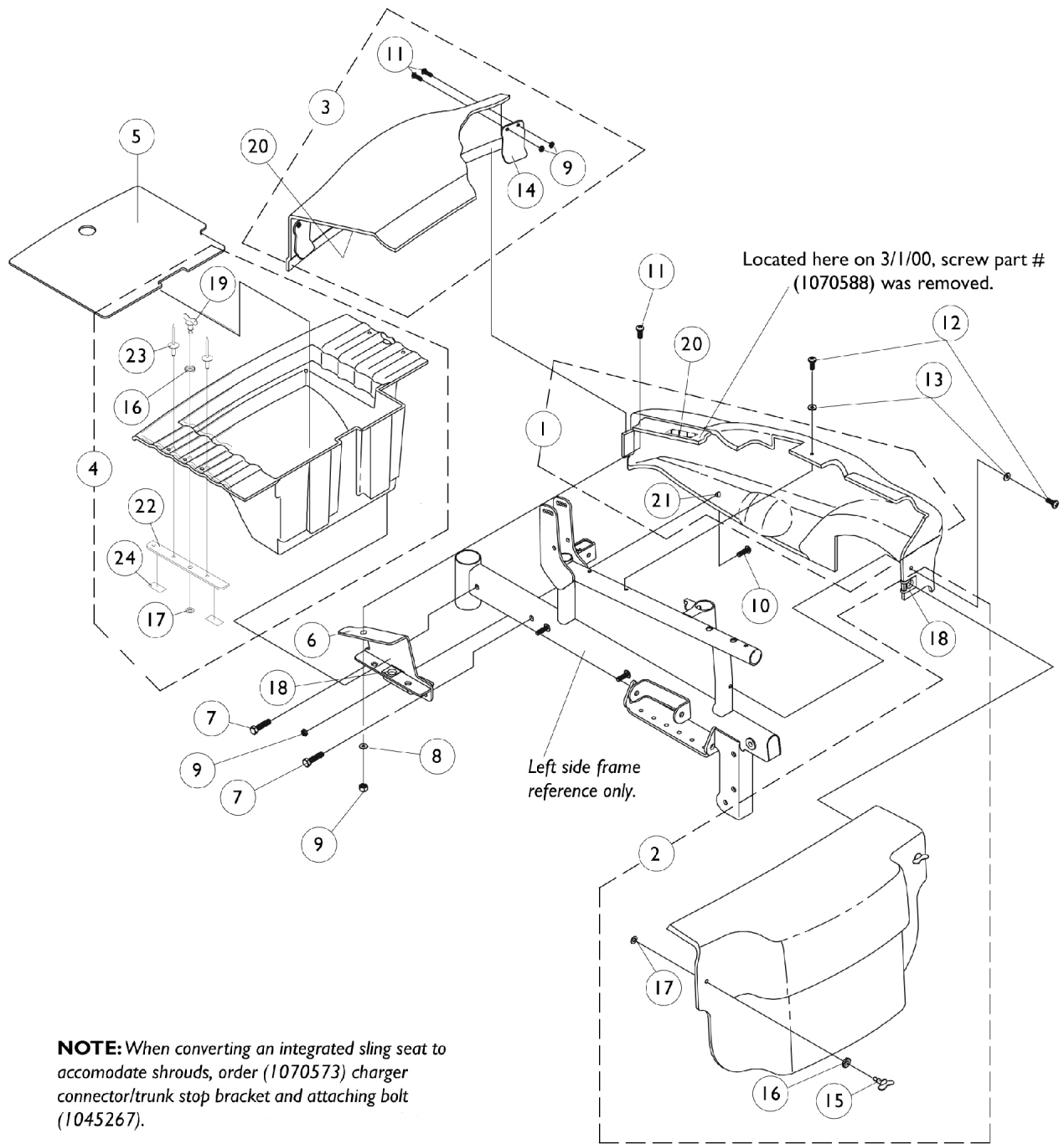

Shroud Covers - After 2/29/00

Item #'s 1-5 Shroud Covers After 2/29/00

Item Number	Description	Ocean Blue	Emerald Green	Liquid Black	Candy Apple Red
1	Side Panel (Captain) - Right	1102266-NLA See Note(s) D, A, B	1102267-NLA See Note(s) D, B, A	1102265-NLA See Note(s) A, B, D	1102264-NLA See Note(s) A, D, B
1	Side Panel (Captain) - Left	1102270-NLA See Note(s) A, D, B	1102271-NLA See Note(s) D, A, B	1102269-NLA See Note(s) B, A, D	1102268-NLA See Note(s) D, A, B
1	Side Panel (Sling Seat) Right See Note(s) B	1102274-NLA See Note(s) A, B, D	1102275-NLA See Note(s) A, B, D	1102273-NLA See Note(s) A, B, D	1102272-NLA See Note(s) A, B, D
1	Side Panel (Sling Seat) Left	1102278-NLA See Note(s) A, B, D	1102279-NLA See Note(s) A, B, D	1102277-NLA See Note(s) A, B, D	1102276-NLA See Note(s) A, B, D
2	Battery Cover - 16" Wide	1094737-NLA See Note(s) C	1094738-NLA See Note(s) C	1094736-NLA See Note(s) C	1094735-NLA See Note(s) C
2	Battery Cover - 18" Wide	1094733-NLA See Note(s) C	1094734-NLA See Note(s) C	1094732-NLA See Note(s) C	1094731-NLA See Note(s) C
2	Battery Cover - 20" Wide	1094729-NLA See Note(s) C	1094730-NLA See Note(s) C	1094728-NLA See Note(s) C	1094727-NLA See Note(s) C
3	Trunk Lid - 16" Wide	1102262-NLA See Note(s) B, C	1102263-NLA See Note(s) B, C	1102261-NLA See Note(s) B, C	1102260-NLA See Note(s) B, C
3	Trunk Lid - 18" Wide	1102258-NLA See Note(s) B, C	1102259-NLA See Note(s) C, B	1102257-NLA See Note(s) C, B	1102256-NLA See Note(s) B, C
3	Trunk Lid - 20" Wide	1102254-NLA See Note(s) C, B	1102255-NLA See Note(s) C, B	1102253-NLA See Note(s) C, B	1102252-NLA See Note(s) B, C
4	Rear Storage Compartment - 16" Wide	1070562-NLA See Note(s) C	1070562-NLA See Note(s) C	1070562-NLA See Note(s) C	1070562-NLA See Note(s) C
4	Rear Storage Compartment - 18" Wide	1070561-NLA See Note(s) C	1070561-NLA See Note(s) C	1070561-NLA See Note(s) C	1070561-NLA See Note(s) C
4	Rear Storage Compartment - 20" Wide	1070560-NLA See Note(s) C	1070560-NLA See Note(s) C	1070560-NLA See Note(s) C	1070560-NLA See Note(s) C
5	Rear Storage Compartment, Shelf-Black	1071655-NLA	1071655-NLA	1071655-NLA	1071655-NLA

NOTE: A - Decals not included. See page titled, "Packaging, Literature and Decals."
 B - Dual lock fasteners and spacers are not included with shrouds. See item 20.
 C - Captain seat shrouds on 18" wide only.
 D - Side Panel includes Rubber Bushing.

Shroud Covers - After 2/29/00

Color Conversion Package, Captain Seat

Ocean Blue	Emerald Green	Liquid Black	Candy Apple Red
1096875-NLA	1096876-NLA	1096874-NLA	1096873-NLA
See Note(s) A, B, C	See Note(s) A, C, B	See Note(s) A, B, C	See Note(s) B, A, C
NOTE: A - Decals not included. See page titled, "Packaging, Literature and Decals." B - Not Shown. C - Includes items 1,3,8,9,10,11,12 and 13.			

Color Conversion Package, Sling Seat

Width	Ocean Blue	Emerald Green	Liquid Black	Candy Apple Red
16" Wide	1096883-NLA	1096884-NLA	1096882-NLA	1096881-NLA
	See Note(s) C, B, A	See Note(s) B, C, A	See Note(s) A, B, C	See Note(s) B, C, A
18" Wide	1096879-NLA	1096880-NLA	1096878-NLA	1096877-NLA
	See Note(s) A, B, C	See Note(s) B, C, A	See Note(s) C, B, A	See Note(s) C, B, A
20" Wide	1096887-NLA	1096888-NLA	1096886-NLA	1096885-NLA
	See Note(s) C, B, A	See Note(s) A, B, C	See Note(s) C, A, B	See Note(s) C, A, B
NOTE: A - Decals not included. See page titled, "Packaging, Literature and Decals." B - Not Shown. C - Includes items 1, 3, 8, 9, 10, 11, 12 and 13.				

Shroud Covers - After 2/29/00

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-5		See Chart	Item #'s 1-5 Shroud Covers After 2/29/00	1	1
	B	See Chart	Color Conversion Package, Captain Seat	1	1
	B	See Chart	Color Conversion Package, Sling Seat	1	1
6		1070565-NLA	NLA - Bracket, Shroud, Black - Right	1	1
6		1070566-NLA	NLA - Bracket, Shroud, Black - Left	1	1
7		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	4
8		1028516	Washer, Steel (3/16 x 7/16 x 1/16")	1	2
9		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
10		1070575-NLA	NLA - Screw, SEMS w/ Washer (#10-32 x 7/8")	1	2
11		1070588-NLA	NLA - Screw, Phillips Truss Head (#10-32 x 1/2")	1	6
12		1028755	Screw, Phillips Pan Head (#10-32 x 5/8")	1	4
13		1070587	Washer, Flat (3/16 x 5/8 x 1/16")	1	4
	B,C	1070564-NLA	NLA - Hardware, Side Panel Attaching	1	1
14		1070577-NLA	NLA - Plate, Trunk Clip	1	2
15		1070583-NLA	NLA - Stud, Wing Head (3/4" Long)	1	2
16		1070581-NLA	NLA - Washer, Nylon Cupped (9/32 x 19/32 x 3/32")	1	4
17		1070580-NLA	NLA - Washer, Nylon Retainer (1/4 x 1/2 x 1/32")	1	4
18		1013497-NLA	NLA - Fastener, Turn, Clip-On (1/4")	1	4
19		1070574-NLA	NLA - Stud, Wing Head (1" Long)	1	2
20	A	1097290	Kit, Dual Lock Fastener/ Spacer	1	1
21		1070594-NLA	NLA - Bushing, Rubber	1	2
22		1070563-NLA	NLA - Plate, Trunk	1	2
23		1070582-NLA	NLA - Rivet	1	4
24		1062096-NLA	NLA - Fastener, Loop (1/2" W x 1" L)	1	4
	B	1102292-NLA	NLA - Instruction Sheet, Dual Lock Fastener/ Spacer Kit	1	1

NOTE: A - Kit includes 2 ea. dual lock fasteners (5/8" W x 2" L) and (1" W x 2" L), 4 ea. dual lock spacers and instruction sheet.
 B - Not Shown.
 C - Includes items 6-13.