


Shroud Covers

Weight Shifting Tilt

Before 3/1/00

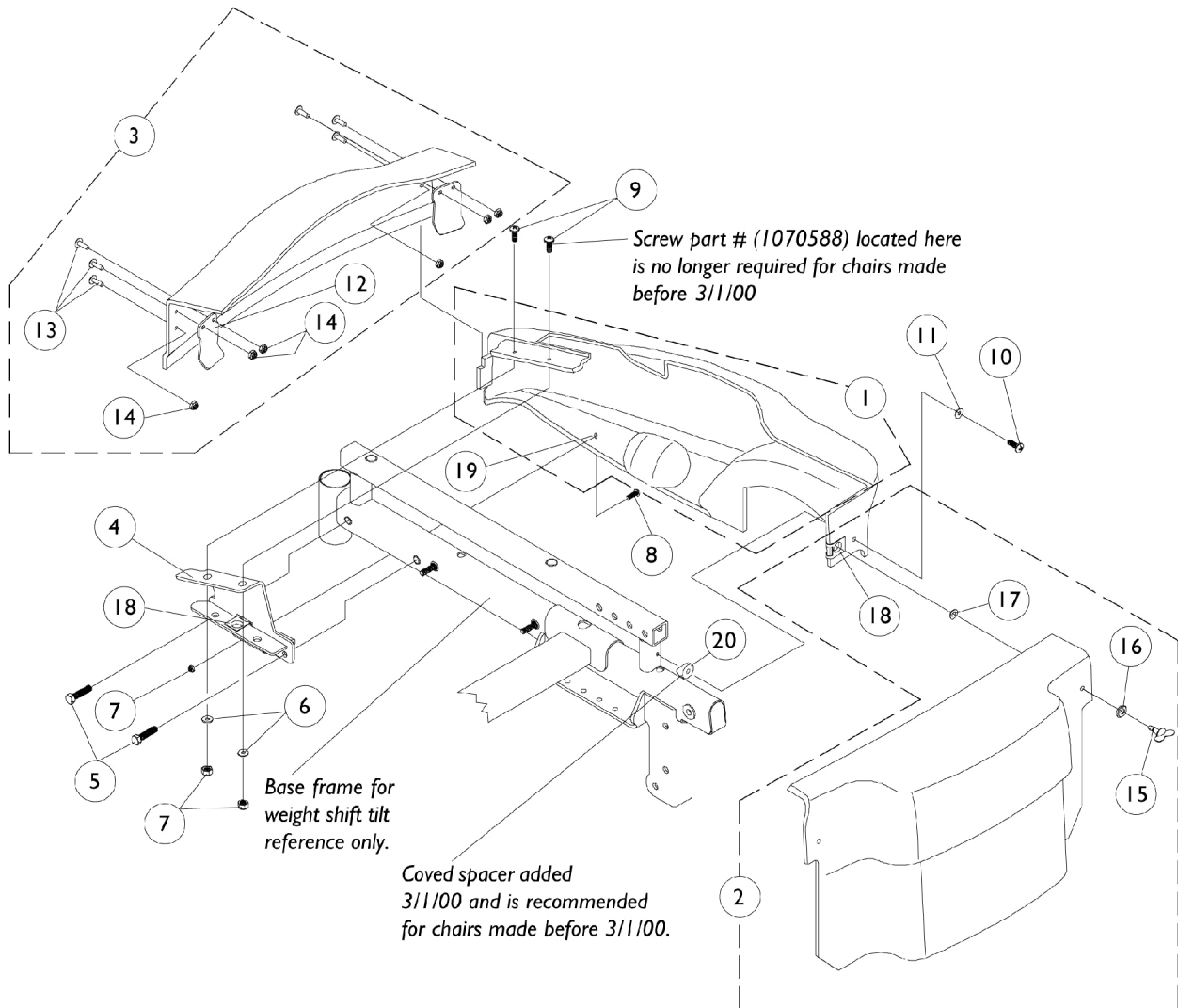

Shroud Covers

Weight Shifting Tilt

Before 3/1/00

Item #'s 1-3 Shroud Covers Before 3/1/00 Weight Shifting Tilt

Item Number	Description	Bright Plum Metallic	Bright Blue Metallic	Blue Green Chameleon	Candy Apple Red
1	Side Panel - Right	1080685 See Note(s) B, A	1080686-NLA See Note(s) B, A	1080687-NLA See Note(s) C, A, B	1085100-NLA See Note(s) A, C, B
1	Side Panel - Left	1080688-NLA See Note(s) A, B	1080689-NLA See Note(s) A, B	1080690-NLA See Note(s) A, C, B	1085101-NLA See Note(s) A, B, C
2	Battery Cover	1080694-NLA	1080695-NLA	1080696-NLA	1085103-NLA See Note(s) C
3	Trunk Lid	1080691-NLA	1080692-NLA	1080693-NLA	1085102-NLA See Note(s) C

NOTE: A - Decals not included. See page titled, "Packaging, Literature and Decals."
 B - Side Panel includes Rubber Bushing.
 C - Shrouds made before 3/1/00 are NLA. See shrouds made after 2/29/00 and select color conversion package.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-3		See Chart	Item #'s 1-3 Shroud Covers Before 3/1/00 Weight Shifting Tilt	1	1
4		1079197-NLA	NLA - Bracket, Side Shroud, Black - Right	1	1
4		1079198-NLA	NLA - Bracket, Side Shroud, Black - Left	1	1
5		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	4
6		1028516	Washer, Steel (3/16 x 7/16 x 1/16")	1	2
7		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
8		1070575-NLA	NLA - Screw, SEMS w/ Washer (#10-32 x 7/8")	1	2
9		1070588-NLA	NLA - Screw, Phillips Truss Head (#10-32 x 1/2")	1	2
10		1028755	Screw, Phillips Pan Head (#10-32 x 5/8")	1	2
11		1070587	Washer, Flat (3/16 x 5/8 x 1/16")	1	2
	A,B	1079196-NLA	NLA - Hardware, Side Panel Attaching	1	1
12		1070577-NLA	NLA - Plate, Trunk Clip	1	2
13		1070588-NLA	NLA - Screw, Phillips Truss Head (#10-32 x 1/2")	1	6
14		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
15		1070583-NLA	NLA - Stud, Wing Head (3/4" Long)	1	2
16		1070581-NLA	NLA - Washer, Nylon Cupped (9/32 x 19/32 x 3/32")	1	4
17		1070580-NLA	NLA - Washer, Nylon Retainer (1/4 x 1/2 x 1/32")	1	4
18		1013497-NLA	NLA - Fastener, Turn, Clip-On (1/4")	1	4
19		1070594-NLA	NLA - Bushing, Rubber	1	2
20		40157X019-NLA	NLA - Spacer, Coved (1/4 x 7/8 x 7/16")	1	2

NOTE: A - Not Shown.
 B - Includes items 4-11 and 20.