


Tilt Assembly Mounting Hardware For TDX

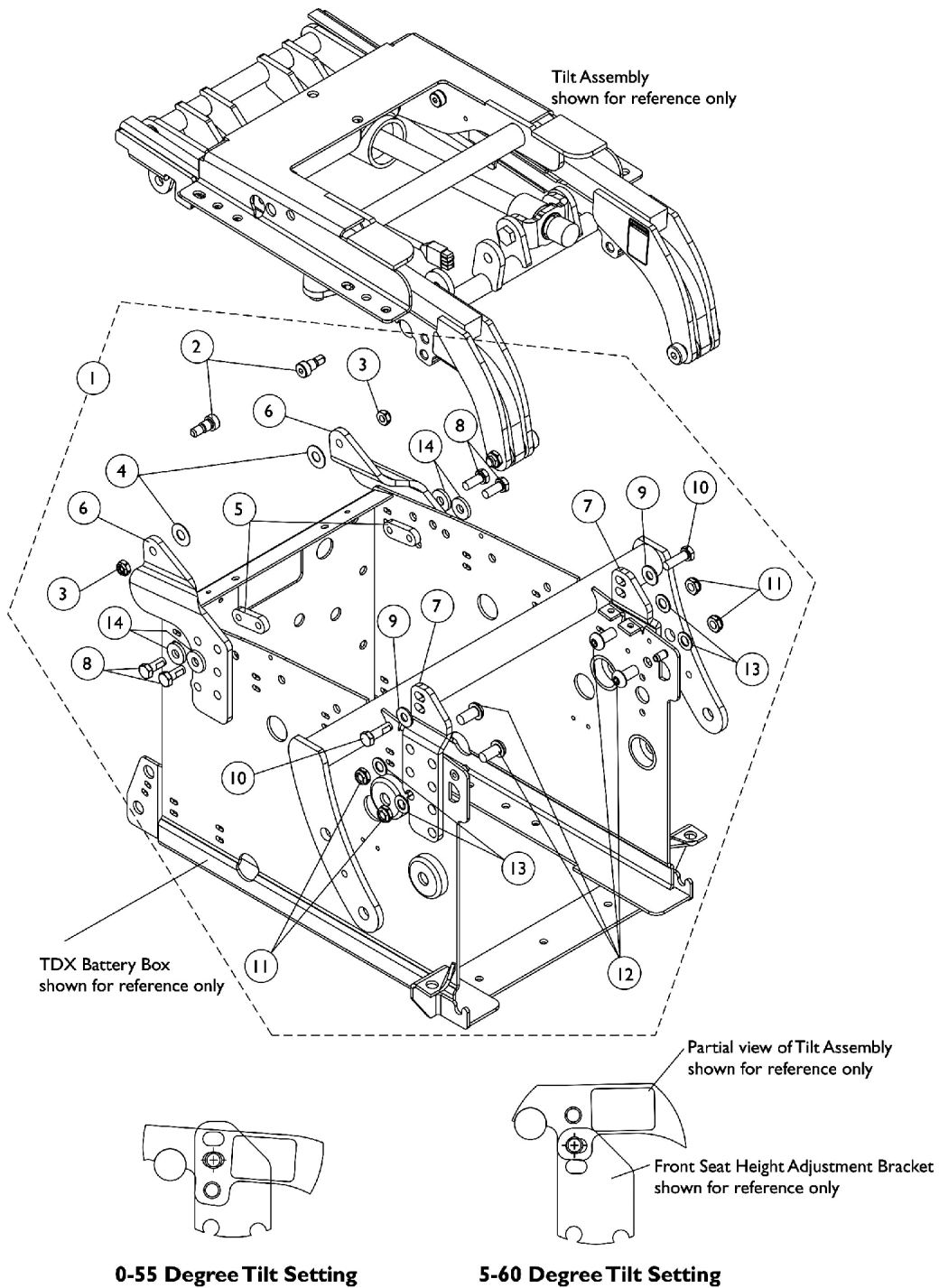

Tilt Assembly Mounting Hardware For TDX

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1126352-NLA	NLA - Kit, Tilt Assembly Mounting Hardware For TDX	1	1
2			Screw, Shoulder Hex Socket Head w/ Patch (3/8" x 3/8", 5/16-18)	1	2
3			Locknut (5/16-18)	1	2
4			NLA - Washer, Nylon (25/64 x 7/8 x 1/32")	1	2
5			NLA - Nut, Strap	1	2
6			NLA - Bracket, Seat Height Adjustment Rear, Black - Right	1	1
6			NLA - Bracket, Seat Height Adjustment Rear, Black - Left	1	1
7			NLA - Bracket, Seat Height Adjustment Front - Right	1	1
7			NLA - Bracket, Seat Height Adjustment Front, Black - Left	1	1
8			Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	4
9			Washer, Steel (5/16 x 3/4 x 1/16")	1	2
10			NLA - Screw, Hex Head w/ Patch (5/16-18 x 1")	1	2
11			Locknut, Low Profile (3/8-16)	1	4
12			NLA - Screw, Socket Button Head (3/8-16 x 3/4")	1	4
13			Washer, Flat (3/8 x 11/16 x 1/16")	1	4
14			Washer, Flat (21/64 x 53/64 x 7/64")	1	4

NOTE: A - Includes items 2-14