



Drive Wheels and Mounting Hardware (12-1/2" x 2-1/4") (For TDX3 w/ 2-Pole or 4-Pole Motors) & (For TDX3 SE w/ 4-Pole Motors)

NOTE: Items 4-7 wheel mounting hardware applies to both item #1 and item #8 12-1/2" drive wheel assemblies listed below.

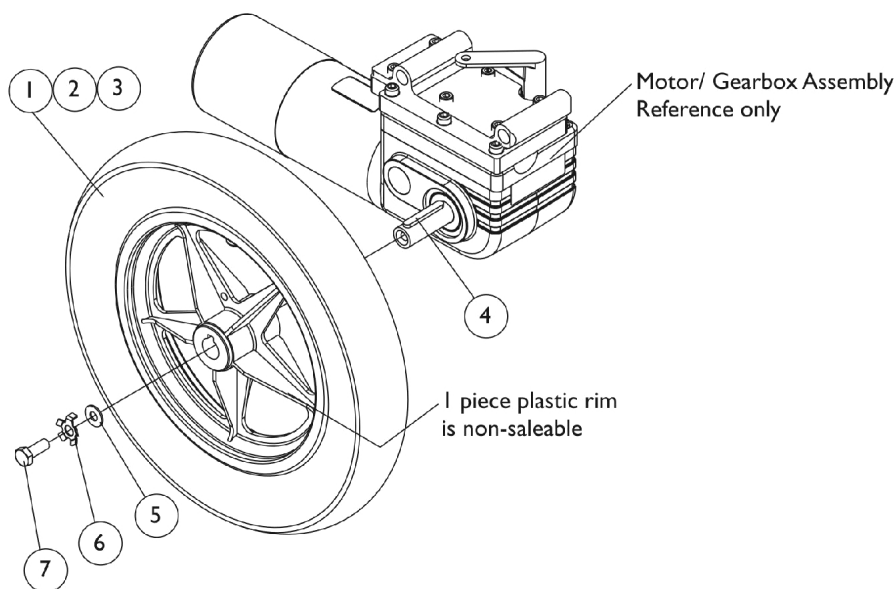

**12-1/2" x 2-1/4" Drive Wheel
For TDX 3 with a 2-Pole Motor
(Standard) Before 10/20/03**

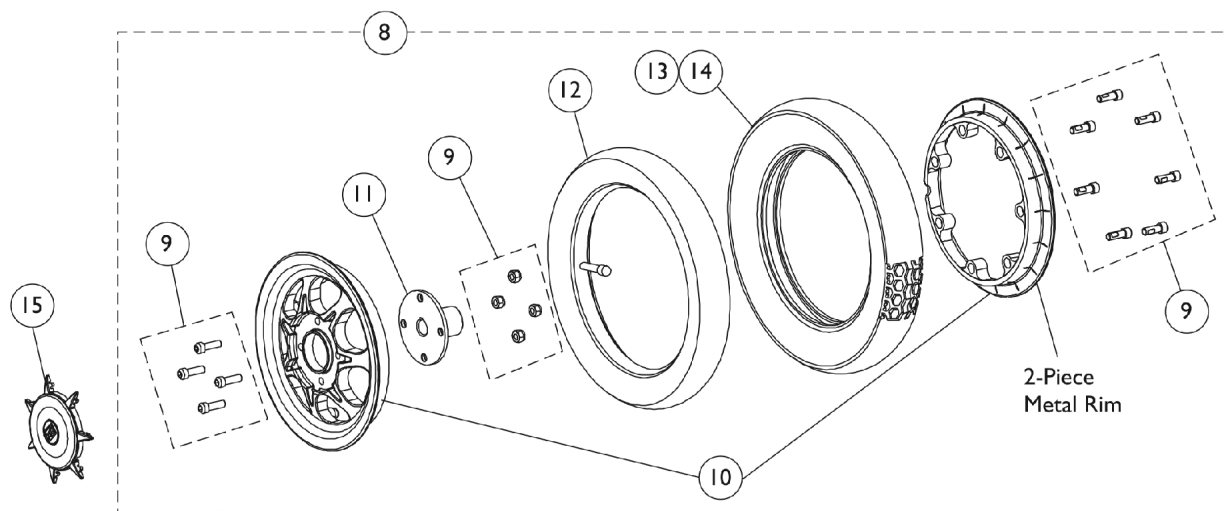

**12-1/2" x 2-1/4" Drive Wheel
For TDX3 with a 4-Pole Motor
(Standard) 10/18/03-10/8/06
For TDX3 SE with a 4-Pole Motor
(Standard) introduced 11/15/06**

Drive Wheels and Mounting Hardware (12-1/2" x 2-1/4") (For TDX3 w/ 2-Pole or 4-Pole Motors) & (For TDX3 SE w/ 4-Pole Motors)

NOTE: Items 4-7 wheel mounting hardware applies to both item #1 and item #8 12-1/2" drive wheel assemblies listed below.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1064178-NLA	NLA - Drive Wheel Assembly, 5 Spoke, Pneumatic (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
1	A	1064177	Drive Wheel Assembly, 5 Spoke w/ Foam Fill Insert (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
2		1011524	Tire, Pneumatic, Grey (12-1/2" x 2-1/4")	1	1
3		1011525	Tube, Inner, Black (12-1/2 x 2-1/4")	1	1
4		1023264	Key (6mm x 6mm x 40mm)	1	1
5		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	1
6		1064179	Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	2	1
7		1023260	Screw (M8 x 1.25 x 20mm)	1	1
8	B	1122836	Drive Wheel Assembly, 7 Spoke, Pneumatic (12-1/2 x 2-1/4")	1	1
8	C	1122838	Drive Wheel Assembly, 7 Spoke with Foam Fill Insert (12-1/2 x 2-1/4")	1	1
9		1123137	Drive Wheel Hardware, Hub and 2-Piece Rim (12-1/2 x 2-1/4") Consists of: 4 ea. Screw, Socket Head Cap (5/16-24 x 1") 4 ea. Locknut (5/16-24) 7 ea. Screw, Socket Head Cap (1/4-20 x 1")	1	1
10		1123138	Rim Assembly, Inner/Outer (12-1/2 x 2-1/4")	1	1
11		1122196	Hub, Wheel	1	1
12		1011525	Tube, Inner, Black (12-1/2 x 2-1/4")	1	1
13		1011524	Tire, Pneumatic, Grey (12-1/2" x 2-1/4")	1	1
14		1122837	Tire w/ Foam Fill Insert, Grey (12-1/2" x 2-1/4")	1	1
15		1122835	Cap, Hub, Spoke	1	1

NOTE: A - Standard for TDX3 with 2-Pole Motor before 10/20/03. The 1 piece 5 spoke plastic rim is a non-saleable item.
B - Includes items 9-13. (Standard for TDX3 with 4-Pole Motor after 10/19/03), (Standard for TDX3 SE with 4-Pole Motor introduced 11/15/06)
C - Includes items 9-11 and 14. (Standard for TDX3 with 4-Pole Motor after 10/19/03), (Standard for TDX3 SE with 4-Pole Motor introduced 11/15/06)

NOTE: Items 4-7 wheel mounting hardware applies to both item #1 and item #8 12-1/2" drive wheel assemblies listed.