


Adjustable Angle Rigidizing Stroller Handle - AASH (Before 5/30/08)

Stroller Handle Hardware (item 1) includes hardware only (items 2-6). If a complete Stroller Handle is needed then items 7 & 9 must be ordered as well.

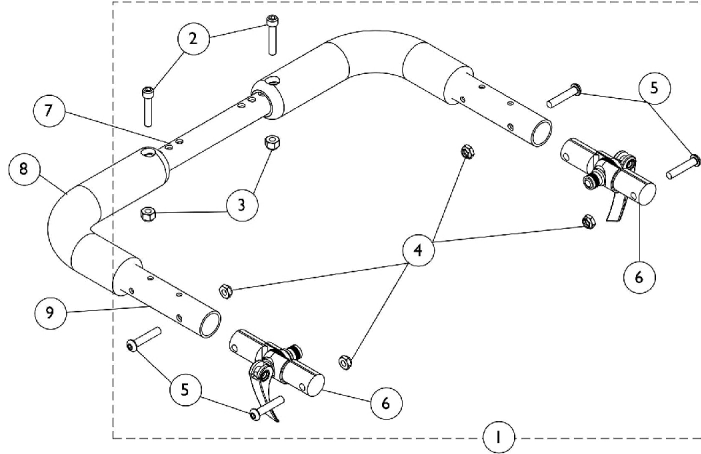

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1127790	NLA - Kit, Adjustable Angle Stroller Handle Hardware	1	1
2			Screw, Allen Head (1/4-28 x 1-1/4")	1	2
3			Locknut (1/4-28)	1	2
4			Locknut (1/4-28)	1	4
5			Screw, Socket Button Head (1/4-28 x 1-1/4")	1	4
6			Folding Hinge Assembly	1	2
7		See Chart	#7 Crossbrace Tube	1	1
8		1127767	Grip, Foam (9")	1	2
9	B	1127789	Stroller Handle Assembly - Right/Left	1	2

NOTE: A - Includes items 2-6
B - Includes item 8, Foam Grip

#7 Crossbrace Tube

12-16" Wide	14-18" Wide	16-20" Wide	18-22" Wide
1129473	1129474	1129475	1129476

Adjustable Angle Rigidizing Stroller Handle - AASH

(Before 5/30/08)

Stroller Handle Hardware (item 1) includes hardware only (items 2-6). If a complete Stroller Handle is needed then items 7 & 9 must be ordered as well.
