


# Articulating Legrest Support Assembly - ATL4A

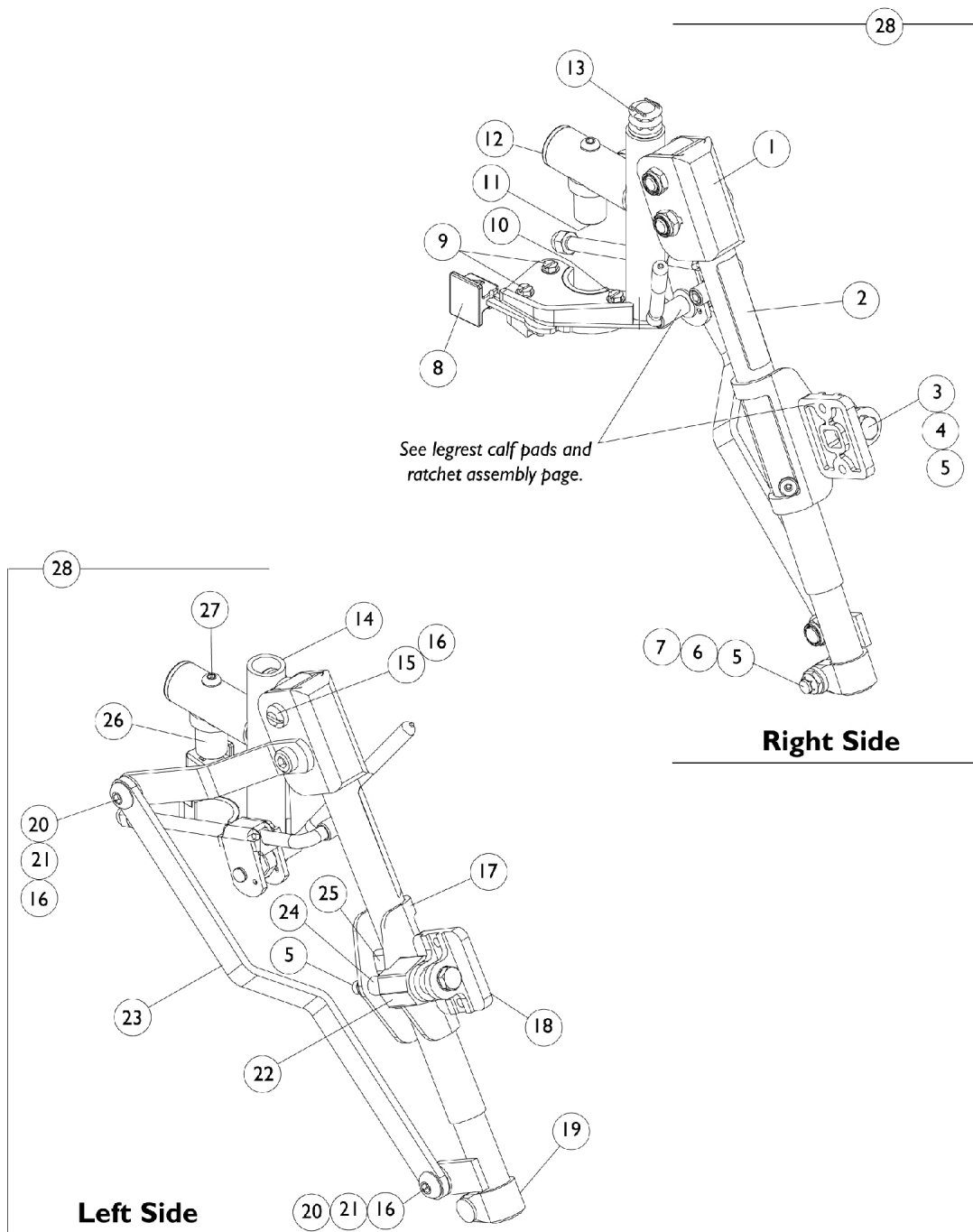

## Articulating Legrest Support Assembly - ATL4A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1022877-NLA	NLA - Tube, Support and Drive Link, Chrome - Right	1	1
1		1022878-NLA	NLA - Tube, Support and Drive Link, Chrome - Left	1	1
2		1024911-NLA	NLA - Decal, Smartleg	1	1
3		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
4		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
5		60264X001	Locknut (5/16-24)	1	2
6		1027377	Washer (#20)	1	1
7		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
8		1025454	Housing, Latch - Right	1	1
8		1025455	Housing, Latch - Left	1	1
9		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	2
10		1017423	Screw, Slotted Hex Washer Head w/ Patch (#10-32 x 1/2")	1	1
11		1025736	Locknut (1/4-20)	1	1
12		1029501	Plug Button, Black (1")	1	1
13		1027097	Plug Button, Black (3/4")	1	1
14		1082538-NLA	NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Right	1	1
14		1082539-NLA	NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Left	1	1
15		1022854-NLA	NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16)	1	1
16		40363X000	Hex Nut (3/8"-16)	1	3
17		1025104	Bracket, Legrest, Chrome	1	1
18		04526X019	Plate, Legrest Pad	1	1
19		1022848	Tube, Slide, Articulating, Chrome - Right	1	1
19		1022849	Tube, Slide, Articulating, Chrome - Left	1	1
20		1022866-NLA	NLA - Screw, Socket Button Head (3/8-16 x 3/4")	1	2
21		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
22		1025133	Spacer, Legrest Pad, Black	1	1
23		1022846-NLA	NLA - Bar, Link, Black	1	1
24		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
25		48823X008	Spring, Leaf	1	1
26		1017148	Pivot, Footrest	1	1
27		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	1
28	A	1082540	Articulating Legrest Support Assembly, Sector, Chrome - Right	1	1
28	A	1082541	Articulating Legrest Support Assembly, Sector, Chrome - Left	1	1

NOTE: A - Includes Items 1-27