


Back Canes and Hardware

Fold Down Adjustable Height Angle Back

Fold Down Back
optional on models
R50LX/R51LX/R51LXP

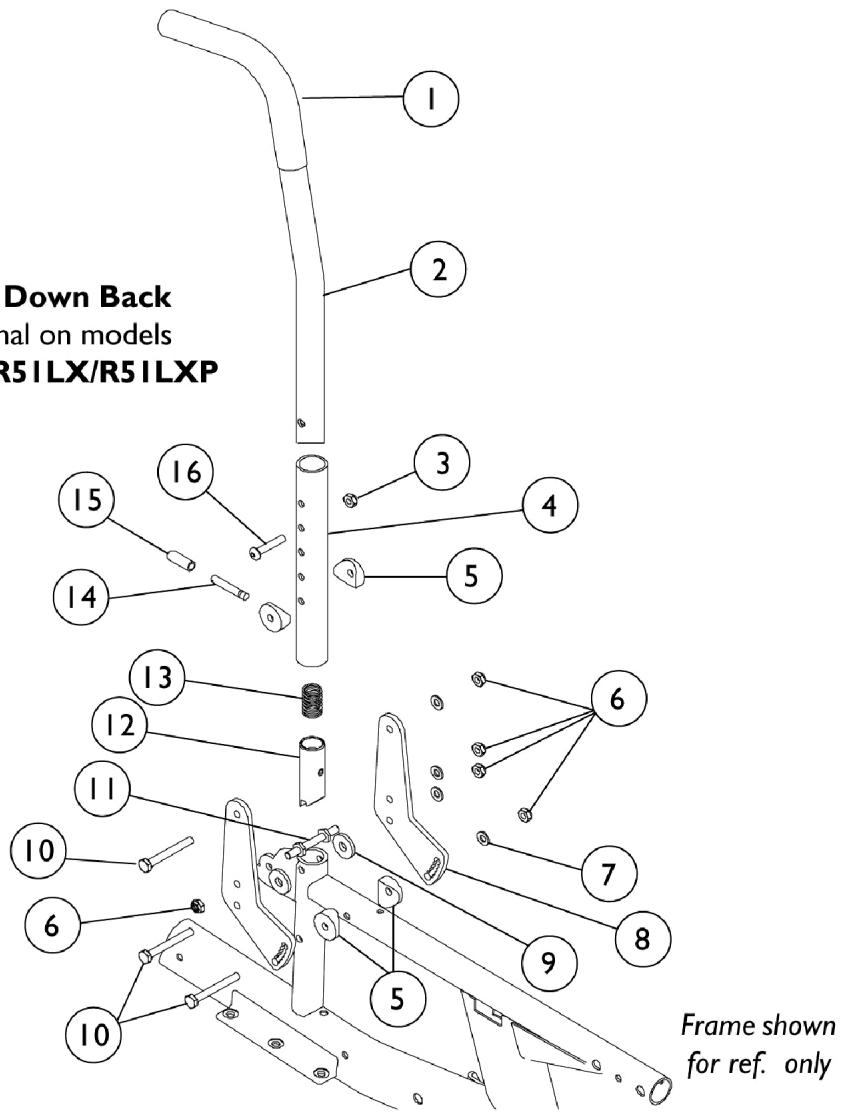

Back Canes and Hardware

Fold Down Adjustable Height Angle Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1006229	Grip, Foam (8")	1	1
2		1027848	Back Cane w/ Hardware, 10 Degree Bend - Right/Left (14-18" H)	1	1
3		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
4		1058832-NLA	NLA - Tube, Backrest, Black	1	1
5		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	4
6		7930334-NLA	NLA - Locknut (1/4-28)	25	5
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
8		1108626-NLA	NLA - Bracket, Pivot, Black	1	2
9		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	2
10		1055349	Screw, Hex Head (1/4-28 x 2")	1	3
11		1059127	Pin, Locking	1	1
12		1058232	Plug, Latch	1	1
13		1011511	Spring, Compression (5/8 x 1 x 3/64")	1	1
14		1058545	Lever, Actuator	1	1
15		1059271	Tip, Plastic, Black	1	1
16		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
	A,B	1107667-NLA	NLA - Fold Down Back Hardware	2	1

NOTE: A - Includes Items 3-16
B - Not Shown