



Back Canes & Adjustable Angle Back Hardware

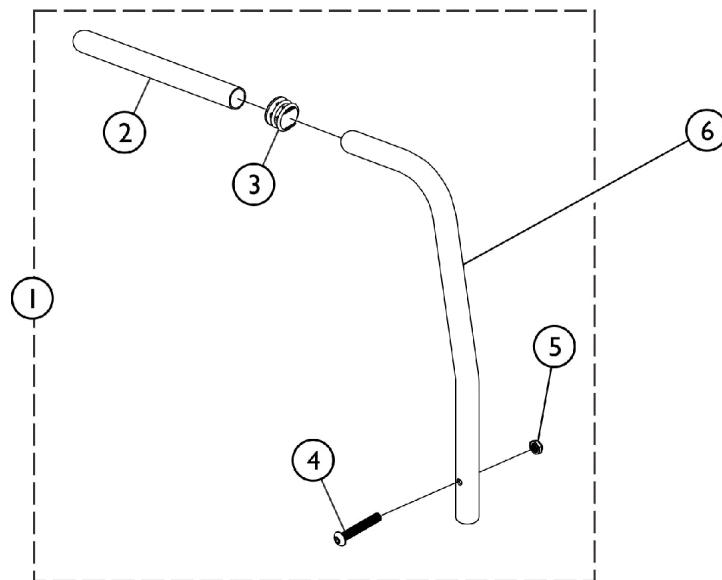

Adjustable Angle Back Hardware

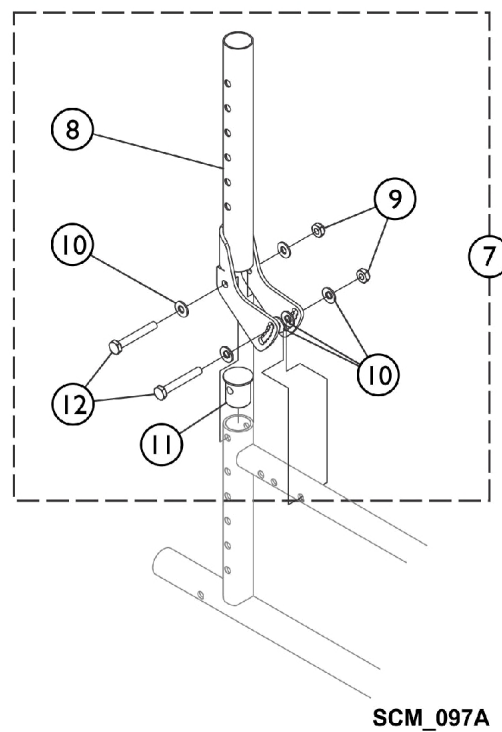

Back Canes & Adjustable Angle Back Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1052784-NLA	NLA - Back Cane w/ Hardware, 10 Degree Bend - Right/Left (16" H)	1	1
1	A	1052785-NLA	NLA - Back Cane w/ Hardware, 10 Degree Bend - Right/Left (18" H)	1	1
1	A	1052786-NLA	NLA - Back Cane w/ Hardware, 10 Degree Bend - Right/Left (20" H)	1	1
1	B	1044924-NLA	NLA - Back Cane w/ Hardware, 10 Degree Bend - Right/Left (20" H)	1	1
1	C	1027848	Back Cane w/ Hardware, 10 Degree Bend - Right/Left (14-18" H)	1	1
1	D	1044031-NLA	NLA - Back Cane w/ Hardware, Straight w/ Push Handle (14-18" and 16-20" H)	1	1
			NLA - Back Cane, Straight w/ Push Handle	1	1
			Grip, Foam (8")	1	1
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
			Locknut (1/4-20)	1	1
2		1006229	Grip, Foam (8")	1	1
3		1027097	Plug Button, Black (3/4")	1	1
4		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
5		1025195	Locknut (1/4-20)	1	1
6	B	1042975-NLA	NLA - Cane, Back, 10 Degree Bend (20" High)	1	1
6	E	1042370-NLA	NLA - Back Cane, Straight w/ Push Handle	1	1
	F,G	1026295	Kit, Back Cane Hardware (Pair)	2	1
7	H,I	1127078-NLA	NLA - Kit, Adjustable Angle Back Tubes with Hardware	2	1
8	I	8881063914-NLA	NLA - Bracket, Adjustable Angle Back Tube, Color	1	2
9		60275X000	Locknut (1/4-20)	1	4
10		16348X009-NLA	NLA - Washer, Steel (3/16 x 9/16 x 1/16")	1	12
11		1127072-NLA	NLA - Plug Button (1")	1	1
12		1013046-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2

NOTE: A - Non-Adjustable Back (Before 6/95)
 B - Non-Adjustable Back (After 5/95)
 C - Adjustable Back Height 14 - 18".
 D - Adjustable Back Height 14 - 18" and 16" - 20".
 E - No Longer Available as a service part. For Chairs manufactured before 10/6/95, order 1044031 (2) Back Cane.
 F - Kit includes items 1 - 4 (Pair).
 G - Not Shown in the illustration.
 H - Kit Includes items 8 - 12, enough for both sides.
 I - Specify frame finish when ordering, please use Configurator Button to select this item.