

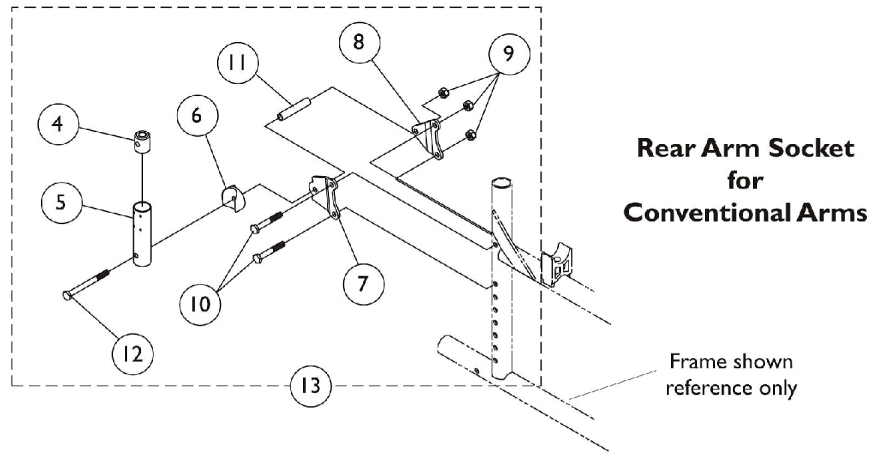


Arm Attaching Hardware (Before 6/24/99)

**Front Arm Socket
for
All Arms**

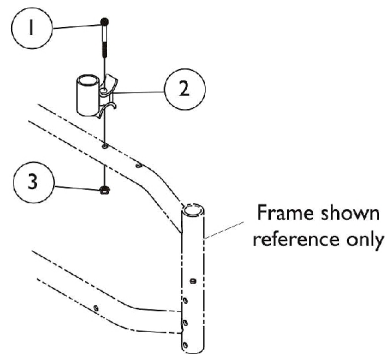

**Rear Arm Socket
for
Space Saver Arms**

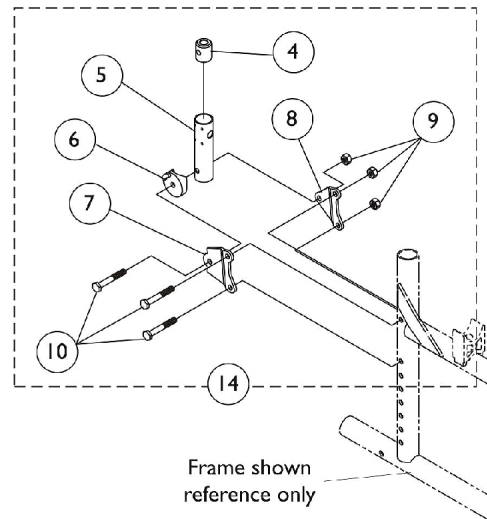

Arm Attaching Hardware (Before 6/24/99)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1059251-NLA	NLA - Screw, Socket Head (1/4-28 x 1-1/2")	1	2
2		1038129-NLA	NLA - Socket, Arm, Front	1	2
3		1025765	Locknut (1/4-28)	1	2
4		1068015-NLA	NLA - Support, Post, Arm	1	2
5		1015856	Socket, Armrest	1	2
6		1015426	Spacer, Coved Cam - Right	1	1
6		1015427	Spacer, Coved Cam - Left	1	1
7		1038131-NLA	NLA - Bracket, Armrest, Outside Mounting, Black	1	2
8		1038132-NLA	NLA - Bracket, Armrest, Inside Mounting, Black	1	2
9		1065362	Locknut (1/4-28)	1	6
10	A	1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	6
11		1028553-NLA	NLA - Spacer (1/4 x 3/8 x 3/4")	1	2
12		1055351-NLA	NLA - Screw, Hex Head (1/4-28 x 2-1/2")	1	2
13	B	1038197-NLA	NLA - Kit, Arm Mounting Hardware, Conventional (Pair)	1	1
14	C	1038196-NLA	NLA - Kit, Arm Mounting Hardware, Space Saver (Pair)	1	1
NOTE: A - 4 ea. required for Conventional Arms. B - Includes items 4-12 C - Includes items 4-10					