
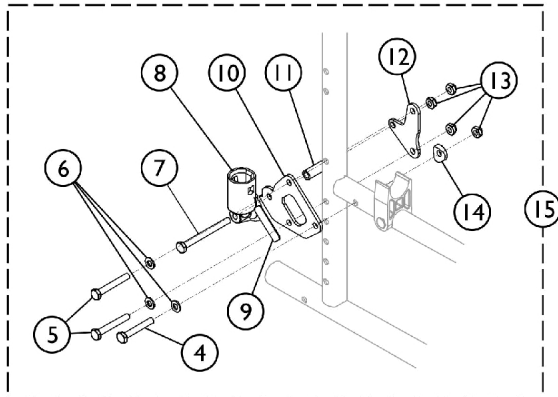



# Arm Attaching Hardware (After 6/23/99)

**Front Arm Socket  
for  
All Arms**

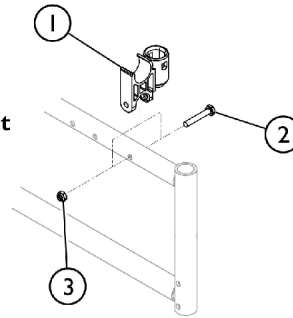


**Rear Arm Socket for  
Conventional Arms  
w/o Adjustable Angle Back**

**Rear Arm Socket for  
Conventional Arms  
with Adjustable Angle Back**

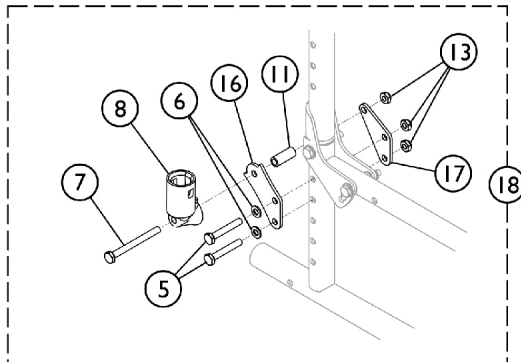

**Rear Arm Socket for  
Space Saver Arms**

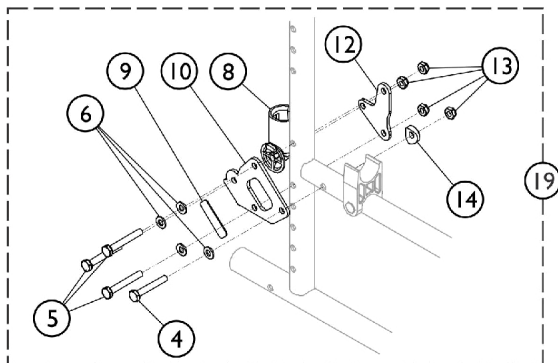

## Arm Attaching Hardware (After 6/23/99)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1056405-NLA	NLA - H-Block with Socket Cup	1	2
2		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
3		1025765	Locknut (1/4-28)	1	2
4		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
5	A	1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	6
6	B	1026907	Washer (1/4 x 1/2 x 1/16")	1	8
7		1087220-NLA	NLA - Screw, Hex Head (1/4-28 x 2-5/8")	1	2
8		1056406	Cup, Socket Pivot	1	2
9		1089569-NLA	NLA - Label, Bracket Warning	1	2
10	G,H	1086663-NLA	NLA - Bracket, Armrest, Outside Mounting	1	1
11		1087219	Spacer (1/4 x 3/8 x 1")	1	2
12		1092642	Bracket, Armrest, Inside Mounting	1	1
13	C	1025765	Locknut (1/4-28)	1	8
14		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
15	D	1087094-NLA	NLA - Kit, Arm Mounting Hardware, Conventional (Pair)	2	1
16	I	1127074-NLA	NLA - Bracket, Armrest, Outside Mounting, Black	1	2
17	I	1127073-NLA	NLA - Bracket, Armrest Mounting (Inside)	1	2
18	E	1127079-NLA	NLA - Kit, Arm Mounting Hardware - Conventional Arm	2	1
19	F	1087095-NLA	NLA - Kit, Arm Mounting Hardware - Space Saver Arm	2	1

NOTE: A - 4 ea. are required for Conventional Arms.  
 B - 6 ea. are required for Conventional Arms w/o adj. angle back. 4 ea. are required for Conventional Arms with adj. angle back.  
 C - 6 ea. are required for Conventional Arms with adj. angle back.  
 D - Chairs without adjustable angle back. Includes items 4-14.  
 E - Chairs with adjustable angle back. Includes items 5-8, 11,13,16, & 17.  
 F - Includes items 4-6, 8-10, 12-14.  
 G - When ordering this bracket, item 9 (Warning Label) must also be ordered.  
 H - No Longer Available.  
 I - No Longer Available individually. See Kit # 1127079 for replacement.