


Walking Beams and Hardware - SureStep

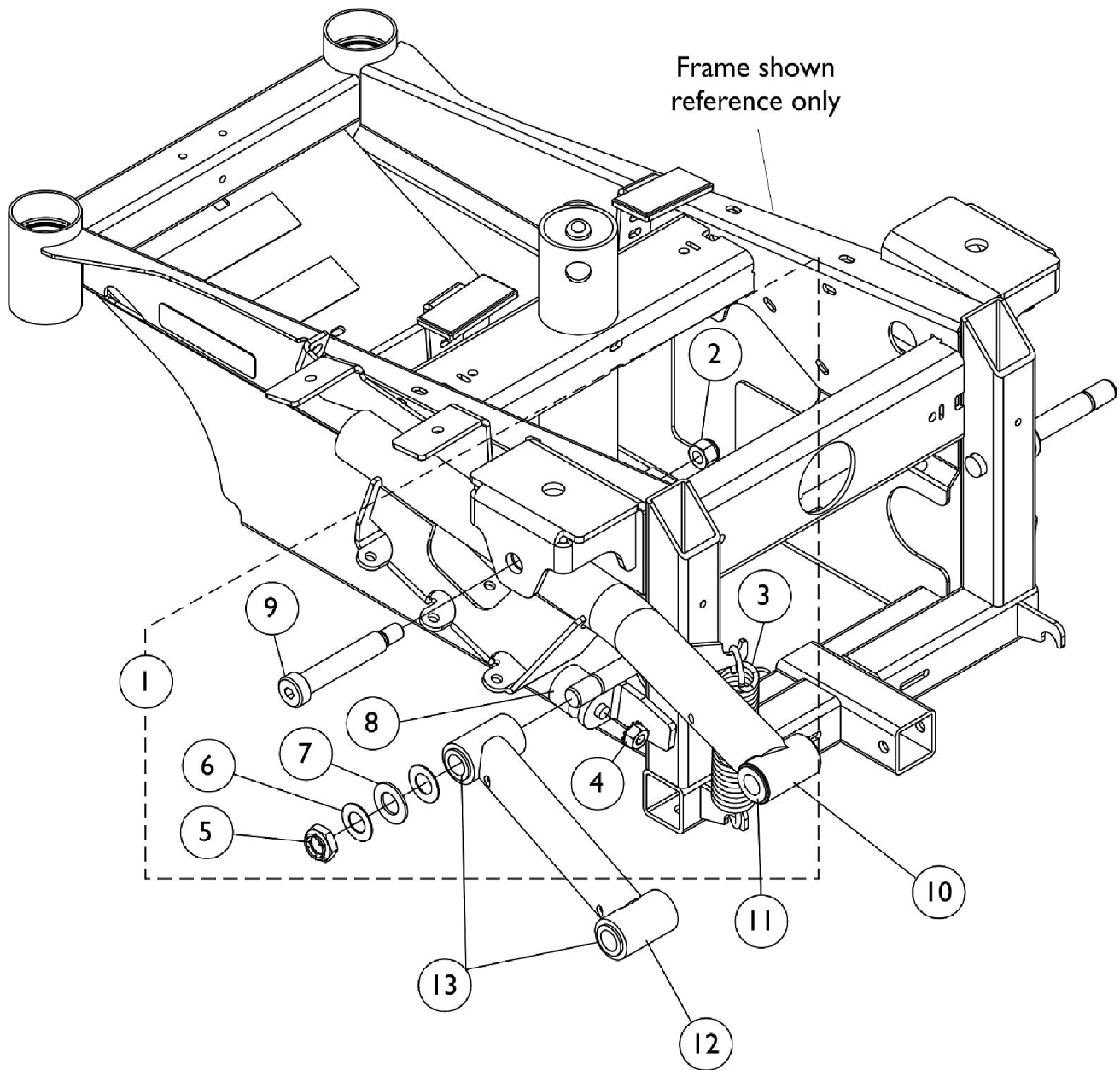

Walking Beams and Hardware - SureStep

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1127398	Kit, SureStep Walking Beam Hardware	1	1
2			NLA - Locknut, High Profile (3/8-16)	1	1
3			Spring, Extension (1 x 4-1/2" L)	1	1
4			NLA - Nut, Keps (5/16-18)	1	1
5			Locknut (1/2-20)	1	1
6			NLA - Washer (1/2 x 29/32 x 1/32")	1	2
7			NLA - Washer, Nylon (9/16 x 1 x 5/8")	1	1
8			NLA - Bumper, Rubber (1" Diameter x 5/8")	1	1
9			Screw, Shoulder Hex Socket Head (1/2 x 2-1/2", 3/8-16)	1	1
10	B	1127368-NLA	NLA - Walking Beam, SureStep - Right	1	1
10	B	1127369-NLA	NLA - Walking Beam, SureStep - Left	1	1
11		1127344	Bearing, Flange (1/2 x 7/8")	1	4
12	C	1109018	Tube, Pivot, Black	1	1
13		1106485	Bushing (1/2 x 5/8")	1	4
NOTE: A - Includes items 2-9 B - Includes item 11 C - Includes item 13					