


Adjustable Flip-Up Removable Footboard - Before 3/1/2000

90FBI (Sling Seat) and 90FB (Captain Seat)

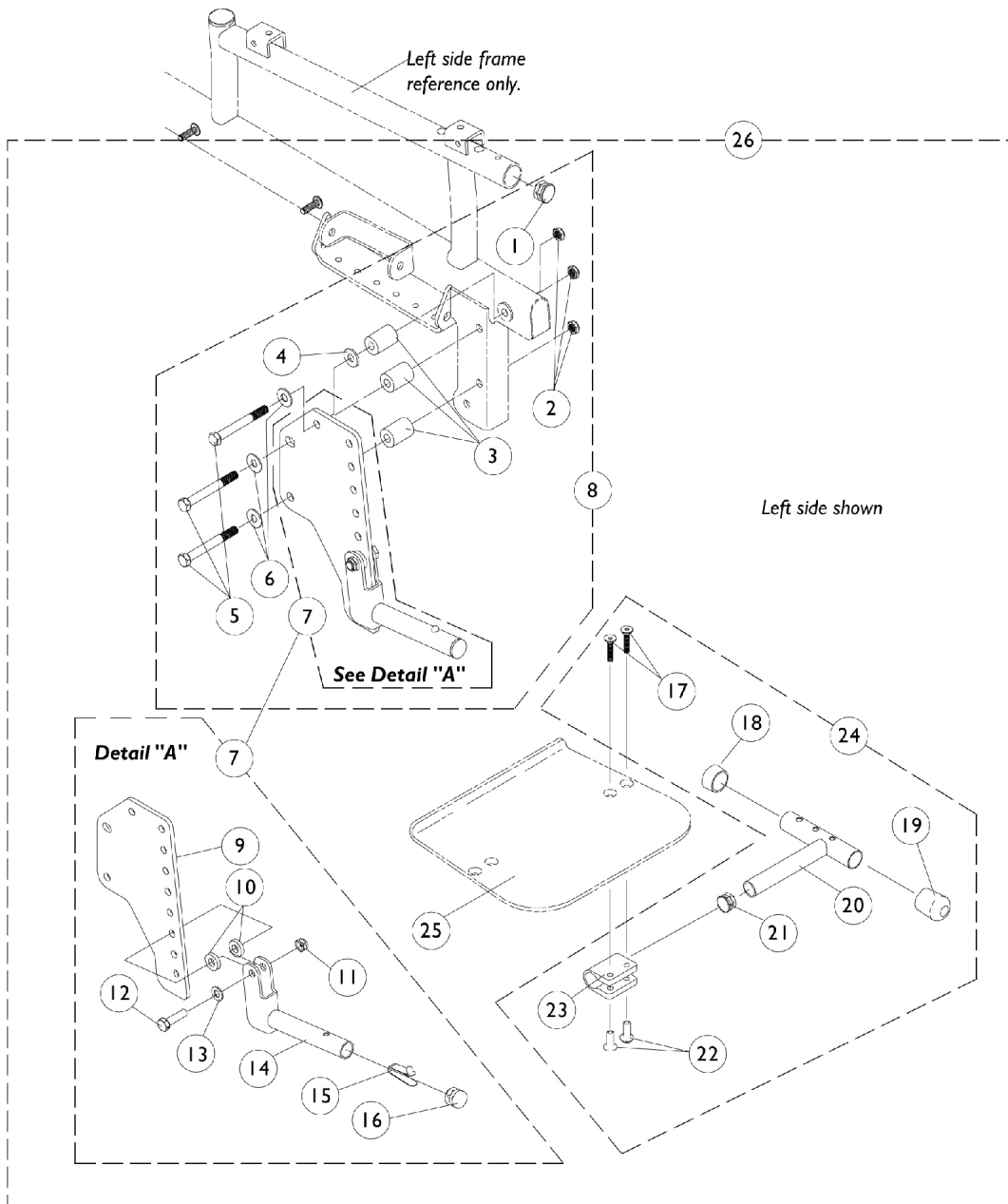

Adjustable Flip-Up Removable Footboard - Before 3/1/2000 90FBI (Sling Seat) and 90FB (Captain Seat)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	2
2		7930330-NLA	NLA - Locknut (5/16-24)	25	1
3		1074138-NLA	NLA - Spacer (5/16 x 3/4 x 1")	1	6
4		1074133	Washer, Steel (5/16 x 3/4 x 1/16")	1	2
5		1028600-NLA	NLA - Screw, Hex Head (5/16-24 x 3")	1	6
6		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	6
7	H	1074154-NLA	NLA - Footboard Pivot Assembly - Right	1	1
7	H	1074155-NLA	NLA - Footboard Pivot Assembly - Left	1	1
8	I	1074132-NLA	NLA - Footboard Mounting Hardware (Pair)	2	1
9		1074136-NLA	NLA - Bracket, Mounting, Black	1	1
10		1074210-NLA	NLA - Washer, Nylon (23/64 x 3/4 x 5/32")	1	2
11		7930330-NLA	NLA - Locknut (5/16-24)	25	1
12		1070552-NLA	NLA - Screw, Hex Head (5/16-24 x 1-1/8)	1	1
13		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	1
14		1074130-NLA	NLA - Tube, Pivot	1	1
15		40188X009	Button, Single Snap	1	1
16		1027097	Plug Button, Black (3/4")	1	1
17	A	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
18		15680X012	Collar, Anti-Rattle (7/8")	1	1
19		1016346	Cap, End, Tube, Black (7/8")	1	1
20		1074137-NLA	NLA - Tube, Footplate Support, Black	1	1
21		1027097	Plug Button, Black (3/4")	1	1
22	A	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
23	A	1058517	Clamp, Footplate	1	1
24	J	1074131-NLA	NLA - Footboard Assembly Hardware (Per Side)	1	2
25	C	1074129-NLA	NLA - Footboard	1	1
26	B,F	1078192-NLA	NLA - Kit, Footboard w/ Instruction Sheet	1	1
	G,K	1078193-NLA	NLA - Instruction Sheet, Footboard Assembly	1	1
	E,G	1094635	Kit, Footboard Conversion (16" W)	1	1
	D,E,G, L	1094636-NLA	NLA - Kit, Footboard Conversion (18" W) (Including 22" W Captain Seats)	1	1
	E,G	1094637-NLA	NLA - Kit, Footboard Conversion (20" W)	1	1

NOTE: A - Items 17, 22 and 23 must be ordered together for chairs made before 8/10/99 due to a hardware change.
 B - Footboard assembly kit before 3/1/00 is NLA. New style footboard assembly must be ordered. Refer to conversion kits listed on this page.
 C - Footboard before 3/1/00 is NLA.
 D - All Captain seats will use footboards for 18" wide chairs.
 E - Footboard assembly conversion kits for chairs made before 3/1/00.
 F - Includes items 1-25.
 G - Not Shown.
 H - Includes items 9-16.
 I - Includes items 1-7.
 J - Includes items 17-23.
 K - Before 3/1/2000.
 L - Includes items 9, 19, 26 and Instruction Sheet.