


Adjustable Flip-Up Removable Footboard - After 2/29/2000

90FBI (Sling Seat) and 90FB (Captain Seat)

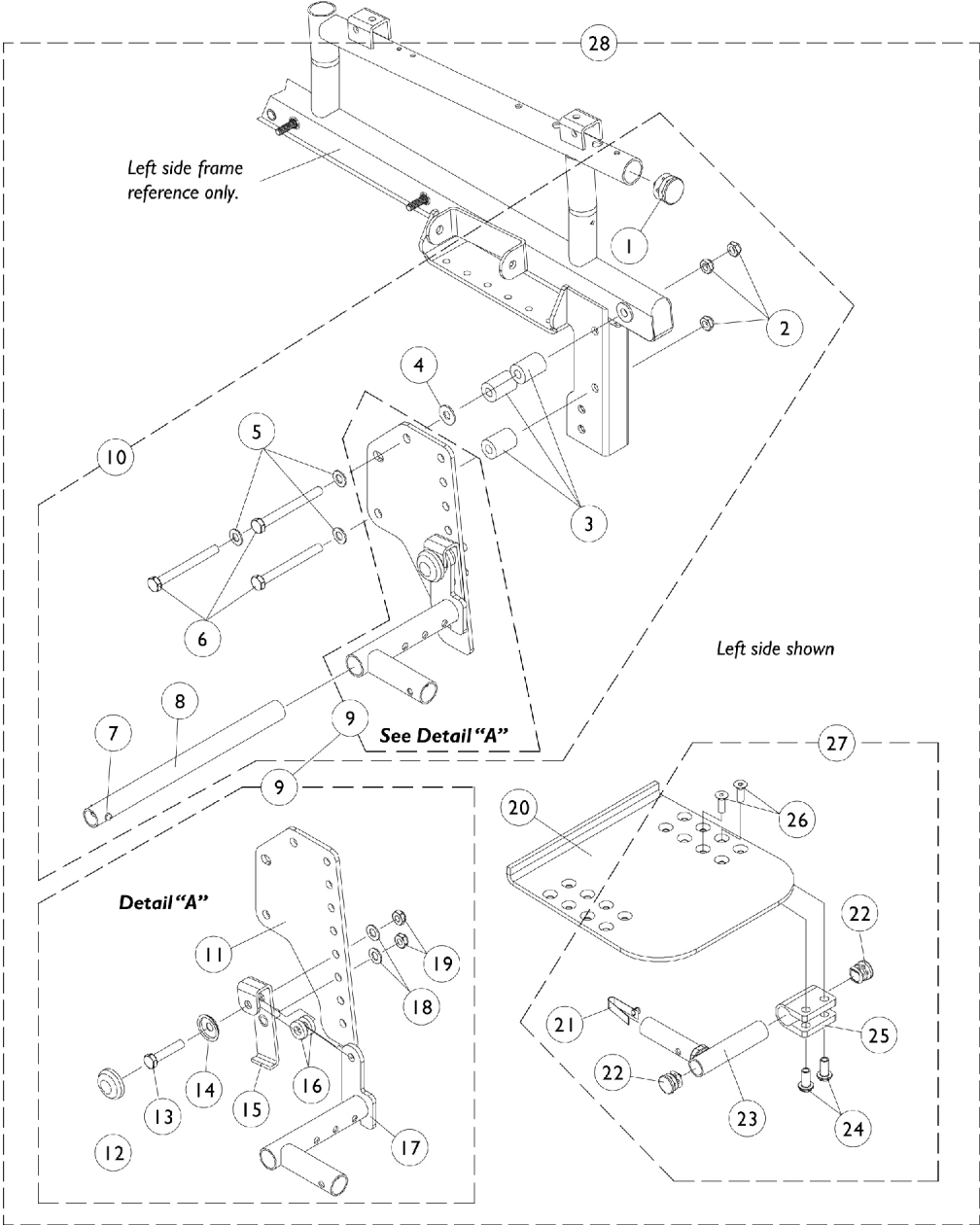

Adjustable Flip-Up Removable Footboard - After 2/29/2000 90FBI (Sling Seat) and 90FB (Captain Seat)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	2
2		7930330-NLA	NLA - Locknut (5/16-24)	25	1
3		1074138-NLA	NLA - Spacer (5/16 x 3/4 x 1")	1	6
4		1074133	Washer, Steel (5/16 x 3/4 x 1/16")	1	2
5		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	6
6		1028600-NLA	NLA - Screw, Hex Head (5/16-24 x 3")	1	6
7		40188X007-NLA	NLA - Button, Spring Double	1	1
8		1095124-NLA	NLA - Support Tube w/ Spring Assembly	1	1
9	C	1094626-NLA	NLA - Footboard Pivot Assembly - Right	1	1
9	C	1094627-NLA	NLA - Footboard Pivot Assembly - Left	1	1
10	D	1094628-NLA	NLA - Footboard Mounting Hardware Assembly (Pair)	1	1
11		1074136-NLA	NLA - Bracket, Mounting, Black	1	1
12		1016291	Cap, Hub, Black	1	1
13		1070398-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	1
14		1028491-NLA	NLA - Washer, Caplugs (11/32" I.D. x 1" O.D.)	1	1
15		1094632-NLA	NLA - Bracket, Stop Pivot Support, Black	1	1
16		1074210-NLA	NLA - Washer, Nylon (23/64 x 3/4 x 5/32")	1	2
17		1094614-NLA	NLA - Bracket, Footboard Receiving, Black - Right	1	1
17		1094615-NLA	NLA - Bracket, Footboard Receiving, Black - Left	1	1
18		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	2
19		1028769	Locknut (5/16-18)	1	2
20		1097175-NLA	NLA - Footboard, Black (16" W)	1	1
20	A	1094613-NLA	NLA - Footboard, Black (18" W)	1	1
20		1094634-NLA	NLA - Footboard, Black (20" W)	1	1
21		40188X009	Button, Single Snap	1	2
22		1027097	Plug Button, Black (3/4")	1	4
23		1094611-NLA	NLA - Support Tube w/ Snap Tube, Black	1	2
24		1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	4
25		1058517	Clamp, Footplate	1	2
26		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	4
27	E	1094629-NLA	NLA - Hardware, Footboard (Pair)	2	1
28	F	1094635	Kit, Footboard Conversion (16" W)	1	1
28	A,F,G	1094636-NLA	NLA - Kit, Footboard Conversion (18" W) (Including 22" W Captain Seats)	1	1
28	F	1094637-NLA	NLA - Kit, Footboard Conversion (20" W)	1	1
	B	1095125-NLA	NLA - Instruction Sheet, Footboard Assembly	1	1

NOTE: A - All Captain seats will use footboards for 18" wide chairs.
B - Not Shown.
C - Includes items 11-19.
D - Includes items 1-6, 8 and 9.
E - Includes items 21-26 (pair).
F - Includes items 10, 20 and 27.
G - Including 22" Wide Captain Seats.