



Standard Back Canes with Grip - 9000SL

9000 SL

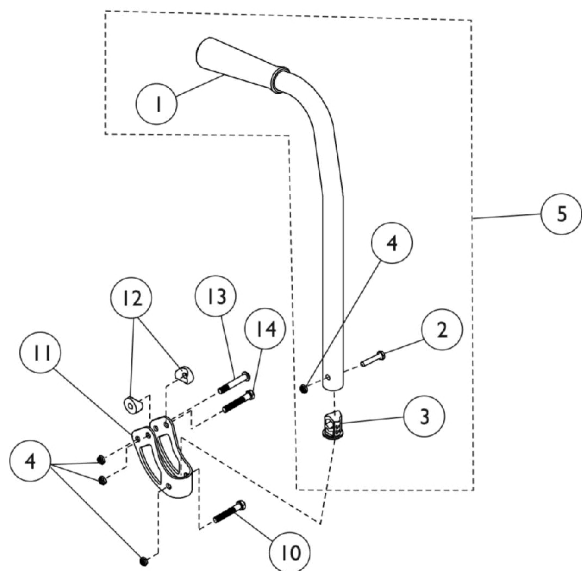

Before 7/12/99

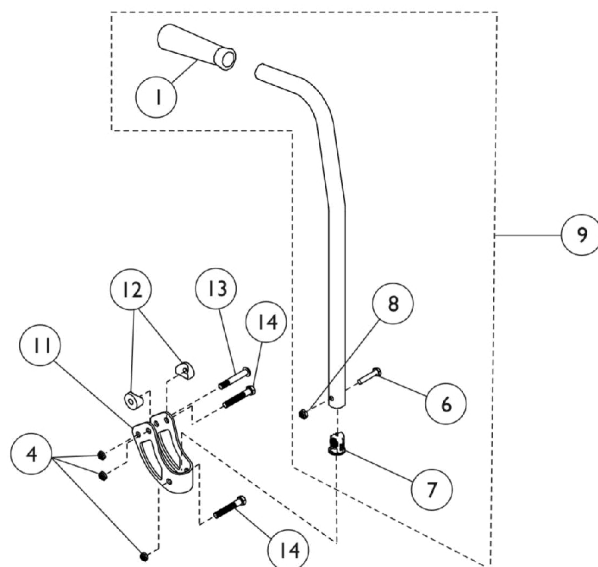

After 7/11/99

Standard Back Canes with Grip - 9000SL

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1057248	Grip, Hand, Foam (4-3/4")	1	1
2		1058561-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/2")	1	1
3		1056551	Insert	1	1
4		60275X000	Locknut (1/4-20)	1	1
5		See Chart	Standard Back Canes with Grip - 9000SL	1	1
6		1057955	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
7		1056551	Insert	1	1
8		1025195	Locknut (1/4-20)	1	1
9	B	See Chart	Standard Back Canes with Grip - 9000SL	1	1
10		16720X004	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
11		1056110-NLA	NLA - Bracket, Armrest, Chrome	1	1
12		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	2
13		1029583-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
14		1013046-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	1
NOTE: A - To be used with models manufactured before 7/12/99. Includes items 1-5. B - To be used with models manufactured after 7/11/99. Includes items 1, 6, 7 and 8.					

Standard Back Canes with Grip - 9000SL

Back Height	Part Number
15"	1058203-NLA See Note(s) A
15 - 17"	1091102 See Note(s) B
16"	1058204-NLA See Note(s) A
17"	1058205-NLA See Note(s) A
17 - 19"	1091103 See Note(s) B
18"	1058206-NLA See Note(s) A
19"	1058207-NLA
NOTE: A - Before 7/19/99 B - After 7/18/99	