
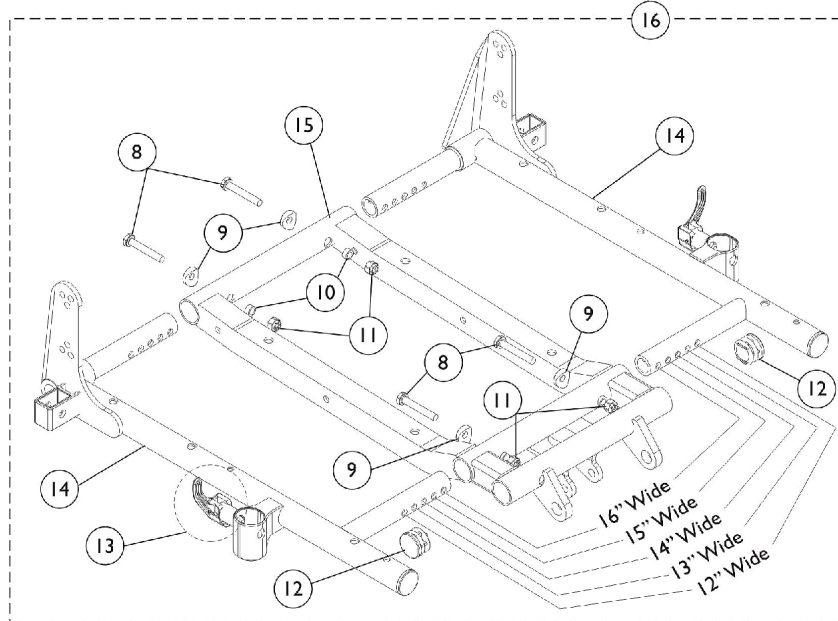


# ASBA Seat Frame, Seat Pan and Hardware

Before 10/27/04

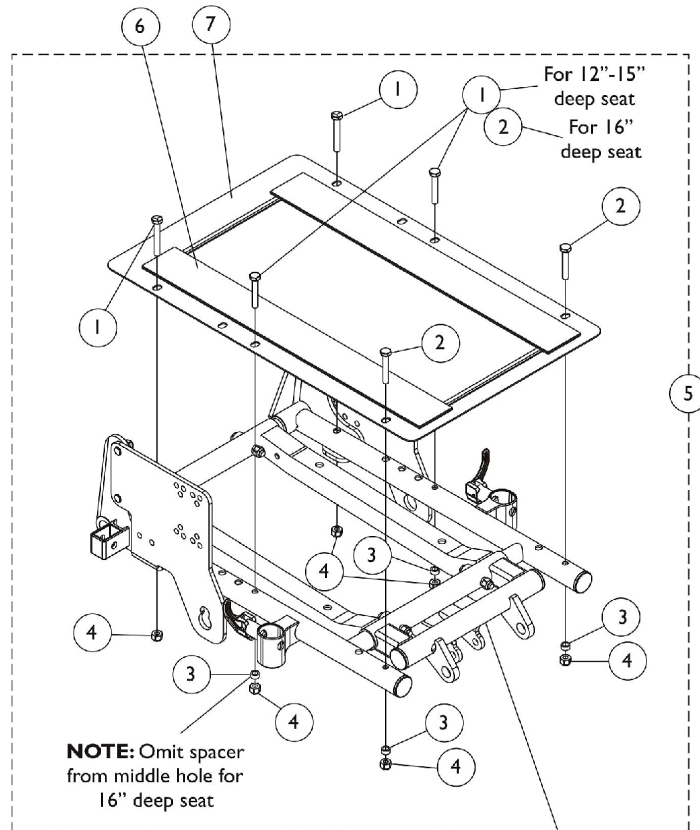

## ASBA Seat Frame, Seat Pan and Hardware Before 10/27/04

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
2	B	1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	4
3	B	1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
4		1025736	Locknut (1/4-20)	1	6
5	C	1123393-NLA	NLA - Kit, Seat Pan Mounting Hardware (12 - 15" Deep Seats)	1	1
5	C	1123394-NLA	NLA - Kit, Seat Pan Mounting Hardware (16" Deep Seat)	1	1
6		1039949-NLA	NLA - Fastener, Hook Adhesive Back (2" W x 16" L)	1	2
6		1044578-NLA	NLA - Fastener, Loop Adhesive Back (2" W x 16" L)	1	2
7		See Chart	#7 Seat Pans	1	1
8		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	4
9		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
10		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
11		1025736	Locknut (1/4-20)	1	4
12		1029501	Plug Button, Black (1")	1	2
13		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
14		See Chart	#14, 15, & 16 ASBA Junior Seat Frame Assembly	1	1
15		See Chart	#14, 15, & 16 ASBA Junior Seat Frame Assembly	1	1
16		See Chart	#14, 15, & 16 ASBA Junior Seat Frame Assembly	1	1

NOTE: A - 16" deep seats use a quantity of 4 each  
B - 16" deep seats use a quantity of 2 each  
C - Includes items 1-4

**ASBA Seat Frame, Seat Pan and Hardware**  
**Before 10/27/04**  
**#7 Seat Pans**

Chair Width	12"-15" Deep	16" Deep
12" Wide	1066009-NLA	1070513
13" Wide	1066010-NLA	1070514-NLA
14" Wide	1066022-NLA	1066011-NLA
15" Wide	1066023-NLA	1066012-NLA
16" Wide	1066024	1066013-NLA

NOTE: 12"-15" seat depths are achieved by using different back cane mounting hardware.

**#14, 15, and 16 Seat Frame Assembly**

Item Number	Seat Frame	12"-16" W 12"-16" D
14	Seat Frame Right See Note(s) A	1070473-NLA
14	Seat Frame Left See Note(s) A	1070474-NLA
15	Seat Frame Center	1121982-NLA
16	Seat Frame Assembly See Note(s) B	1121991-NLA

NOTE: A - Seat frame includes item 13 arm lock assembly.  
B - Includes items 8-15