


# 90FB Footboard Assembly

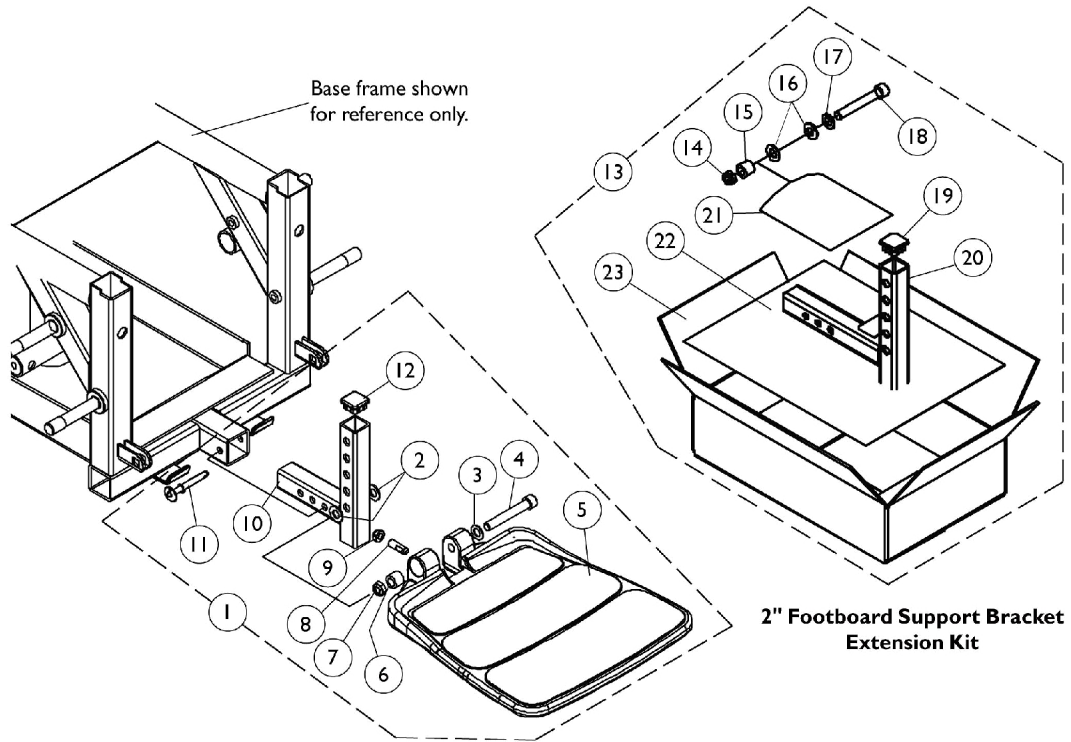

## 90FB Footboard Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,E	1105027-NLA	NLA - Footboard Assembly	1	1
2			Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
3			Washer, Flat (3/8 x 11/16 x 1/16")	1	1
4			Screw, Socket Head (3/8-16 x 2-1/2")	1	1
5			NLA - Footboard with Cover	1	1
6			Bushing (3/8 x 5/8 x 1/2")	1	1
7			Locknut, Low Profile (3/8-16)	1	1
8			Screw, Set Socket Flat Point (5/16-18 x 1")	1	1
9			Locknut (5/16-18)	1	1
10			Bracket, Footboard Support, Black	1	1
11			Pin, Quick Release	1	1
12			Plug Button, Black (1" Square)	1	1
13	C,D	1115378	Kit, 2" Footboard Extension	1	1
14			Locknut, Low Profile (3/8-16)	1	1
15			Bushing (3/8 x 5/8 x 1/2")	1	1
16			Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
17			Washer, Flat (3/8 x 11/16 x 1/16")	1	1
18			Screw, Socket Head (3/8-16 x 2-1/2")	1	1
19			Plug Button, Black (1" Square)	1	1
20			Bracket, Footboard Support, Black (2" Extension)	1	1
21			NLA - Bag, Zip-Lip (4" L x 4" W)	1	1
22			NLA - Instruction Sheet, 2" Footboard Extension	1	1
23			Carton, Shipper	1	1

NOTE: A - Includes items 2-12  
 B - When ordering a footboard assembly for the first time that requires a 2" extension kit, items 5, 8, 9, 11 and 13 must be ordered  
 C - Includes items 14-23  
 D - Part number 1114833 is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.  
 E - Part #1105027 is No Longer Available and a replacement is currently not available.