


# Walker (Blue-Release)

Models (6241-5F, 6241-JR, and 6241-JR5F - Discontinued 6/07)

Model 6241-5F (4 Pack)

Model 6241-JR5F (4 Pack)

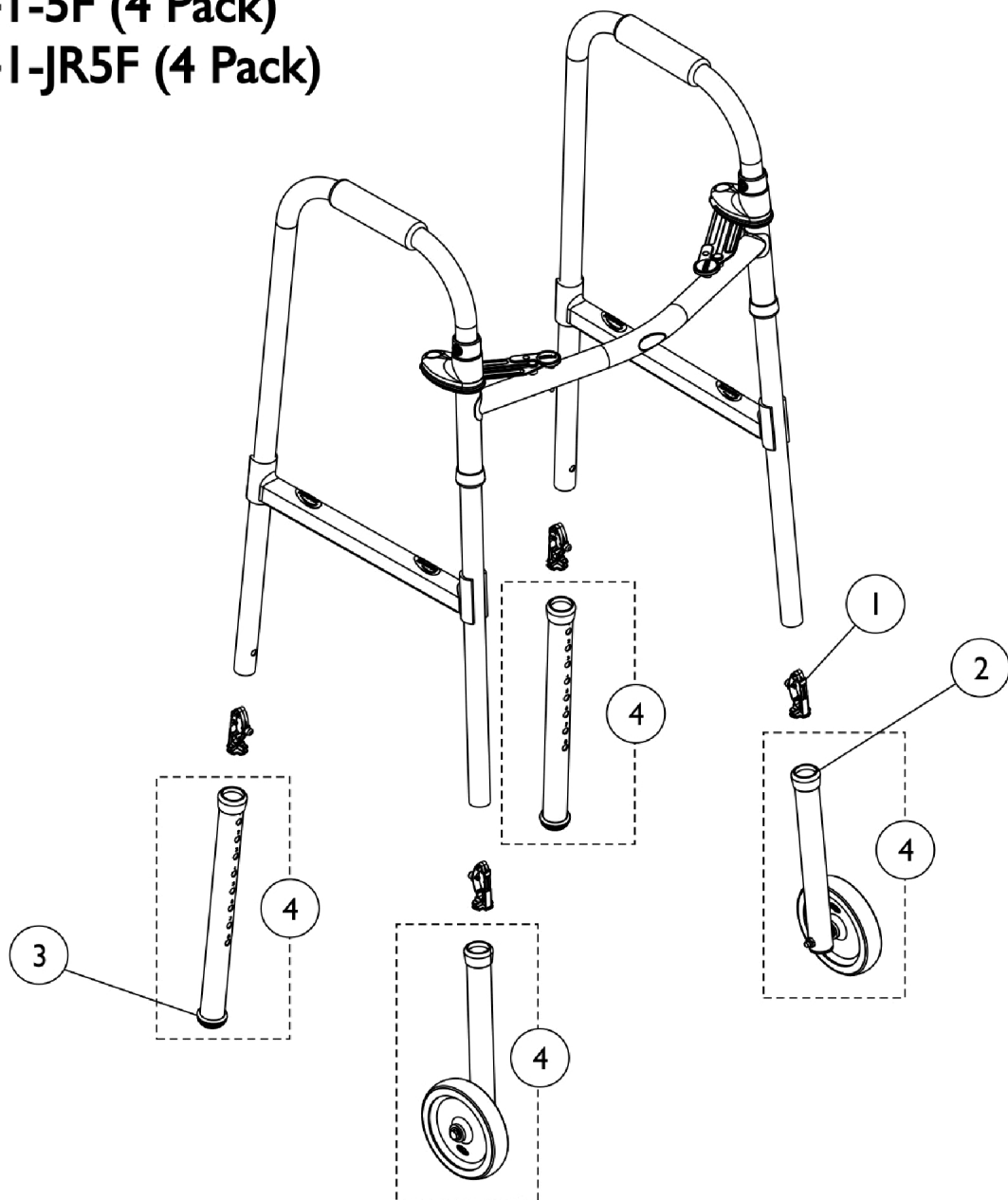

# Walker (Blue-Release)

## Models (6241-5F, 6241-JR, and 6241-JR5F - Discontinued 6/07)

### Model 6241-5F and 6241-JR5F

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1074462-NLA	NLA - Button, Snap	1	4
1	C	1130780-NLA	NLA - Kit, Snap Buttons	10	1
1		1094945	Button, Snap	1	4
2		1040655	Anti-Rattle, Grey (1-1/8")	1	2
3	B	1041531-NLA	NLA - Tip, Glide, Grey (1-1/8")	1	2
3		1130781	Kit, Glide Tips	10	1
4		1086797	Kit, Leg Extension (5"), Fixed	1	1
			Leg Extension Tube with Axle Hole	1	2
			Bushing (1")	1	2
			Wheel (5" x 1-1/4")	1	2
			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	6
			Bushing (1/4 x 1-3/8")	1	2
			Screw, Hex Head (1/4-20 x 2-3/4")	1	2
			Anti-Rattle, Grey (1-1/8")	1	4
			Locknut (1/4-20)	1	2
			Tube, Leg Extension (13")	1	2
			NLA - Tip, Glide, Grey (1-1/8")	1	2

NOTE: A - Individual item no longer available, use 1130780 as its replacement.  
 B - Individual item no longer available, use 1130781 as its replacement.  
 C - Kit is no longer available, use 1094945 as its replacement.