




Accessories for Model 7030 & 7030P (Octagon Tube)

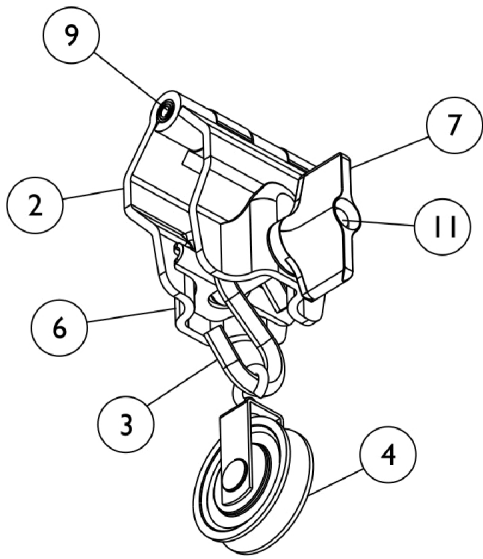


**7801 & 7801P
Pulley Clamp**

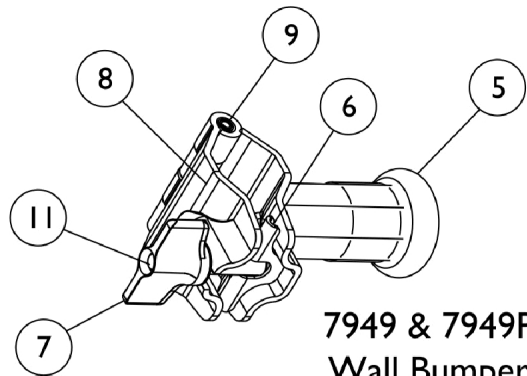


**7949 & 7949P
Wall Bumper
Post with Clamp**

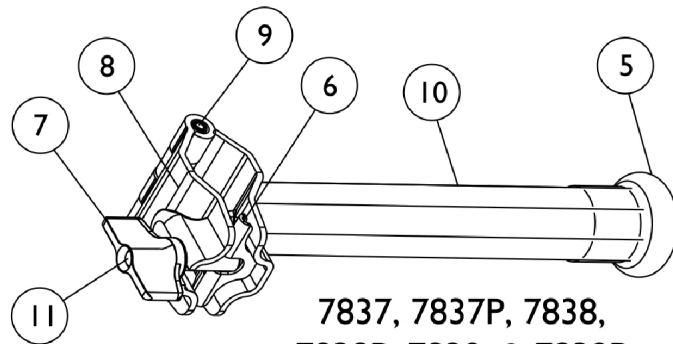

**7837, 7837P, 7838,
7838P, 7839, & 7839P
Traction Arms
(9", 12" & 18")**

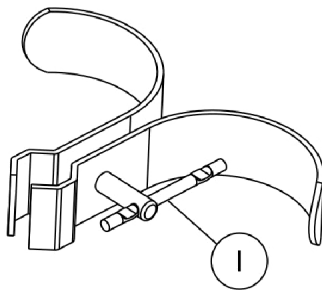

**7948 & 7948P
Wall Bumper
with Horns**

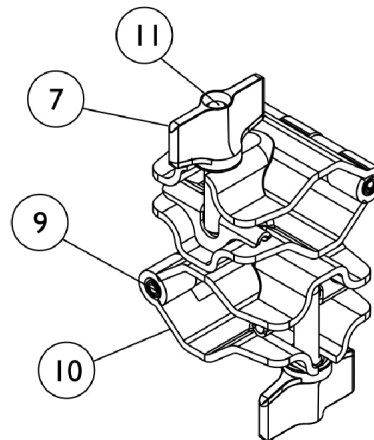

**7803 & 7803P
Double Clamp**

Accessories for Model 7030 & 7030P (Octagon Tube)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		17573X000-NLA	NLA - Key	1	1
2	B,J	07800X001-NLA	NLA - Clamp, Hinge, Chrome	1	1
2	C	1125219-NLA	NLA - Clamp, Hinge, Grey	1	1
3		17103X000-NLA	NLA - Hook, "S"	1	1
4		17130X000-NLA	NLA -Pulley, Swivel	1	1
5		17167X000-NLA	NLA - Bumper, Grey	1	1
6	A	17127X000	Pin, Spiral (5/32" x 1-3/4")	1	1
7	A	17129X000-NLA	NLA - Knob, Wing	1	1
8	A,B,D	17122X000-NLA	NLA - Clamp, Hinge, Slotted Side, Chrome	1	1
8	C	1135726-NLA	NLA - Clamp, Hinge, Slotted Side, Grey	1	1
9	A	17128X000-NLA	NLA - Pin, Spiral (5/16" x 2-1/2")	1	1
10	B,H	17803X000-NLA	NLA - Arm, Traction, Chrome (9")	1	1
10	C	1135731-NLA	NLA - Arm, Traction, Grey (9")	1	1
10	B,G	17802X000-NLA	NLA - Arm, Traction, Chrome (12")	1	1
10	C	1135730-NLA	NLA - Arm, Traction, Grey (12")	1	1
10	B,F	17801X000-NLA	NLA - Arm, Traction, Chrome (18")	1	1
10	C	1135729-NLA	NLA - Arm, Traction, Grey (18")	1	1
	E	7837P-NLA	NLA - Traction Arm Assembly, Grey (18")	1	1
	E	7838P-NLA	NLA - Traction Arm Assembly, Grey (12")	1	1
	E,K	7839P-NLA	NLA - Traction Arm Assembly, Grey (9")	1	1
11	A	1008630-NLA	NLA - Stud, Threaded (3/8-16 x 2-1/4")	1	1
	I	7801P-NLA	NLA -Pulley Clamp Assembly, Grey	1	1

NOTE: A - When ordering replacement parts for 7803 Double Clamp, you will need 2 per assembly.
 B - Before 9/9/05.
 C - After 9/8/05.
 D - This part is No Longer Available, please order 1135726 as a replacement part instead.
 E - Not Shown.
 F - This part is No Longer Available, please order 7837P as a replacement part instead.
 G - This part is No Longer Available, please order 7838P as a replacement part instead.
 H - This part is No Longer Available, please order 7839P as a replacement part instead.
 I - Includes items 2-4, 6, 7, 9 and 1. Model 7801 is no longer available.
 J - No Longer Available.
 K - 7839 is No Longer Available. Please order 7839P as a replacement part instead.