




Shrouds and Attaching Hardware

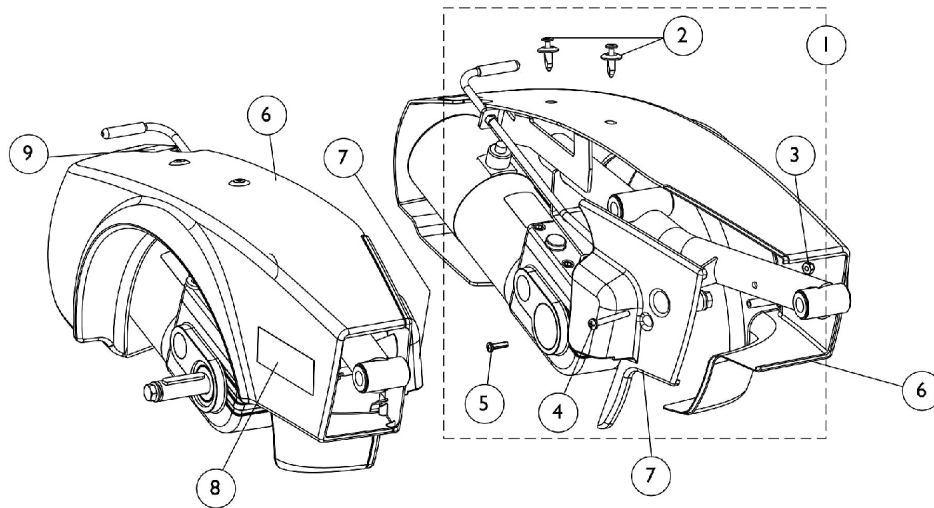


Side Shrouds

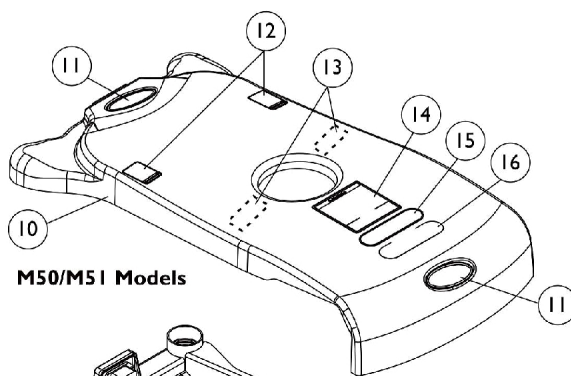

M50/M51 Models

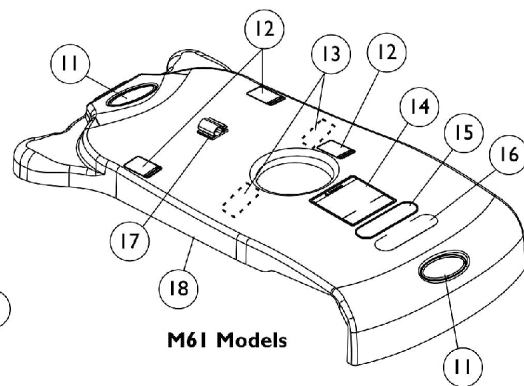

M61 Models

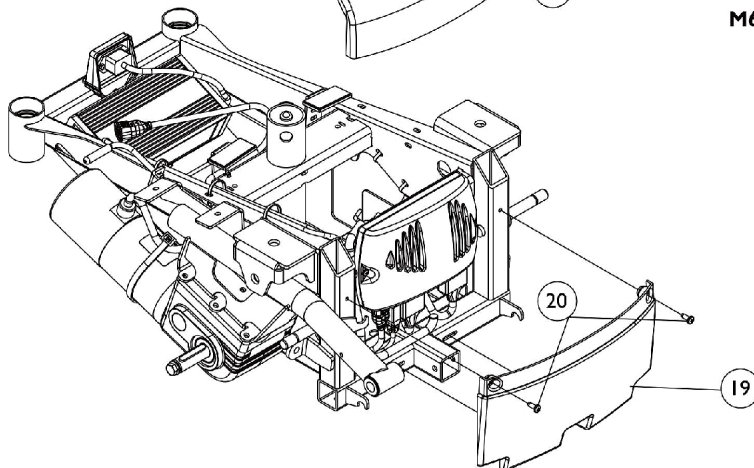

Top and Front Shrouds

Shrouds and Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1127709	Kit, Side and Inner Shroud Hardware	1	1
2			Screw, TUFLOK (1/4-20 x 11/16")	1	4
3			Locknut (#10-32)	1	2
4			Screw, Phillips Pan Head (10-32 x 1-5/8")	1	2
5			Screw, Phillips Pan Head (#8-16 x 5/8")	1	2
6	B	See Chart	#6 Side Shroud Assembly	1	1
7		1127380	Shroud, Inner - Right	1	1
7		1127381	Shroud, Inner - Left	1	1
8		1110604-NLA	NLA - Decal, Shroud - M50	1	1
8		1110605	Decal, Shroud - M51	1	1
8	C	1127292-NLA	NLA - Decal, Shroud - M61	1	1
9		1104797	Decal, Drive/Push	1	1
10		1127336	Top Shroud Assembly	1	1
11		1095489	Decal, Invacare 3-D Logo (2-1/2" L)	1	2
12		1079203	Decal, Warning Pinch Point	1	2
13		1070509-NLA	NLA - Fastener, Loop with Adhesive Back (1" W x 2" L)	1	2
14		1122126-NLA	NLA - Label, Battery Warning	1	1
15		1119281-NLA	NLA - Plate, Name, Patent Numbers	1	1
16		1035900	Decal, Transportation	1	1
17		1127276-NLA	NLA - Clip, Wire with Adhesive Back	1	1
18	D	1127275	Top Shroud Assembly	1	1
19		1127383	Shroud - Front	1	1
20		1028484	Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	2
NOTE: A - Includes items 2-5 B - Includes Decals, items 8 & 9 C - Includes items 11-16 D - Includes items 11-17					

#6 Side Shroud Assembly

Includes Decals, items 8 & 9

Chair Model	Red	Blue
M50 - Right	1127324-NLA	1127326-NLA
M50 - Left	1127325-NLA	1127327-NLA
M51 - Right	1127328 See Note(s) A	1127330
M51 - Left	1127329 See Note(s) A	1127331
M61 - Right	1127332 See Note(s) A	1127334
M61 - Left	1127333 See Note(s) A	1127335
NOTE: A - Due to a supplier change it is recommended to order the right and left Side Shrouds to match in color shade for wheelchairs manufactured before 1/3/12.		