


Swingaway Half Arms (AT99)

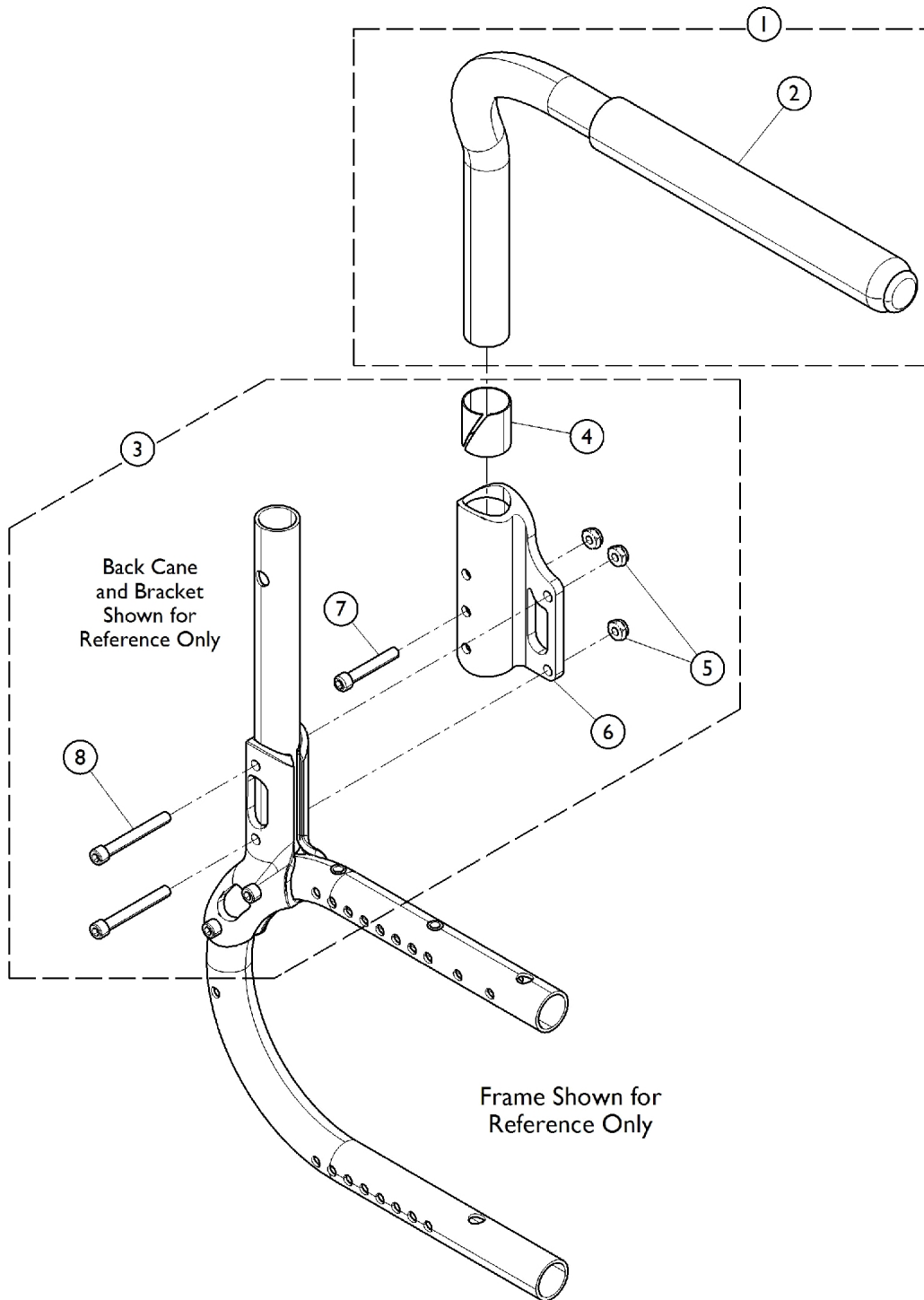

Swingaway Half Arms (AT99)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1130576	Half Arm Assembly, Black - Right	1	1
1	A	1130577	Half Arm Assembly, Black - Left	1	1
2		1032244	Grip, Foam Hand (9" L)	1	2
3	B	1130578	Kit, Half Arm Hardware	2	1
4			Sleeve, Bearing	1	2
5			Locknut (1/4-28)	1	6
6			Socket, Arm - Right	1	1
6			Socket, Arm - Left	1	1
7			Screw, Socket Head (1/4-28 x 1-3/4")	1	2
8			Screw, Socket Head (1/4-28 x 2")	1	4
NOTE: A - Includes item 2, Foam Grip B - Includes items 4-8					