
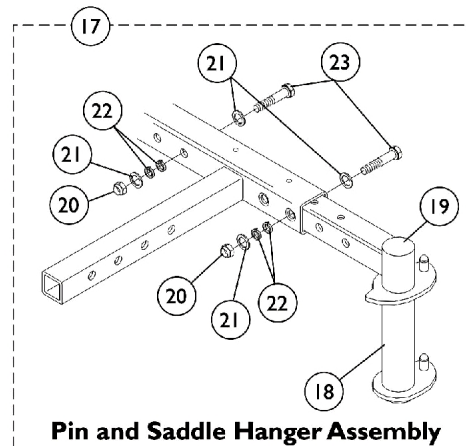
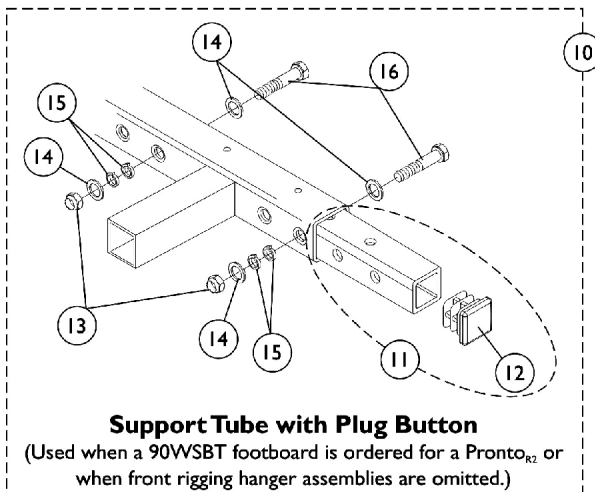


Front Rigging Hanger Assembly

Center Pivot Sector, 90WSBT Footboard, Pin & Saddle Style

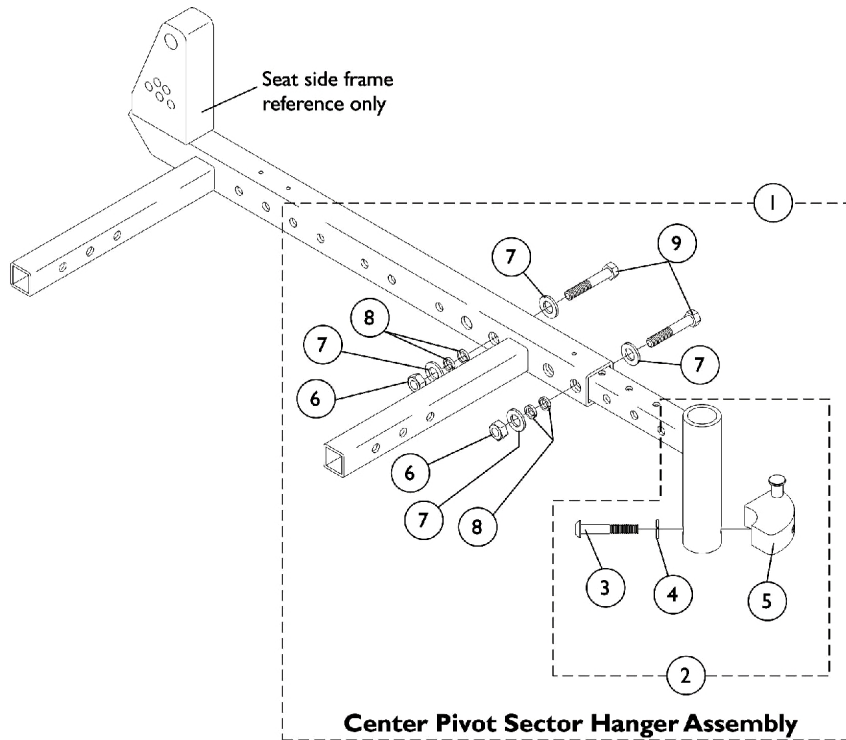

Front Rigging Hanger Assembly

Center Pivot Sector, 90WSBT Footboard, Pin & Saddle Style

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1072724-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Center Pivot, Storm, Black - Right	1	1
1	A	1072726-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Center Pivot, Storm, Black - Left	1	1
1	A	0197500-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Storm, Black (Pair)	2	1
2	B	1072722-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Center Pivot, Storm, Black - Right	1	1
2	B	1072723-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Center Pivot, Storm, Black - Left	1	1
3		0139506	Screw, Button Head, Socket (1/4-20 x 1-1/2")	1	1
4		1007453	Lockwasher, Split (1/4")	1	1
5		0117000	Block, Sector - Right	1	1
5		0117100	Block, Sector - Left	1	1
6		1025736	Locknut (1/4-20)	1	2
7		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
8		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	4
9		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
10	C	1102792-NLA	NLA - Support Tube w/ Plug Button and Hardware, Black	1	1
11		1096519-NLA	NLA - Support Tube w/ Plug Button, Black	1	1
12		0062200-NLA	NLA - Plug Button, Square (3/4")	1	1
13		1025736	Locknut (1/4-20)	1	2
14		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
15		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	4
16		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
17	D	1081425-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black - Right	1	1
17	D	1081426-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black - Left	1	1
17	D	1081446-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black (Pair)	2	1
18		1078020-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Right	1	1
18		1078021-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Left	1	1
19		1027097	Plug Button, Black (3/4")	1	1
20		1025736	Locknut (1/4-20)	1	2
21		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
22		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	4
23		0051506	Screw, Hex Head (1/4-20 x 1-3/4")	1	2

NOTE: A - Includes items 2 and 6-9
B - Includes the front rigging hanger and items 3-5
C - Includes items 11 and 13-16
D - Includes items 18 and 20-23