
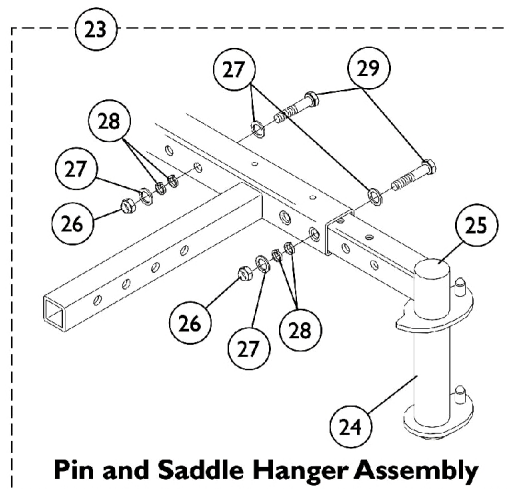
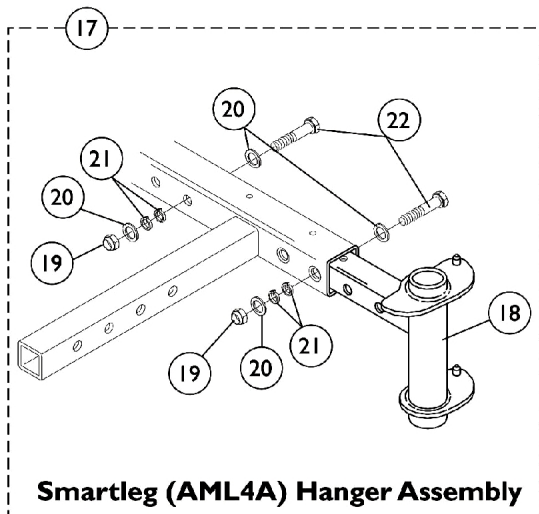
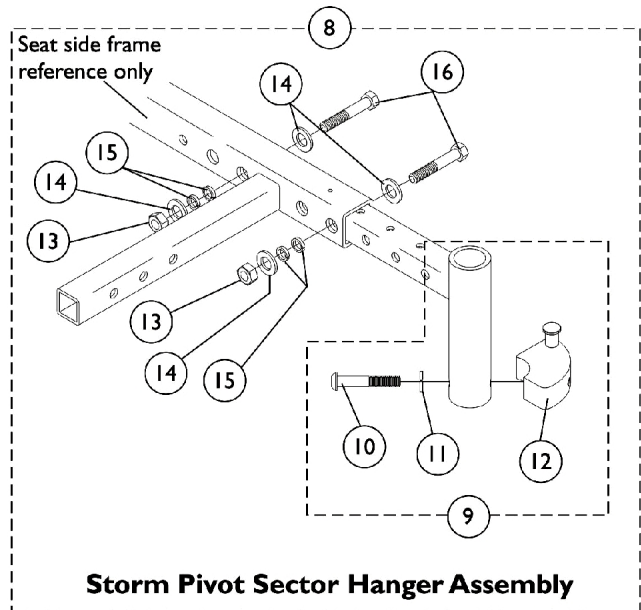


# Front Rigging Hanger Assembly

## Sector, Genius, Pin, AML4A Style

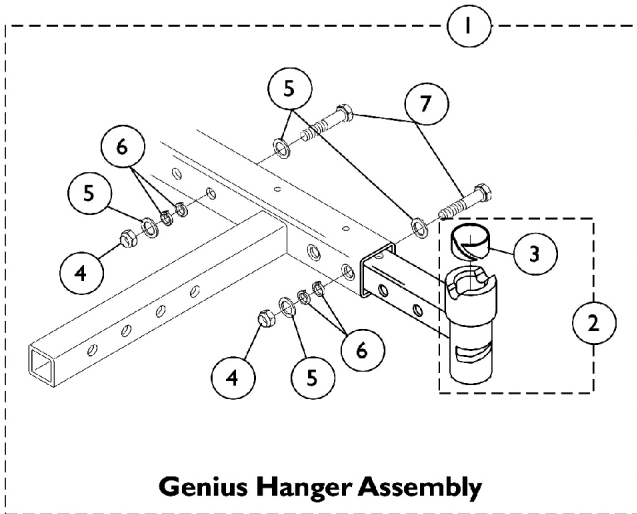

## Front Rigging Hanger Assembly Sector, Genius, Pin, AML4A Style

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1072925-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Genius Style, Black - Right	1	1
1	A	1072924-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Genius Style, Black - Left	1	1
1	A	1072926-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Genius Style, Black (Pair)	2	1
2	B	1072923-NLA	NLA - Front Rigging Hanger Assembly w/ Nylon Liner, Genius - Right	1	1
2	B	1072922-NLA	NLA - Front Rigging Hanger Assembly w/ Nylon Liner, Genius - Left	1	1
3		0181000-NLA	NLA - Liner, Nylon (3/4")	1	1
4		1025736	Locknut (1/4-20)	1	2
5		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
6		0050506	Lockwasher, High Collar, Split (1/4")	1	4
7		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
8	C	1072724-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Center Pivot, Storm, Black - Right	1	1
8	C	1072726-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Center Pivot, Storm, Black - Left	1	1
8	C	0197500-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Storm, Black (Pair)	2	1
9	D	1072722-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Center Pivot, Storm, Black - Right	1	1
9	D	1072723-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Center Pivot, Storm, Black - Left	1	1
10		0139506	Screw, Button Head, Socket (1/4-20 x 1-1/2")	1	1
11		1007453	Lockwasher, Split (1/4")	1	1
12		0117000	Block, Sector - Right	1	1
12		0117100	Block, Sector - Left	1	1
13		1025736	Locknut (1/4-20)	1	2
14		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
15		0050506	Lockwasher, High Collar, Split (1/4")	1	4
16		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
17	E	1072906	Kit, Front Rigging Hanger Assembly w/ Hardware, Smartleg, Black (AML4A) - Right	1	1
17	E	1072907	Kit, Front Rigging Hanger Assembly w/ Hardware, Smartleg, Black (AML4A) - Left	1	1
17	E	1072743	Kit, Front Rigging Hanger Assembly w/ Hardware, Smartleg, Black (AML4A) (Pair)	2	1
18		0117302-NLA	NLA - Hanger, Front Rigging, Smartleg (AML4A), Black - Right	1	1
18		0117402-NLA	NLA - Hanger, Front Rigging, Smartleg (AML4A), Black - Left	1	1
19		1025736	Locknut (1/4-20)	1	2
20		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
21		0050506	Lockwasher, High Collar, Split (1/4")	1	4
22		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
23	F	1081425-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black - Right	1	1

## Front Rigging Hanger Assembly Sector, Genius, Pin, AML4A Style

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
23	F	1081426-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black - Left	1	1
23	F	1081446-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black (Pair)	2	1
24		1078020-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Right	1	1
24		1078021-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Left	1	1
25		1027097	Plug Button, Black (3/4")	1	1
26		1025736	Locknut (1/4-20)	1	2
27		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
28		0050506	Lockwasher, High Collar, Split (1/4")	1	4
29		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
NOTE: A - Includes front rigging hanger, items 2 and 4-7 B - Includes front rigging hanger and item 3 C - Includes front rigging hanger, items 9 and 13-16 D - Includes front rigging hanger and items 10-12 E - Includes items 18-22 F - Includes items 24-29					