
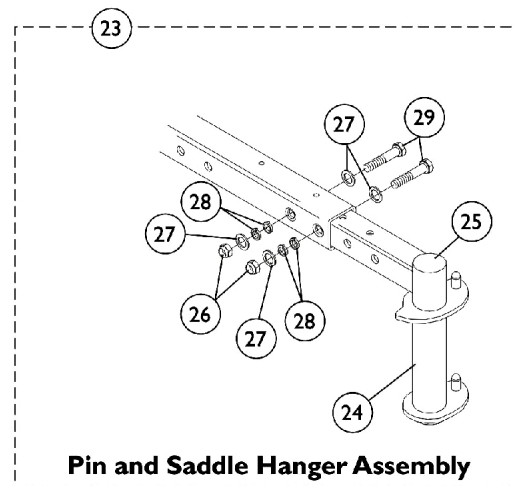
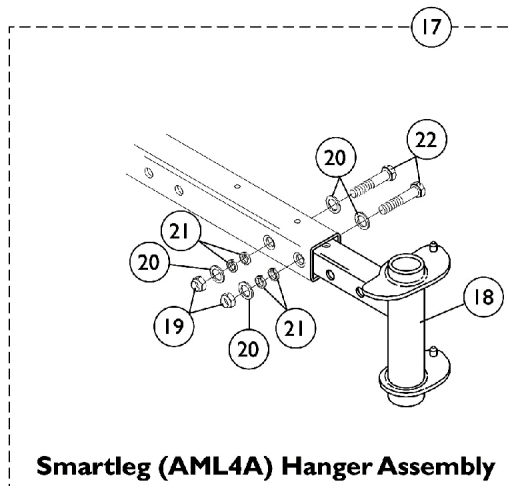
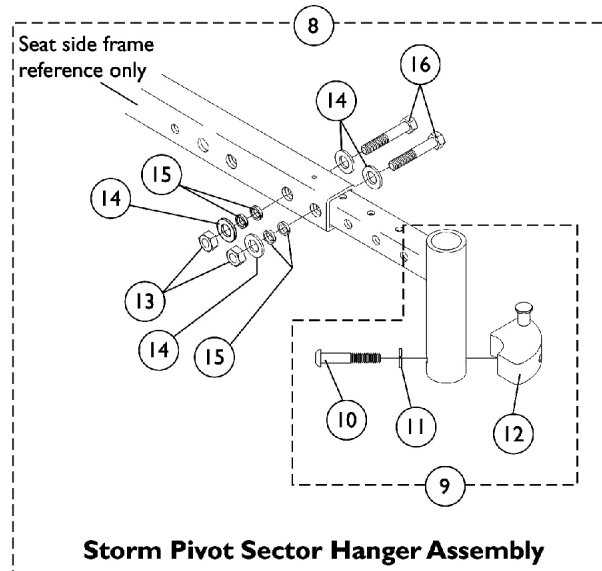


Front Rigging Hanger Assembly

Sector, Genius, Pin, AML4A Style (Basic Recline 4XP)

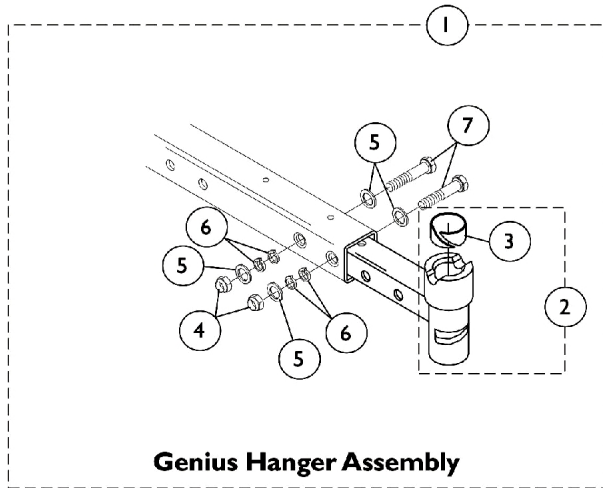

Front Rigging Hanger Assembly

Sector, Genius, Pin, AML4A Style (Basic Recline 4XP)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1077393	Kit, Front Rigging Hanger Assembly w/ Hardware, Genius Style, Black (4XP) - Right	1	1
1	A	1077394	Kit, Front Rigging Hanger Assembly w/ Hardware, Genius Style, Black (4XP) - Left	1	1
1	A	1077395-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Genius Style, Black (Pair)	2	1
2	B	1072923-NLA	NLA - Front Rigging Hanger Assembly w/ Nylon Liner, Genius - Right	1	1
2	B	1072922-NLA	NLA - Front Rigging Hanger Assembly w/ Nylon Liner, Genius - Left	1	1
3		0181000-NLA	NLA - Liner, Nylon (3/4")	1	1
4		1025736	Locknut (1/4-20)	1	2
5		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
6		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	4
7		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
8	C	1077399-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Storm, Black (4XP) - Right	1	1
8	C	1077400-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Storm, Black (4XP) - Left	1	1
8	C	1077401-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Storm, Black (Pair)	2	1
9	D	1072722-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Center Pivot, Storm, Black - Right	1	1
9	D	1072723-NLA	NLA - Front Rigging Hanger Assembly w/ Sector Block, Center Pivot, Storm, Black - Left	1	1
10		0139506	Screw, Button Head, Socket (1/4-20 x 1-1/2")	1	1
11		1007453	Lockwasher, Split (1/4")	1	1
12		0117000	Block, Sector - Right	1	1
12		0117100	Block, Sector - Left	1	1
13		1025736	Locknut (1/4-20)	1	2
14		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
15		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	4
16		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
17	E	1072906	Kit, Front Rigging Hanger Assembly w/ Hardware, Smartleg, Black (AML4A) - Right	1	1
17	E	1072907	Kit, Front Rigging Hanger Assembly w/ Hardware, Smartleg, Black (AML4A) - Left	1	1
17	E	1077398	Kit, Front Rigging Hanger Assembly w/ Hardware, Smartleg, Black (AML4A) (Pair)	2	1
18		0117302-NLA	NLA - Hanger, Front Rigging, Smartleg (AML4A), Black - Right	1	1
18		0117402-NLA	NLA - Hanger, Front Rigging, Smartleg (AML4A), Black - Left	1	1
19		1025736	Locknut (1/4-20)	1	2
20		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
21		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	4
22		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
23	F	1081425-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black - Right	1	1

Front Rigging Hanger Assembly

Sector, Genius, Pin, AML4A Style (Basic Recline 4XP)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
23	F	1081426-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black - Left	1	1
23	F	1081446-NLA	NLA - Kit, Front Rigging Hanger Assembly w/ Hardware, Pin Style, Black (Pair)	2	1
24		1078020-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Right	1	1
24		1078021-NLA	NLA - Hanger, Front Rigging, Pin Style, Black - Left	1	1
25		1027097	Plug Button, Black (3/4")	1	1
26		1025736	Locknut (1/4-20)	1	2
27		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	4
28		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	4
29		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
NOTE: A - Includes front rigging hanger, items 2 and 4-7 B - Includes front rigging hanger and item 3 C - Includes front rigging hanger, items 9 and 13-16 D - Includes front rigging hanger and items 10-12 E - Includes items 18-22 F - Includes items 24-29					