


Rigid Side Guard

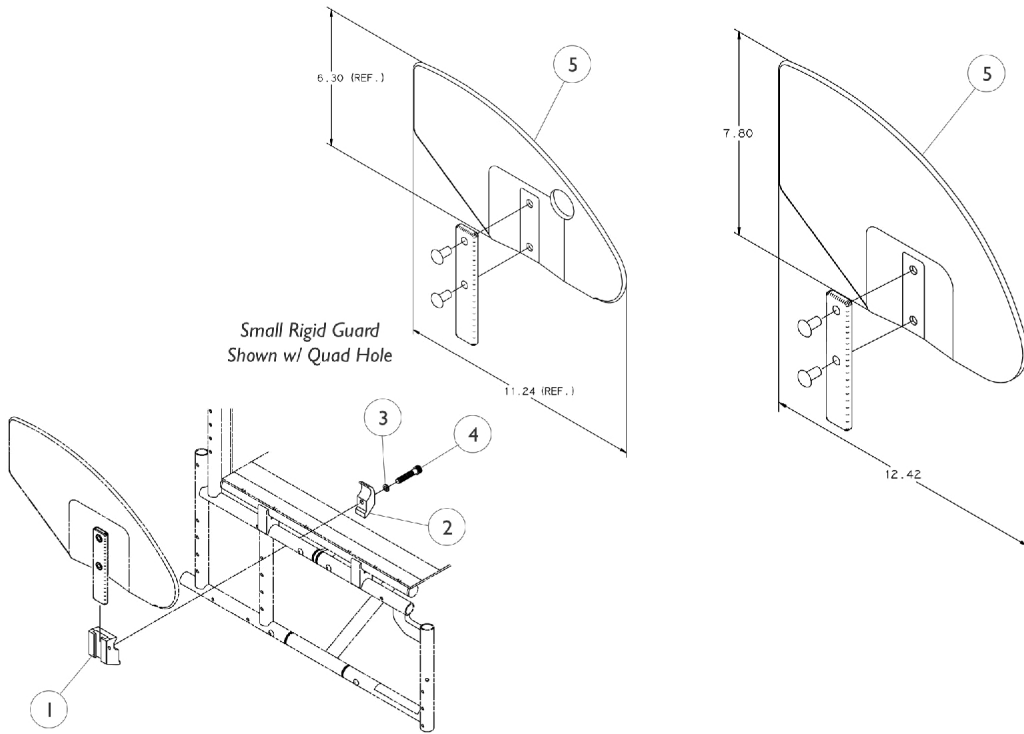

Rigid Side Guard

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1032249	Clamp, Half, Threaded	1	2
2		1018692-NLA	NLA - Clamp, Half, Non-Threaded	1	2
3		1013971	Washer, Nylon (1/4 x 7/16 x 5/64")	1	2
4		1003545	Screw, Socket Head (1/4-20 x 1-1/4")	1	2
5		1036342	Kit, Side Guard Hardware (Pair)	1	1
5		1036346	Guard, Side, Small w/o Quad Hole - Right (6-1/4" H x 11-1/4" D)	1	1
5		1036347	Guard, Side, Small w/o Quad Hole - Left (6-1/4" H x 11-1/4" D)	1	1
5		1036407	Guard, Side, Small w/ Quad Hole - Right (6-1/4" H x 11-1/4" D)	1	1
5		1036408	Guard, Side, Small w/ Quad Hole - Left (6-1/4" H x 11-1/4" D)	1	1
5		1036344	Guard, Side, Large w/o Quad Hole - Right (7-3/4" H x 12-3/8" D)	1	1
5		1036345	Guard, Side, Large w/o Quad Hole - Left (7-3/4" H x 12-3/8" D)	1	1
5		1036405	Guard, Side, Large w/ Quad Hole - Right (7-3/4" H x 12-3/8" D)	1	1
5		1036406	Guard, Side, Large w/ Quad Hole - Left (7-3/4" H x 12-3/8" D)	1	1

NOTE: A - Includes Items 1-4.
B - Not Shown