
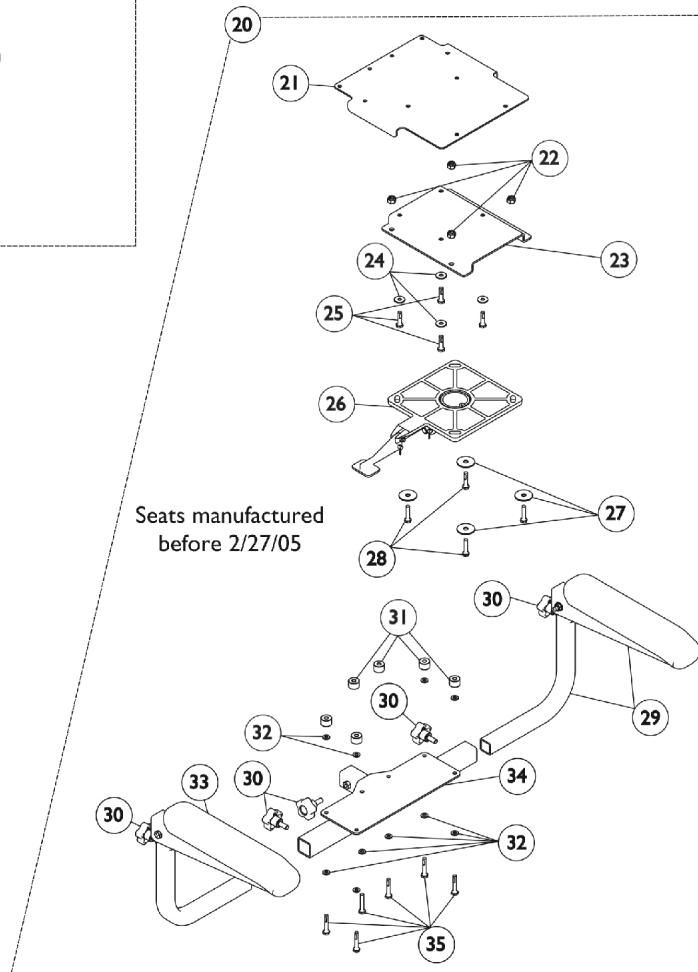
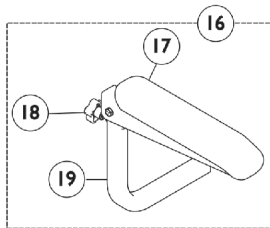
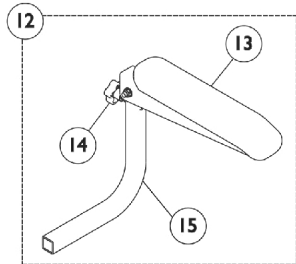
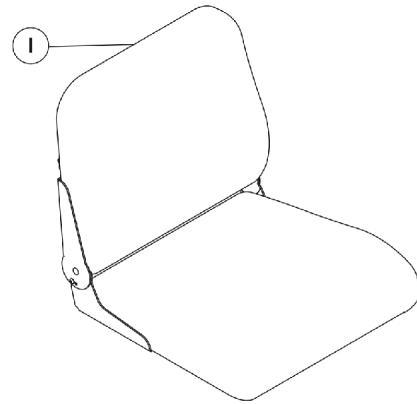


# 18-20" Seat and Arms

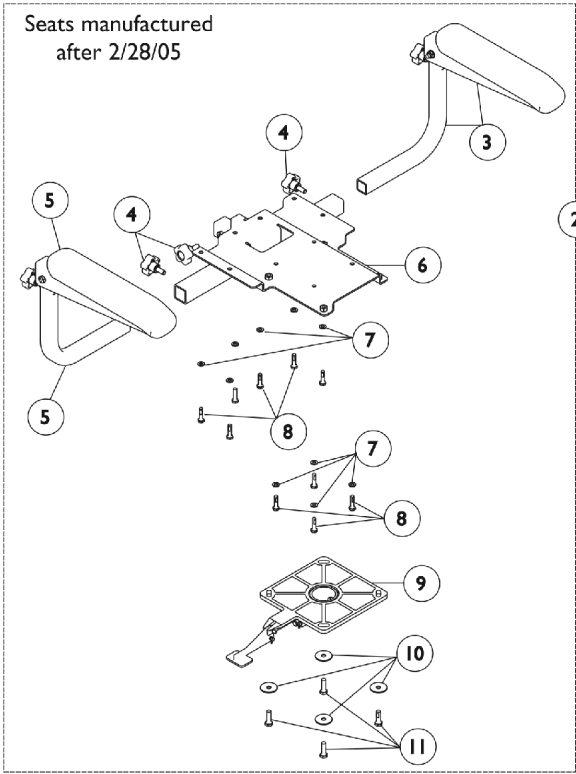

## 18-20" Seat and Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1116973-NLA	NLA - Seat Assembly, Grey (18" W)	1	1
1		1125341-NLA	NLA - Seat Assembly, Grey ( 20" W)	1	1
2	A,F	1133152	NLA - Kit, Arm Assembly with Seat Pivot (18-20" Wide Seat)	1	1
3			NLA - Arm Assembly, Flip-Up, Black - Left	1	1
4			Knob, Adjustment, Black (5/16-18 x 3/4")	1	3
5			NLA - Arm Assembly, Flip-Up, Black - Right	1	1
6			NLA - Plate, Reinforcement (18-20" Wide Chairs)	1	1
7			Washer (1/4 x 1/2 x 1/16")	1	10
8			NLA - Screw, Hex Head Cap w/ Patch (1/4-20 x 1")	1	10
9			NLA - Pivot, Seat	1	1
10			Washer, Steel (5/16 x 1-1/4 x 1/16")	1	4
11			Screw, Hex Head w/ Patch (5/16-18 x 1")	1	4
12	B,G	1114228-NLA	NLA - Arm Assembly, Flip-Up, Black - Left	1	1
12	B	1137221-NLA	NLA - Arm Assembly, Flip-Up, Black - Left	1	1
13		1116976-NLA	NLA - Arm Pad Assembly, Desk Length with Hardware - Left	1	1
14		1114910	Knob, Adjustment, Black (5/16-18 x 3/4")	1	1
15	G	1116906-NLA	NLA - Tube, Lower Arm - Left	1	1
15		1113647	Tube, Arm, Lower Adjustable Width, Black - Left	1	1
16	C,G	1114227-NLA	NLA - Arm Assembly, Flip-Up, Black - Right	1	1
16	C	1137220-NLA	NLA - Arm Assembly, Flip-Up, Black - Right	1	1
17		1116975-NLA	NLA - Arm Pad Assembly, Desk Length with Hardware - Right	1	1
18		1114910	Knob, Adjustment, Black (5/16-18 x 3/4")	1	1
19	G	1116905-NLA	NLA - Tube, Lower Arm - Right	1	1
19		1113646	Tube, Arm, Lower Adjustable Width, Black - Right	1	1
20	D,E	1119791-NLA	NLA - Kit, Arm Assembly with Seat Pivot (18-20" Wide Seat)	1	1
21			Plate, Reinforcement	1	1
22			Locknut (5/16-18)	1	4
23			NLA - Bracket, Seat Mounting, Black	1	1
24			Washer (1/4 x 23/32 x 1/16")	1	4
25			NLA - Screw, Hex Head Cap w/ Patch (1/4-20 x 1")	1	4
26			NLA - Pivot, Seat	1	1
27			Washer, Steel (5/16 x 1-1/4 x 1/16")	1	4
28			Screw, Hex Head w/ Patch (5/16-18 x 1")	1	4
29			NLA - Arm Assembly, Flip-Up, Black - Left	1	1
30			Knob, Adjustment, Black (5/16-18 x 3/4")	1	3
31			NLA - Bushing (1/4 x 3/4 x 1/2")	1	6
32			Washer (1/4 x 1/2 x 1/16")	1	6
33			NLA - Arm Assembly, Flip-Up, Black - Right	1	1
34			NLA - Bracket, Arm Support	1	1
35			NLA - Screw, Hex Head w/Patch (1/4-20 x 1-3/8")	1	6

## 18-20" Seat and Arms

NOTE: A - Includes: Items 3-11  
B - Includes: Items 13-15  
C - Includes: Items 17-19  
D - Includes: Items 21-35  
E - For Seat Assemblies Manufactured BEFORE 2/27/05  
F - For Seat Assemblies Manufactured AFTER 2/28/05  
G - Due to style change, Arm Rests need to be replaced in pairs. Order 1 ea. of 1137221 Left and 1137220 Right