
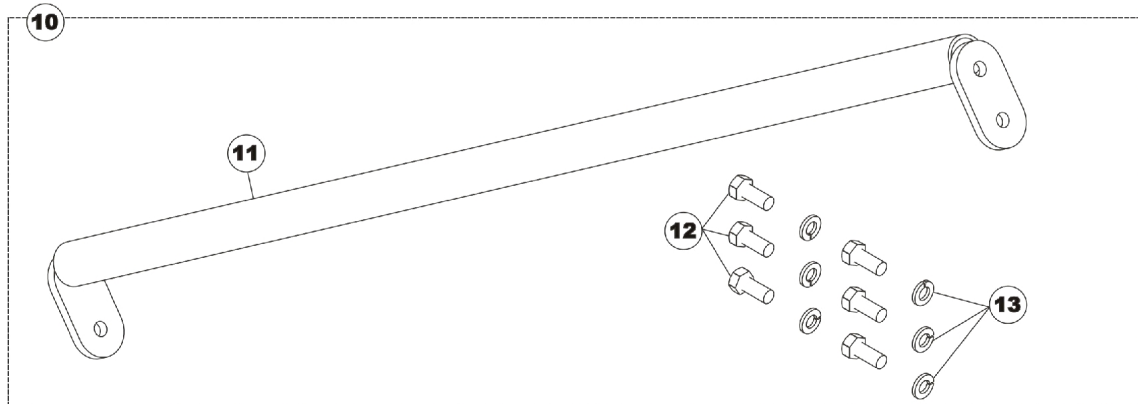


## Half Length Assist Rail - 6632

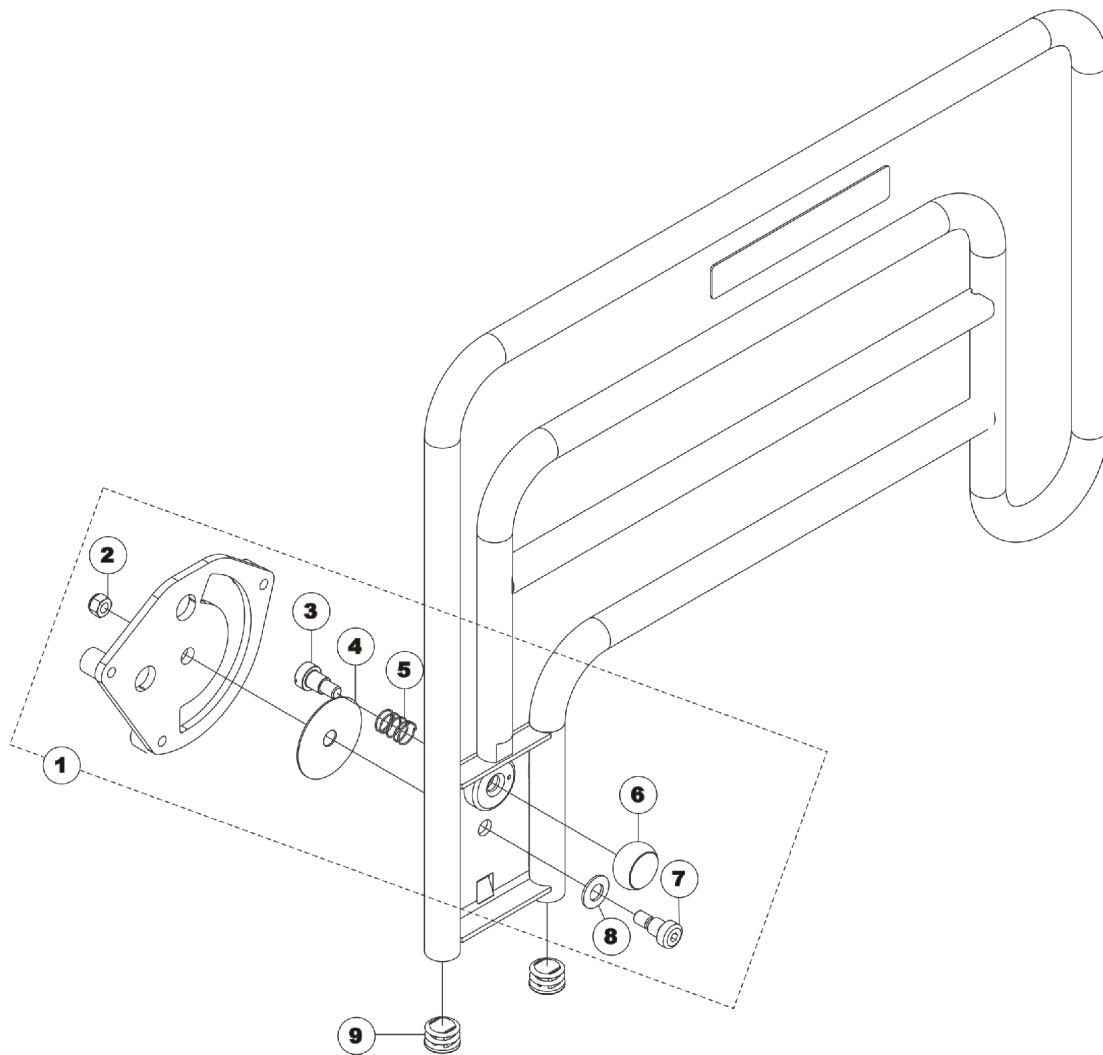

## Half Length Assist Rail - 6632

| Item Number                                                | Note | Part Number | Description                                          | Quantity Per |          |
|------------------------------------------------------------|------|-------------|------------------------------------------------------|--------------|----------|
|                                                            |      |             |                                                      | Package      | Assembly |
| 1                                                          | A    | 1134721     | Kit, Bed Rail Hardware                               | 1            | 1        |
| 2                                                          |      |             | Locknut (3/8-16)                                     | 1            | 1        |
| 3                                                          |      |             | Screw, Shoulder Slotted Head (1/2 x 1/2", 3/8-16)    | 1            | 1        |
| 4                                                          |      |             | Washer, Nylon (33/64 x 2-1/2 x 3/64")                | 1            | 1        |
| 5                                                          |      |             | Spring, Compression (3/4 x 1 x 3/64")                | 1            | 1        |
| 6                                                          |      |             | Knob, LATCH-0002 (3/8-16 x 1-3/8")                   | 1            | 1        |
| 7                                                          |      |             | Screw, Shoulder Hex Socket Head (1/2 x 1/2", 3/8-16) | 1            | 1        |
| 8                                                          |      |             | Washer, Belleville (33/64 x 1-1/64 x 3/32")          | 1            | 1        |
| 9                                                          | B    | 1029501     | Plug Button, Black (1")                              | 1            | 2        |
| 10                                                         |      | 1134720-NLA | NLA - Kit, Cross Tube Replacement                    | 1            | 1        |
| 11                                                         |      |             | NLA - Crossbrace, Assist Rail, Brown                 | 1            | 1        |
| 12                                                         |      |             | NLA - Screw, Hex Head (5/16-18 x 3/4")               | 1            | 6        |
| 13                                                         |      |             | NLA - Washer, Split Lock (5/16")                     | 1            | 6        |
| NOTE: A - Includes: Items 2-8<br>B - Includes: Items 11-13 |      |             |                                                      |              |          |