


Walking Beam Assembly (Before 4/26/05)

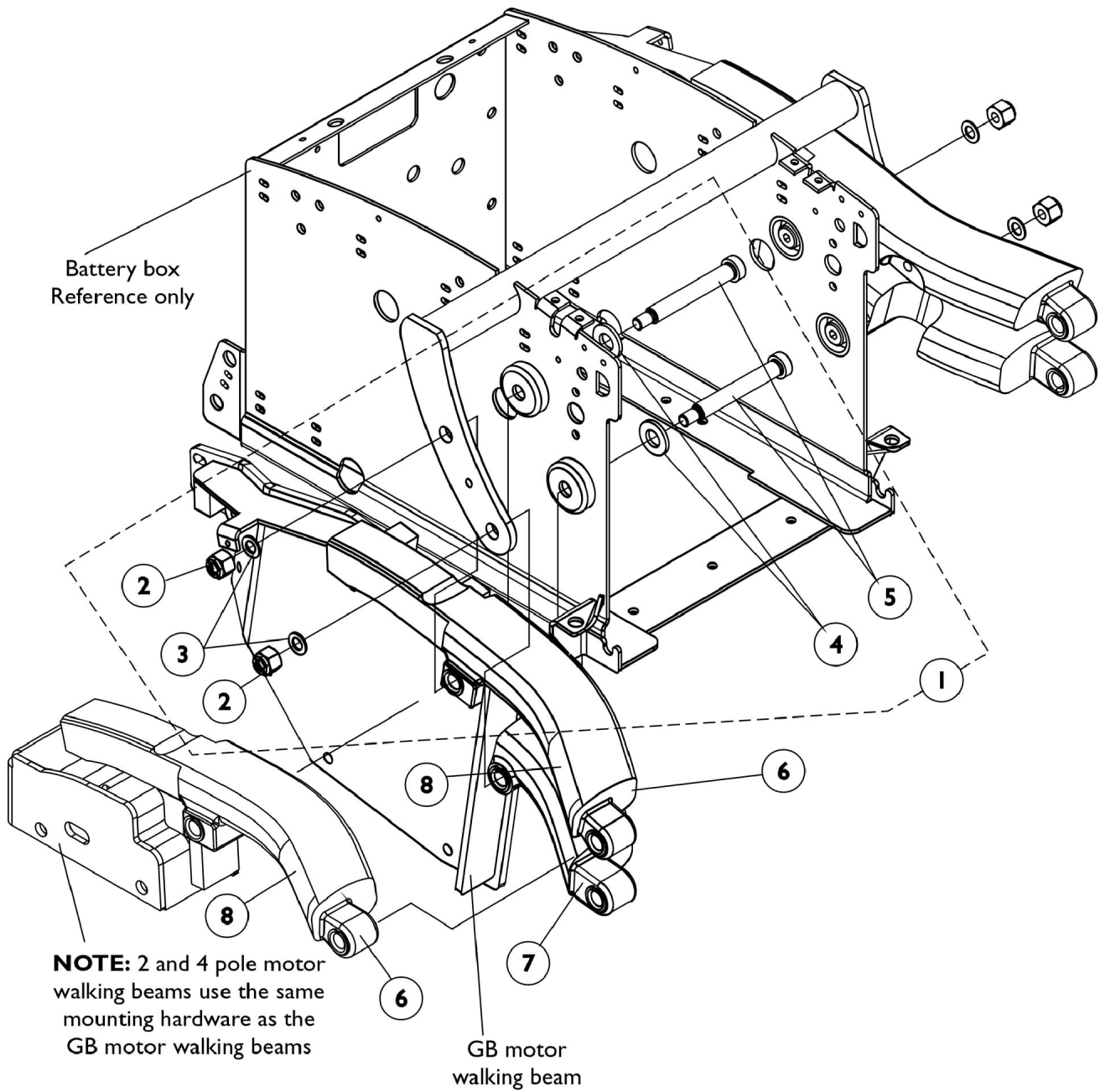

Walking Beam Assembly (Before 4/26/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1120355-NLA	NLA - Hardware, Walking Beam Mounting	1	1
2		1065216	Locknut (3/8-16)	1	2
3		1065214	Washer, Flat (3/8 x 11/16 x 1/16")	1	2
4		1003766-NLA	NLA - Washer, Plain (1/2 x 1 x 1/8")	1	2
5		1113246-NLA	NLA - Screw, Shoulder Hex Socket Head (1/2 x 3-1/4", 3/8-16)	1	2
6	B	8881113144-NL A	NLA - Beam, Walking without Model Decal GB - Right	1	1
6	B	8881113145	NLA - Beam, Walking without Model Decal GB - Left	1	1
6	B	8881113158-NL A	NLA - Walking Beam w/o Model Decal, 2 and 4 Pole Motor - Right	1	1
6	B	8881113159-NL A	NLA - Walking Beam w/o Model Decal, 2 and 4 Pole Motor - Left	1	1
7	B	8881113218-NL A	NLA - Walking Beam, Lower - Right	1	1
7	B	8881113219-NL A	NLA - Walking Beam, Lower - Left	1	1
8		1117207-NLA	NLA - Decal, TDX 3, Yellow	1	1
8		1117206-NLA	NLA - Decal, TDX 4, Yellow	1	1
8		1117205-NLA	NLA - Decal, TDX 5, Yellow	1	1

NOTE: A - Includes items 2-5
B - Specify frame color when ordering, see page titled "Frame Finishes and Touch-Up Paints".