

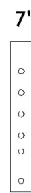


Backrest Frame and Rear Upright Tubes

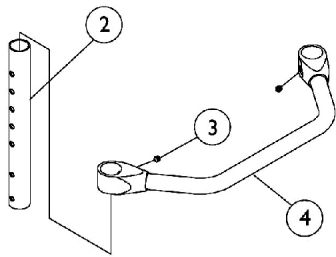

1042514

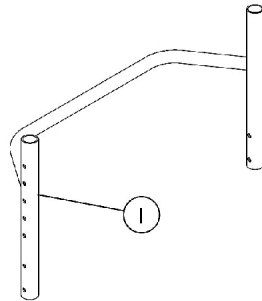


1042513



1043358



1042515

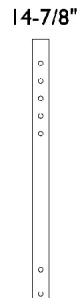

1052226

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 One Piece Backrest Frame	1	1
2		1042514-NLA	NLA - Tube, Rear Upright, Black (8 - 11" H)	1	2
2		1042513-NLA	NLA - Tube, Rear Upright, Black (10 - 14" H)	1	2
2		1043358	Tube, Rear Upright, Black (12 - 16" H)	1	2
2		1042515-NLA	NLA - Tube, Rear Upright, Black (14 - 18" H)	1	2
2		1052226-NLA	NLA - Tube, Rear Upright, Black (16 - 20" H)	1	2
3		1045495-NLA	NLA - Screw, Set Socket (1/4-20 x 1/4")	1	2
4		See Chart	#4 Backrest Crossmember	1	1

Backrest Frame and Rear Upright Tubes

#1 One Piece Backrest Frame

Chair Width	8-11" High	10-14" High	12-16" High	14-18" High	16-20" High
14"	1024706-NLA	1026188-NLA	1042982-NLA	1026757-NLA	1026764-NLA
15"	1024707-NLA	1026189-NLA	1042986-NLA	1026758-NLA	1026765-NLA
16"	1024708-NLA	1026190-NLA	1042987-NLA	1026759-NLA	1026766-NLA
17"	1024709-NLA	1026191-NLA	1042988-NLA	1026760-NLA	1026767-NLA
18"	1024710-NLA	1026192-NLA	1042989-NLA	1026761-NLA	1026768-NLA
19"	1024711-NLA	1026193-NLA	1042990-NLA	1026762-NLA	1026769-NLA
20"	1024712-NLA	1026194-NLA	1042991	1026763-NLA	1026770-NLA

#4 Backrest Crossmember

14" Wide	15" Wide	16" Wide	17" Wide	18" Wide	19" Wide	20" Wide
1042510	1042511	1042512	1050083	1050080	1050077	1050074