


ASBA Seat Frame and Hardware (12"-15" Wide) (Before 7/5/05)

The 16" wide setting is not used for 3G Storm Series models

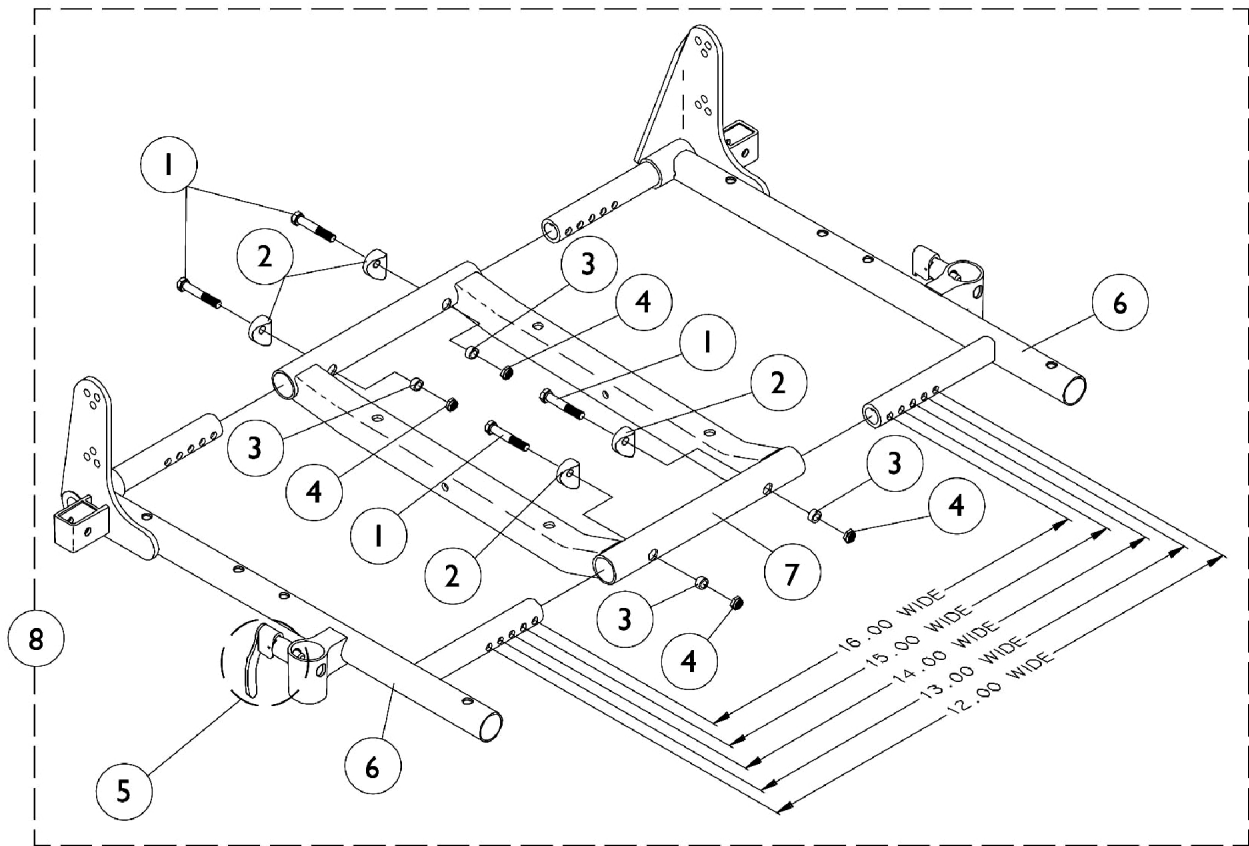

ASBA Seat Frame and Hardware (12"-15" Wide) (Before 7/5/05)

The 16" wide setting is not used for 3G Storm Series models

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	4
2		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
3		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
4		1025736	Locknut (1/4-20)	1	4
5		47471X074	Spring, Arm Lock	1	1
5		1028594	Pin, Arm Lock, Black	1	1
5		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
6-8		See Chart	#'s 6-8 ASBA Seat Frame and Hardware	1	1

#'s 6-8 Seat Frames

Item Number	Description	12"-13" W 12"-15" D	14"-15" W 12"-15" D	14"-15" W 16"-17" D	14"-15" W 18"-19" D	14"-15" W 20"-21" D	14"-15" W 22" D
6	Seat Frame Right See Note(s) A	1070473-NLA	1070473-NLA	1070473-NLA	1070526-NLA	1070528-NLA	1070530-NLA
6	Seat Frame Left See Note(s) A	1070474-NLA	1070474-NLA	1070474-NLA	1070527-NLA	1070529-NLA	1070531-NLA
7	Seat Frame Center	1070476-NLA	1070476-NLA	1070476-NLA	1070476-NLA	1070533-NLA	1070533-NLA
8	Seat Frame Assembly See Note(s) B	1070469-NLA	1070469-NLA	1070469-NLA	1070534-NLA	1070535-NLA	1070536-NLA
NOTE: A - Seat frame includes item 5 arm lock assembly. B - Includes items 1-7							