


Seat Frame Mounting Hardware

(12"-15" Wide ASBA), (12"-16" Wide ADJASBA Seats & TRBKTS Option)

NOTE: The ASBA and ADJASBA seat frame assemblies use the same mounting hardware.

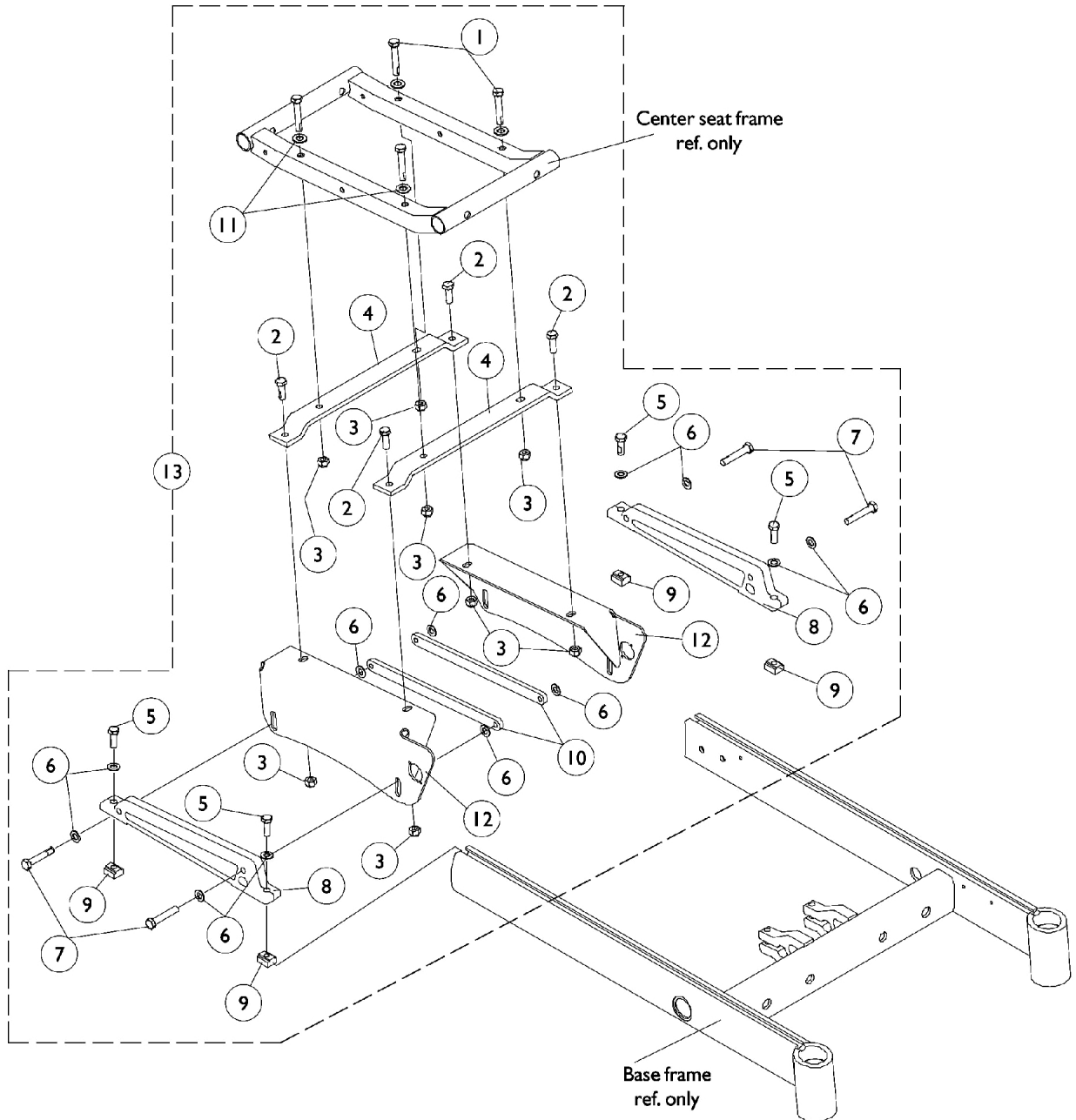

Seat Frame Mounting Hardware (12"-15" Wide ASBA), (12"-16" Wide ADJASBA Seats & TRBKTS Option)

NOTE: The ASBA and ADJASBA seat frame assemblies use the same mounting hardware.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1042484-NLA	NLA - Screw, Hex Head (5/16-18 x 2")	1	4
2		1070462-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 7/8")	1	4
3		1026158	Locknut (5/16-18)	1	8
4		1070465-NLA	NLA - Bracket, Mounting, Black - Junior	1	2
5		1070462-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 7/8")	1	4
6		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	12
7		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
8		1070388	Support, Seat, Black	1	2
9		1069724	T-Nut (5/16-18)	1	4
10		1070396-NLA	NLA - Strap, Bolt, Black	1	2
11		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	4
12	B	1104428	Plate, Low Seat Mounting - Right	1	1
12	B	1104429	Plate, Low Seat Mounting - Left	1	1
12	C	1104430	Plate, Medium Seat Mounting - Right	1	1
12	C	1104431	Plate, Medium Seat Mounting - Left	1	1
12	D	1104432	Plate, Tall Seat Mounting - Right	1	1
12	D	1104433	Plate, Tall Seat Mounting - Left	1	1
13	A,E	1104765	Hardware, Seat Frame Mounting	1	1

NOTE: A - Item number 12 not included
 B - Low 17-1/2" Seat to Floor
 C - Medium 19-3/4" Seat to Floor
 D - Tall 21-1/2" Seat to Floor
 E - Includes items 1-11