


Seat Frame Mounting Hardware (16" - 24" Wide) ASBA (Before 7/5/05)

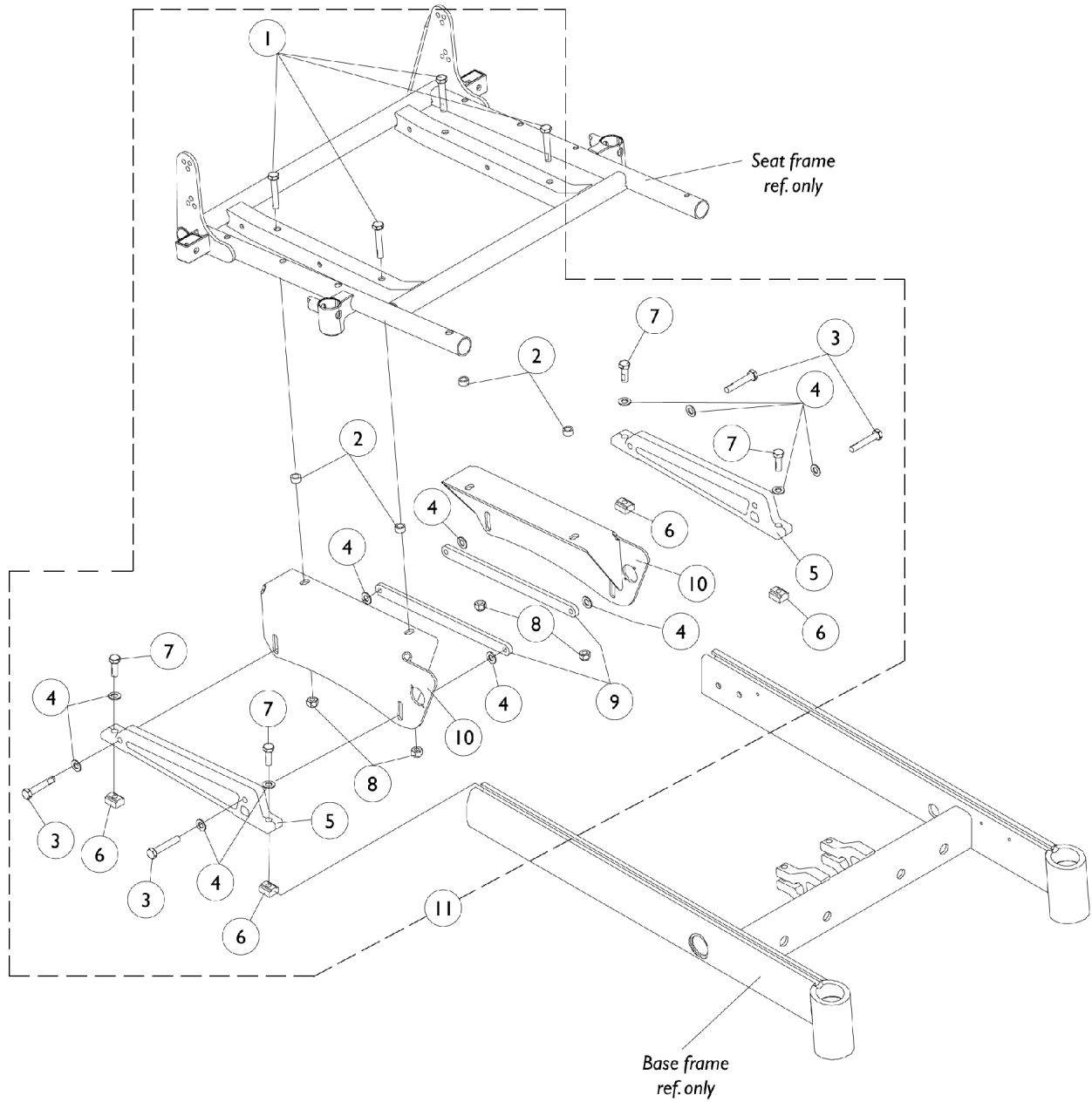

Seat Frame Mounting Hardware (16" - 24" Wide) ASBA (Before 7/5/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
1	B	1070398-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	4
2	A	1070413-NLA	NLA - Spacer (21/64 x 1/2 x 17/64")	1	4
3		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	4
4		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	12
5		1070388	Support, Seat, Black	1	2
6		1069724	T-Nut (5/16-18)	1	4
7		1070462-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 7/8")	1	4
8		1026158	Locknut (5/16-18)	1	4
9		1070396-NLA	NLA - Strap, Bolt, Black	1	2
10	D	1104428	Plate, Low Seat Mounting - Right	1	1
10	D	1104429	Plate, Low Seat Mounting - Left	1	1
10	E	1104430	Plate, Medium Seat Mounting - Right	1	1
10	E	1104431	Plate, Medium Seat Mounting - Left	1	1
10	F	1104432	Plate, Tall Seat Mounting - Right	1	1
10	F	1104433	Plate, Tall Seat Mounting - Left	1	1
11	C,G	1104764-NLA	NLA - Hardware, Seat Frame Mounting (16" W)	1	1
11	C,G	1104763-NLA	NLA - Hardware, Seat Frame Mounting (17" W and Wider)	1	1

NOTE: A - Part number 1043627 and 1071413 are used on the 16" wide chairs only
B - Part number 1070398 is used on 17" and wider chairs only
C - Item number 10 not included
D - Low 17-1/2" Seat to Floor
E - Medium 19-3/4" Seat to Floor
F - Tall 21-1/2" Seat to Floor
G - Includes items 1-9