


Seat Pan and Attaching Hardware

For ASBA Seat (Before 7/5/05)

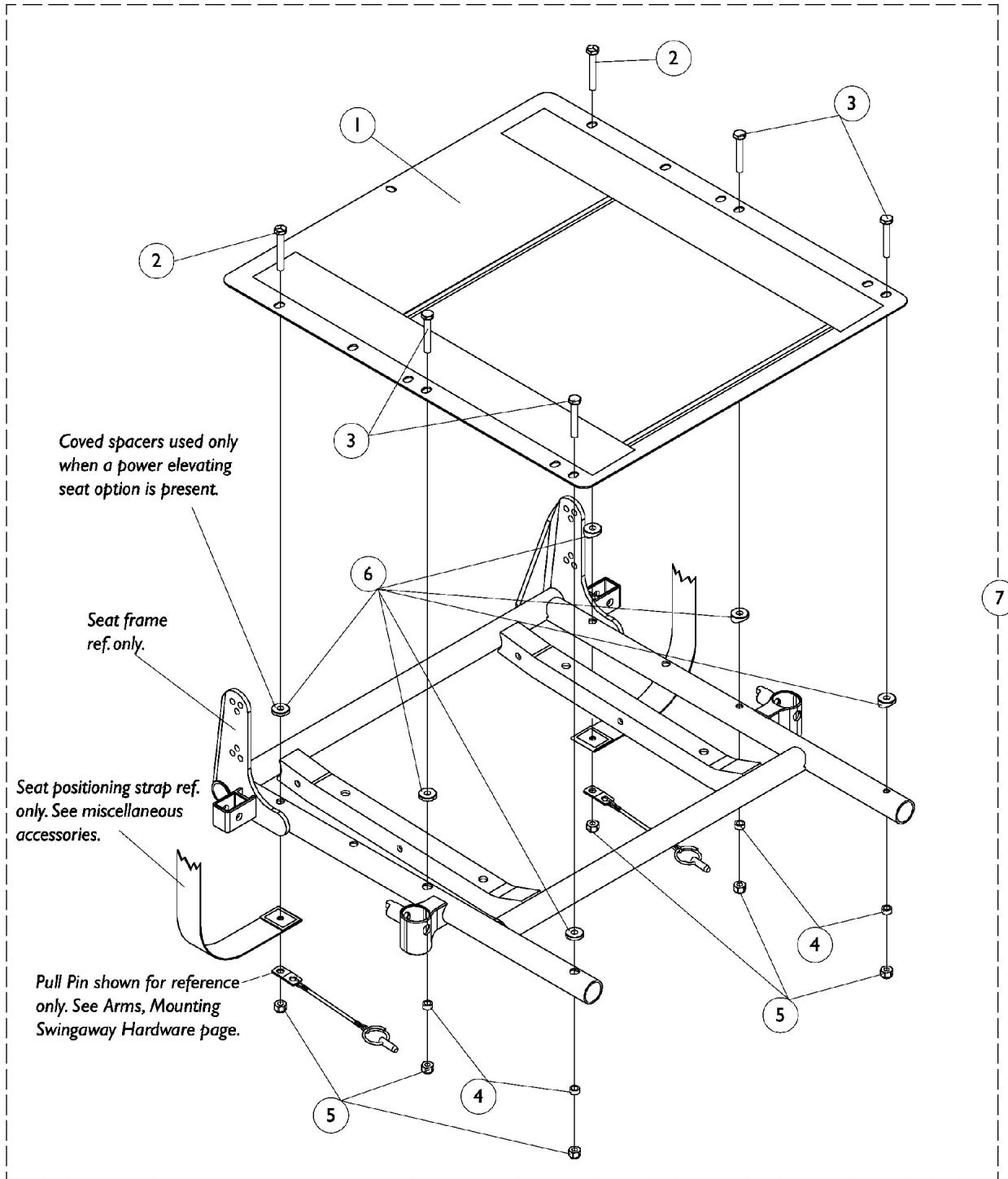

Seat Pan and Attaching Hardware For ASBA Seat (Before 7/5/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Seat Pan - 12"-18" Depth	1	1
1		See Chart	#1 Seat Pan - 19"-22" Depth	1	1
7	B	1070459-NLA	NLA - Kit, Seat Pan Mounting Hardware	1	1
2			Screw, Hex Head (1/4-20 x 1-5/8")	1	2
3			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
4			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
5			Locknut (1/4-20)	1	6
7	A,C	1109336-NLA	Kit, Seat Pan Mounting Hardware	1	1
2			NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
3			Screw, Hex Head (1/4-20 x 1-5/8")	1	4
4			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
5			Locknut (1/4-20)	1	6
6			Spacer, Coved (1/4 x 11/16 x 1/2")	1	6

NOTE: A - Hardware used when a power elevating seat option is present.
 B - Includes items 2-5
 C - Includes items 2-6

#1 Seat Pan

12"-18" Depth

Chair Width	12"-15" Deep	16" Deep	17" Deep	18" Deep
12" W	1066009-NLA	1070513	N/A	N/A
13" W	1066010-NLA	1070514-NLA	N/A	N/A
14" W	1066022-NLA	1066011-NLA	1066022-NLA	1066033-NLA
15" W	1066023-NLA	1066012-NLA	1066023-NLA	1066034-NLA
16" W	1066024	1066013-NLA	1066024	1066035-NLA
17" W	N/A	1066014-NLA	1066025-NLA	1066036-NLA
18" W	N/A	1066015-NLA	1066026	1066037-NLA
19" W	N/A	1066016-NLA	1066027-NLA	1066038-NLA
20" W	N/A	1066017-NLA	1066028-NLA	1066039-NLA
21" W	N/A	1066018-NLA	1066029-NLA	1066040-NLA
22" W	N/A	1066019-NLA	1066030-NLA	1066041-NLA
23" W	N/A	1066020-NLA	1066031-NLA	1066042-NLA
24" W	N/A	1066021-NLA	1066032-NLA	1066043-NLA

12-15" seat depths are achieved by using CJ back cane mounting hardware.

Seat Pan and Attaching Hardware For ASBA Seat (Before 7/5/05)

#1 Seat Pan

19"-22" Depth

Chair Width	19" Deep	20" Deep	21" Deep	22" Deep
14" W	1066044-NLA	1066055-NLA	1066066-NLA	1066077-NLA
15" W	1066045-NLA	1066056-NLA	1066067-NLA	1066078-NLA
16" W	1066046-NLA	1066057-NLA	1066068-NLA	1066079-NLA
17" W	1066047-NLA	1066058-NLA	1066069-NLA	1066080-NLA
18" W	1066048-NLA	1066059-NLA	1066070-NLA	1066081-NLA
19" W	1066049-NLA	1066060	1066071-NLA	1066082-NLA
20" W	1066050-NLA	1066061-NLA	1066072-NLA	1066083-NLA
21" W	1066051-NLA	1066062-NLA	1066073-NLA	1066084-NLA
22" W	1066052-NLA	1066063-NLA	1066074-NLA	1066085-NLA
23" W	1066053-NLA	1066064-NLA	1066075-NLA	1066086-NLA
24" W	1066054-NLA	1066065-NLA	1066076-NLA	1066087-NLA