


Seat Pan and Mounting Hardware

For ASBA Seats (12-15" Wide) & (16-22" Wide Before 7/5/05)

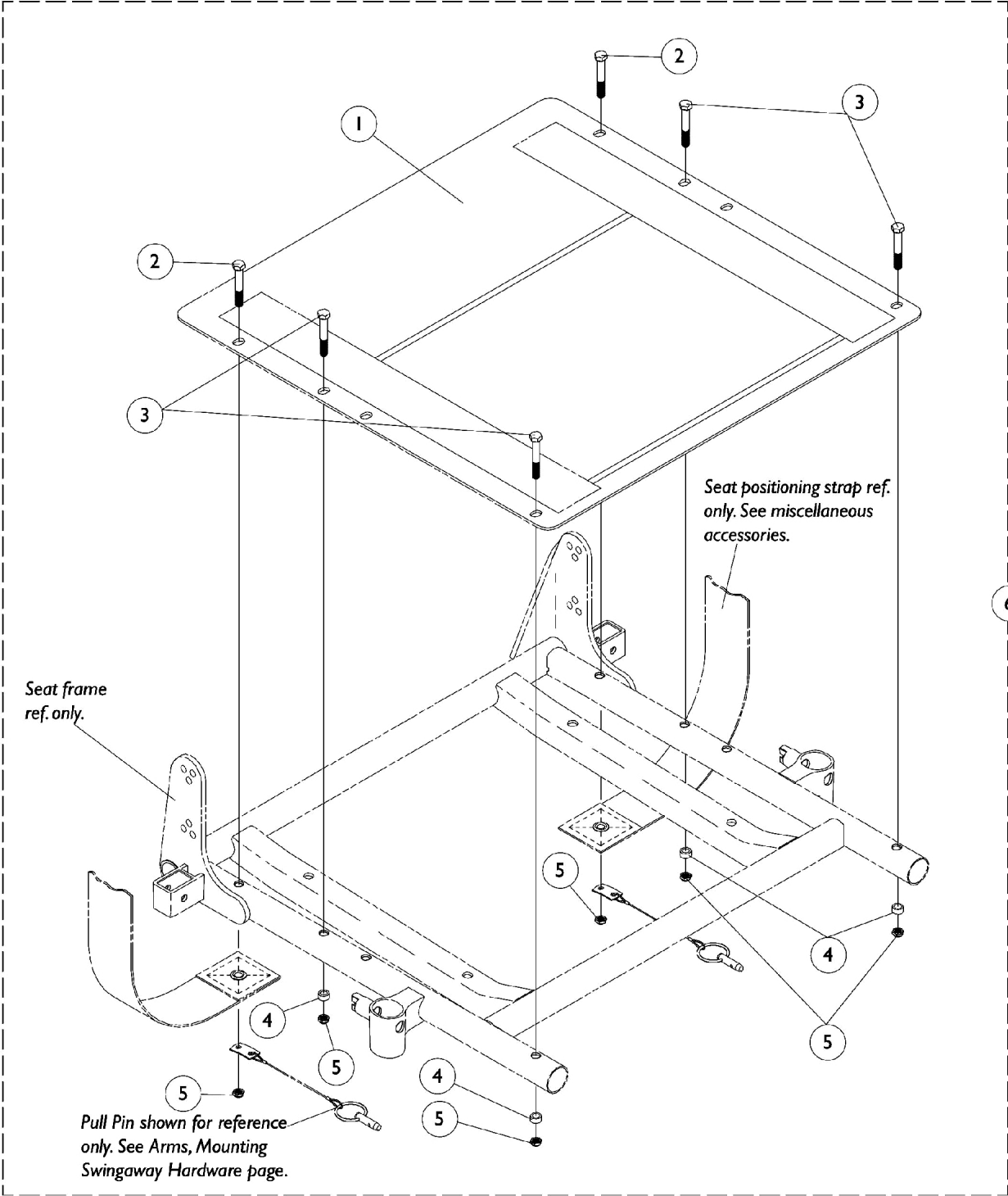

Seat Pan and Mounting Hardware

For ASBA Seats (12-15" Wide) & (16-22" Wide Before 7/5/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Seat Pan 12"-18"	1	1
1		See Chart	#1 Seat Pan 19"-22"	1	1
6	A	1070459-NLA	NLA - Kit, Seat Pan Mounting Hardware	1	1
2			Screw, Hex Head (1/4-20 x 1-5/8")	1	2
3			Screw, Hex Head (1/4-20 x 1-1/2")	1	4
4			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
5			Locknut (1/4-20)	1	6

NOTE: A - Includes items 2-5

#1 Seat Pan - 12" D to 18" D

Seat Width	12-15" Deep	16" Deep	17" Deep	18" Deep
12" Wide	1066009-NLA	1070513	N/A	N/A
13" Wide	1066010-NLA	1070514-NLA	N/A	N/A
14" Wide	1066022-NLA	1066011-NLA	1066022-NLA	1066033-NLA
15" Wide	1066023-NLA	1066012-NLA	1066023-NLA	1066034-NLA
16" Wide	1066024	1066013-NLA	1066024	1066035-NLA
17" Wide	N/A	1066014-NLA	1066025-NLA	1066036-NLA
18" Wide	N/A	1066015-NLA	1066026	1066037-NLA
19" Wide	N/A	1066016-NLA	1066027-NLA	1066038-NLA
20" Wide	N/A	1066017-NLA	1066028-NLA	1066039-NLA
21" Wide	N/A	1066018-NLA	1066029-NLA	1066040-NLA
22" Wide	N/A	1066019-NLA	1066030-NLA	1066041-NLA

NOTE: 12-15" seat depths are achieved by using CJ back cane mounting hardware.

Seat Pan and Mounting Hardware
For ASBA Seats (12-15" Wide) & (16-22" Wide Before 7/5/05)
#1 Seat Pan - 19" D to 22" D

Seat Width	19" Deep	20" Deep	21" Deep	22" Deep
12" Wide	N/A	N/A	N/A	N/A
13" Wide	N/A	N/A	N/A	N/A
14" Wide	1066044-NLA	1066055-NLA	1066066-NLA	1066077-NLA
15" Wide	1066045-NLA	1066056-NLA	1066067-NLA	1066078-NLA
16" Wide	1066046-NLA	1066057-NLA	1066068-NLA	1066079-NLA
17" Wide	1066047-NLA	1066058-NLA	1066069-NLA	1066080-NLA
18" Wide	1066048-NLA	1066059-NLA	1066070-NLA	1066081-NLA
19" Wide	1066049-NLA	1066060	1066071-NLA	1066082-NLA
20" Wide	1066050-NLA	1066061-NLA	1066072-NLA	1066083-NLA
21" Wide	1066051-NLA	1066062-NLA	1066073-NLA	1066084-NLA
22" Wide	1066052-NLA	1066063-NLA	1066074-NLA	1066085-NLA