


# ASBA Seat Frame and Hardware (12"-15" Wide) (Before 7/5/05)

**NOTE:** Center seat frame was changed 10/9/03 to accommodate Junior Center Mount Riggings. The center seat frame is backward compatible to all Junior Seat Frame Assemblies made before 10/9/03.

**NOTE:** The 16" wide setting is not used for TDX models.

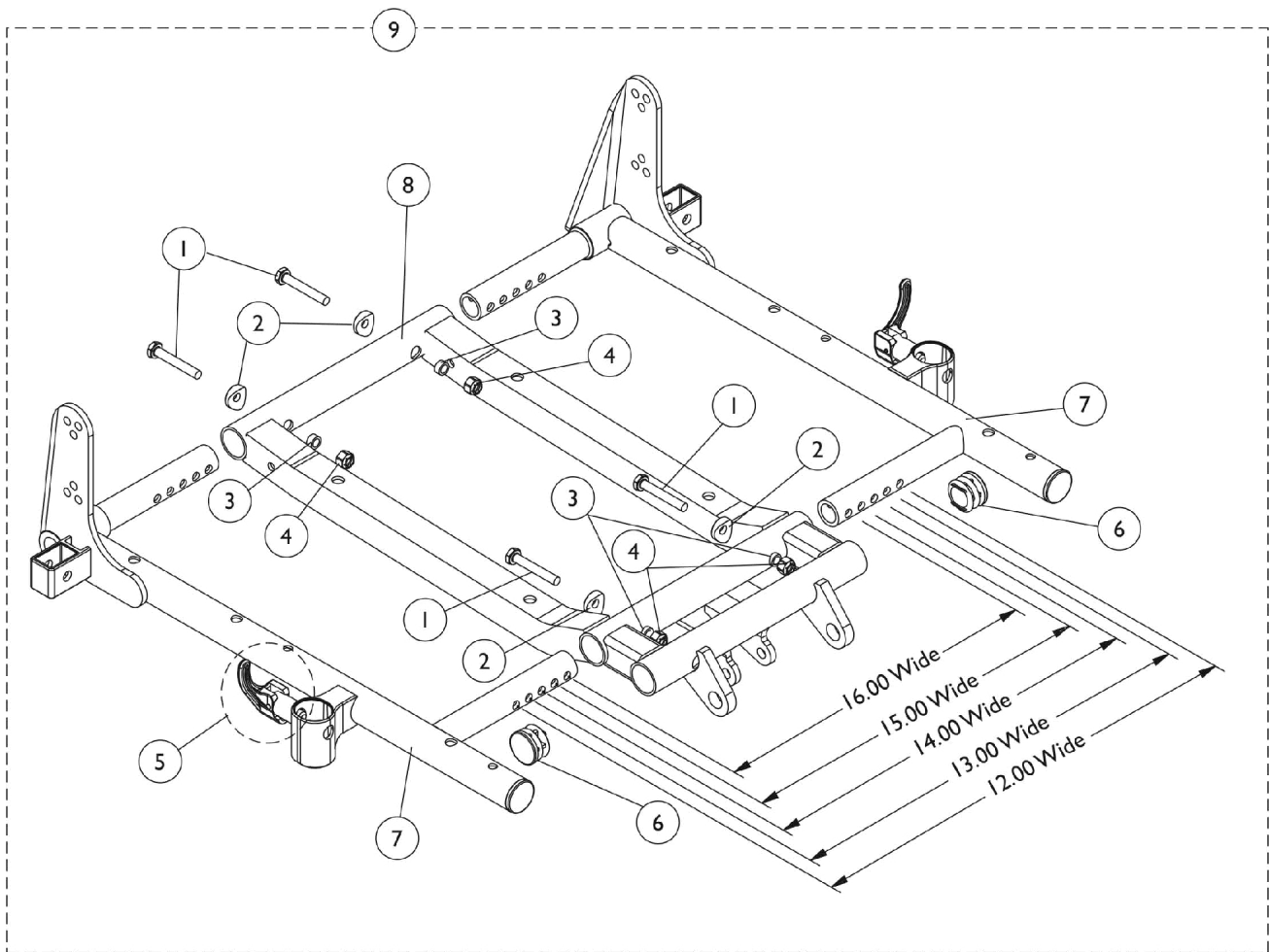

## ASBA Seat Frame and Hardware (12"-15" Wide) (Before 7/5/05)

**NOTE: Center seat frame was changed 10/9/03 to accommodate Junior Center Mount Riggings. The center seat frame is backward compatible to all Junior Seat Frame Assemblies made before 10/9/03.**

**NOTE: The 16" wide setting is not used for TDX models.**

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	4
2		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
3		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
4		7930333	Locknut (1/4-20)	25	4
5		47471X074	Spring, Arm Lock	1	1
5		1028594	Pin, Arm Lock, Black	1	1
5		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
6		1029501	Plug Button, Black (1")	1	2
7-9		See Chart	#'s 7-9 ASBA Seat Frame and Hardware	1	1

### #'s 7-9 Seat Frames

Item Number	Description	12"-13" W	14"-15" W	14"-15" W	14"-15" W	14"-15" W	14"-15" W
		12"-16" D	12"-15" D	16"-17" D	18"-19" D	20"-21" D	22" D
7	Seat Frame Right See Note(s) A	1070473-NLA	1070473-NLA	1070473-NLA	1070526-NLA	1070528-NLA	1070530-NLA
7	Seat Frame Left See Note(s) A	1070474-NLA	1070474-NLA	1070474-NLA	1070527-NLA	1070529-NLA	1070531-NLA
8	Seat Frame Center See Note(s) C	1121982-NLA	1121982-NLA	1121982-NLA	1121982-NLA	1121995-NLA	1121995-NLA
9	Seat Frame Assembly See Note(s) B	1121991-NLA	1121991-NLA	1121991-NLA	1121992-NLA	1121993-NLA	1121994-NLA

**NOTE:** A - Seat frame includes item 5 arm lock assembly.  
 B - Includes items 1-8  
 C - Center seat frame revised 10/1/03 to accommodate Junior Center Mount Riggings. Center seat frame is backward compatible for all chairs made before 10/1/03 as well.