
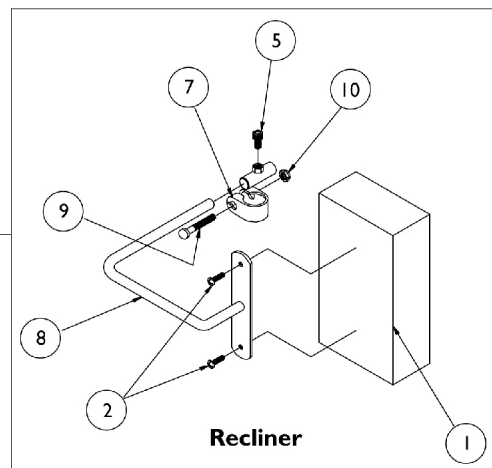


Torso Support

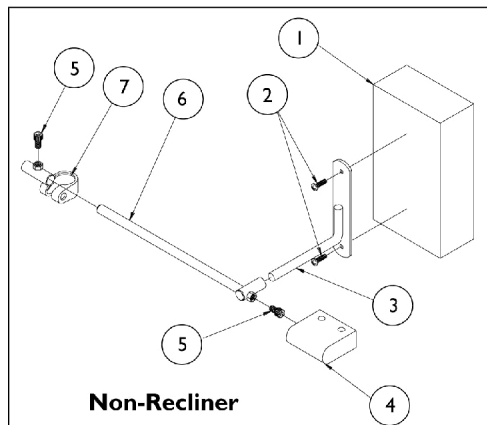

11

Torso Support

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,G	8881123986-NL A	NLA - Pad, Headrest	1	1
1	A	88804529X040- NLA	NLA - Pad, Headrest	1	1
2		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
3		03018X016-NLA	NLA - Bracket, Support, Chrome	1	1
4		88804529X043- NLA	NLA - Cover Torso Bracket	1	1
5		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	1
6		03018X012-NLA	NLA - Rod, Support w/ Guide Tube	1	1
7	E	03018X043	Bracket, Clamp, Black- Right	1	1
7	E	03018X046	Bracket, Clamp, Black - Left	1	1
7	F	1154437-NLA	NLA - Bracket Clamp, Silver Vein - Right	1	1
7	F	1154438-NLA	NLA - Bracket Clamp, Silver Vein - Left	1	1
8	E	03018X041-NLA	NLA - Bracket, Pad Support, Chrome	1	1
8	F	1154436-NLA	NLA - Bracket Pad Support, Silver Vein	1	1
9		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		60264X001	Locknut (5/16-24)	1	1
11	A,B	8881050189-NL A	NLA - Torso Support Assembly (Pair)	2	1
11	A,C	8881050190-NL A	NLA - Torso Support Assembly (Pair)	2	1
	D,H	1023610-NLA	NLA - Instruction Sheet	1	1
NOTE: A - Specify upholstery color when ordering, see category titled, "Upholstery Color Options and Packaging." B - Non-Recliner C - Recliner D - Not Shown E - Before 7/6/07. F - After 7/5/07. G - No Longer Available. Use 88804529X040 as replacement. H - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					