


ASBA Seat Frame Mounting Hardware (16"-24" Wide) (Before 7/5/05)

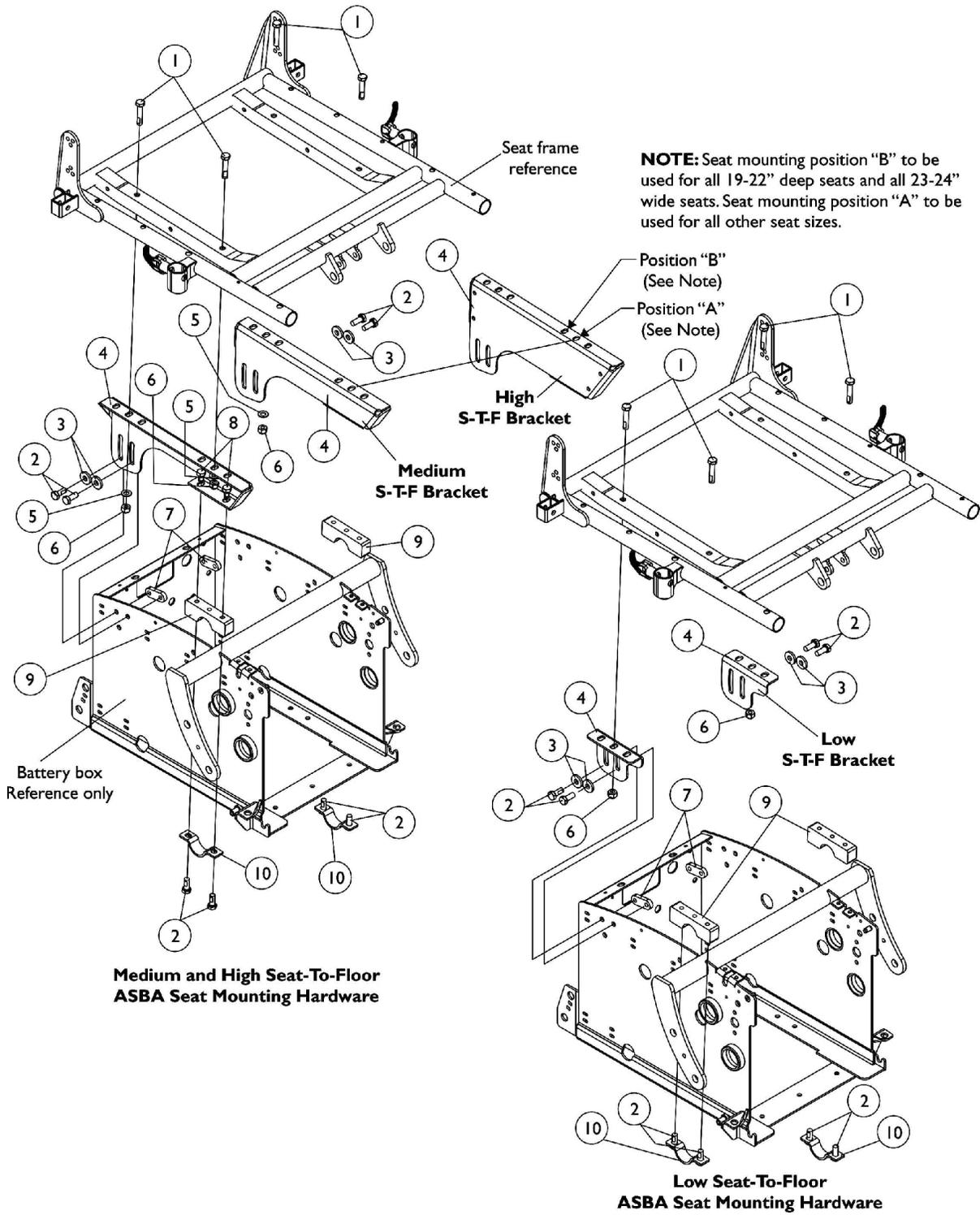

ASBA Seat Frame Mounting Hardware (16"-24" Wide) (Before 7/5/05)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1116983-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-1/2")	1	4
2		1039860	Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	8
3		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	4
4	A	1113228-NLA	NLA - Plate, Low Seat-To-Floor Mounting, Rear - Right	1	1
4	A	1113229-NLA	NLA - Plate, Low Seat-To-Floor Mounting, Rear - Left	1	1
4	B	1115850-NLA	NLA - Plate, Medium Seat-To-Floor Mounting - Right	1	1
4	B	1115851-NLA	NLA - Plate, Medium Seat-To-Floor Mounting - Left	1	1
4	C	1115852-NLA	NLA - Plate, High Seat-To-Floor Mounting - Right	1	1
4	C	1115853-NLA	NLA - Plate, High Seat-To-Floor Mounting - Left	1	1
5		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	4
6		1026158	Locknut (5/16-18)	1	4
7		1113341-NLA	NLA - Nut, Strap	1	2
8		1061922-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1/2")	1	4
9		1113114-NLA	NLA - Bracket, Upper Seat, Front, Black	1	2
10		1113230-NLA	NLA - Bracket, Lower Seat, Front, Black	1	2
<p>NOTE: A - 17" Seat-To-Floor @ 5 degree tilt B - 19" Seat-To-Floor @ 5 degree tilt. No longer available as a service part individually. See category Titled "Frames - Seat ADJASBA Mtg. Hdwr. (16" - 24" W) Med S-T-F (After 7/4/05)" and order part #1133758 as a replacement. C - 21" Seat-To-Floor @ 5 degree tilt. No longer available as a service part individually. See category titled "Frames - Seat ADJASBA Mtg. Hdwr. (16" - 24" W) High S-T-F (After 7/4/05)" and order part #1133757 as a replacement.</p>					