


Back Canes and Spreader Bars

For ADJASBA Seats (12"-16" Wide) (After 7/4/05)

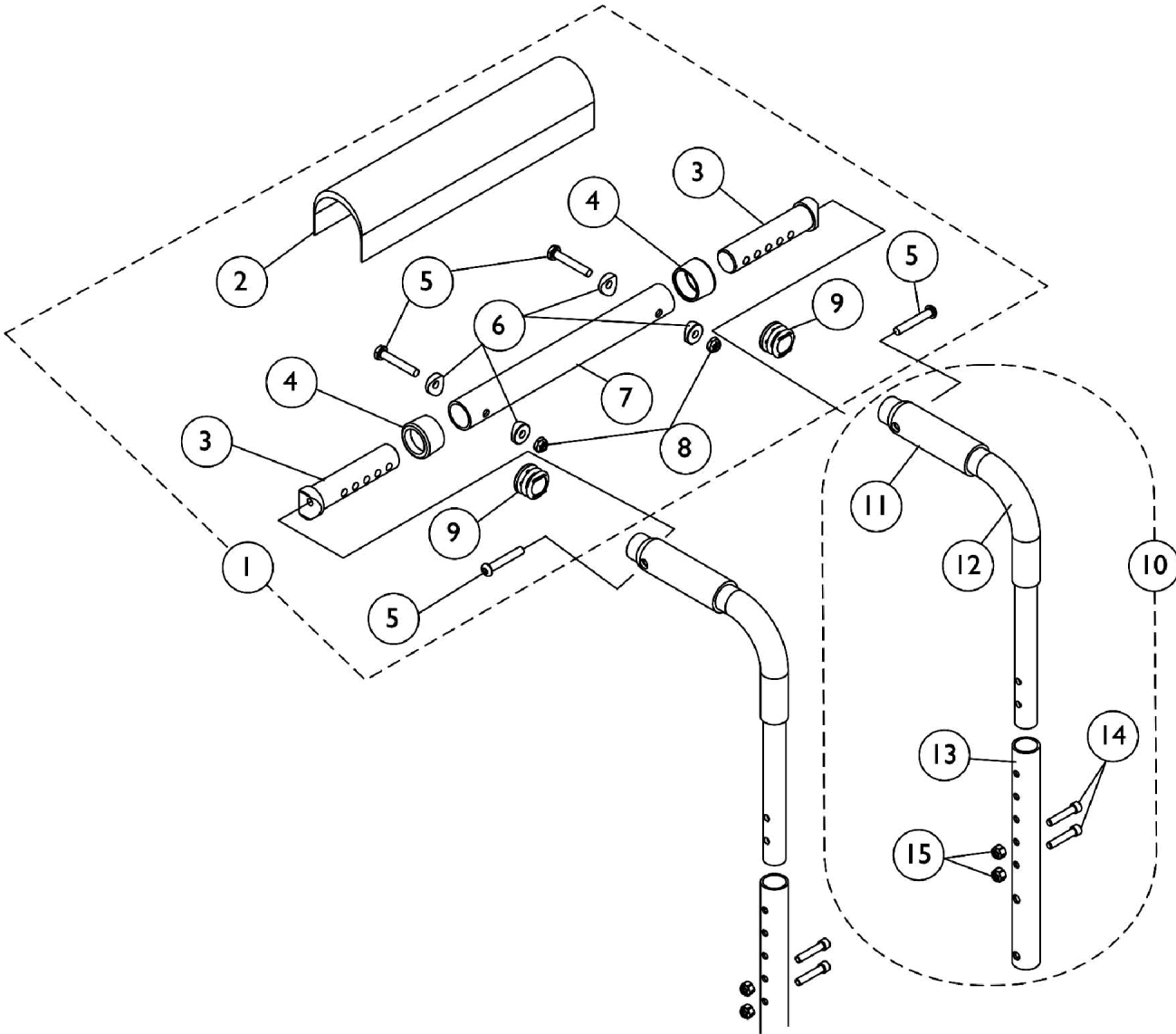

Back Canes and Spreader Bars

For ADJASBA Seats (12"-16" Wide) (After 7/4/05)

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|---|--------------|----------|
| | | | | Package | Assembly |
| 1 | A | 1126147 | Kit, Spreader Bar Assembly (12"-16" Wide) | 1 | 1 |
| 2 | | | Cover, Spreader Bar (16-20" W) | 1 | 1 |
| 3 | | | NLA - Insert, Spreader Bar Connector, Black | 1 | 2 |
| 4 | | | Anti-Rattle (1") | 1 | 2 |
| 5 | | | Screw, Socket Button Head (1/4-20 x 1-1/2") | 1 | 4 |
| 6 | | | Spacer, Coved (1/4 x 11/16 x 1/2") | 1 | 4 |
| 7 | | | NLA - Tube, Spreader Bar, Black | 1 | 1 |
| 8 | | | Locknut (1/4-20) | 1 | 2 |
| 9 | | | Plug Button, Black (1") | 1 | 2 |
| 10 | B | 1127154-NLA | NLA - Kit, Back Cane Assembly with Hardware (12"-15" Back Height) | 1 | 1 |
| 11 | | | Grip, Hand, Foam (4") | 1 | 1 |
| 12 | | | Handle, Push, Black | 1 | 1 |
| 13 | | | Cane, Back, Lower, Black - Right/Left (12-15" H) | 1 | 1 |
| 14 | | | Screw, Socket Head (1/4-20 x 1-1/4") | 1 | 2 |
| 15 | | | Locknut (1/4-20) | 1 | 2 |
| 10 | B | 1127155-NLA | NLA - Kit, Back Cane Assembly with Hardware (16"-19" Back Height) | 1 | 1 |
| 11 | | | Grip, Hand, Foam (4") | 1 | 1 |
| 12 | | | Handle, Push, Black | 1 | 1 |
| 13 | | | Cane, Back, Lower, Black - Right/Left (16-19" H) | 1 | 1 |
| 14 | | | Screw, Socket Head (1/4-20 x 1-1/4") | 1 | 2 |
| 15 | | | Locknut (1/4-20) | 1 | 2 |
| 10 | B | 1127156 | Kit, Back Cane Assembly with Hardware (20"-22" Back Height) | 1 | 1 |
| 11 | | | Grip, Hand, Foam (4") | 1 | 1 |
| 12 | | | Handle, Push, Black | 1 | 1 |
| 13 | | | Cane, Back, Lower, Black - Right/Left (20-22" H) | 1 | 1 |
| 14 | | | Screw, Socket Head (1/4-20 x 1-1/4") | 1 | 2 |
| 15 | | | Locknut (1/4-20) | 1 | 2 |
| 11 | | 1135505 | Package, Foam Hand Grip (4" L) | 10 | 1 |

NOTE: A - Includes items 2-9
 B - Includes items 11-15