


ASBA Seat Frame

(TDX Before 7/5/05) (M71 Before 8/8/05) (M91 Before 10/4/05)

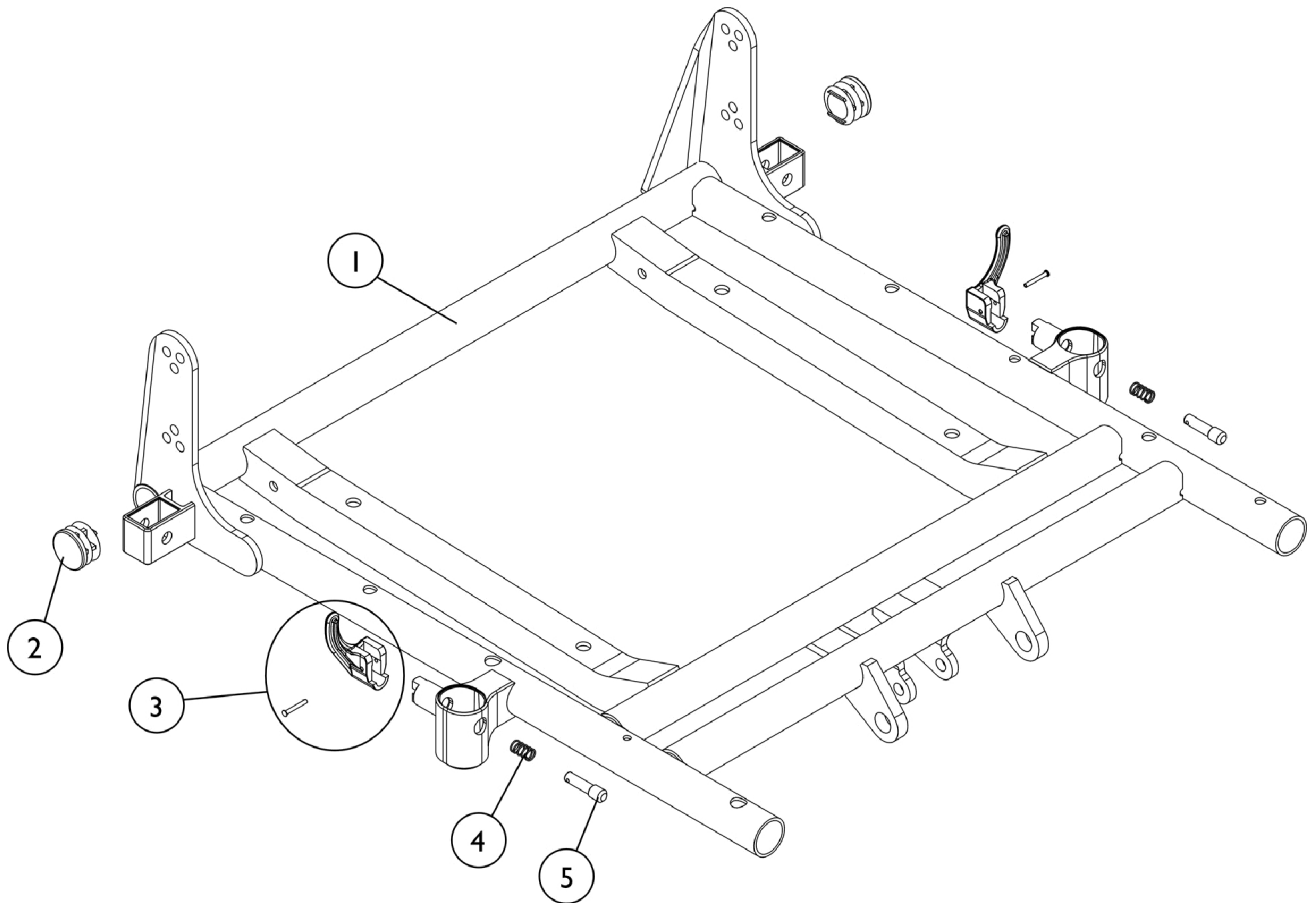

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Seat Frame with Hardware	1	1
2		1029501	Plug Button, Black (1")	1	2
3		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
4		47471X074	Spring, Arm Lock	1	2
5		1028594	Pin, Arm Lock, Black	1	2

#1 Seat Frame with Hardware

Chair Width	16"-17" Deep	18" - 19" Deep	20" - 21" Deep	22" Deep
16" Wide	1118313-NLA	1118322-NLA	1118331-NLA	1118340-NLA
17" Wide	1118314-NLA	1118323-NLA	1118332-NLA	1118341-NLA
18" Wide	1118315-NLA	1118324-NLA	1118333-NLA	1118342-NLA
19" Wide	1118316-NLA	1118325-NLA	1118334-NLA	1118343-NLA
20" Wide	1118317-NLA	1118326-NLA	1118335-NLA	1118344-NLA
21" Wide	1118318-NLA	1118327-NLA	1118336-NLA	1118345-NLA
22" Wide	1118319-NLA	1118328-NLA	1118337-NLA	1118346-NLA

ASBA Seat Frame

(TDX Before 7/5/05) (M71 Before 8/8/05) (M91 Before 10/4/05)

NOTE: Seat frame includes items 2-5