


Forks w/ Hardware

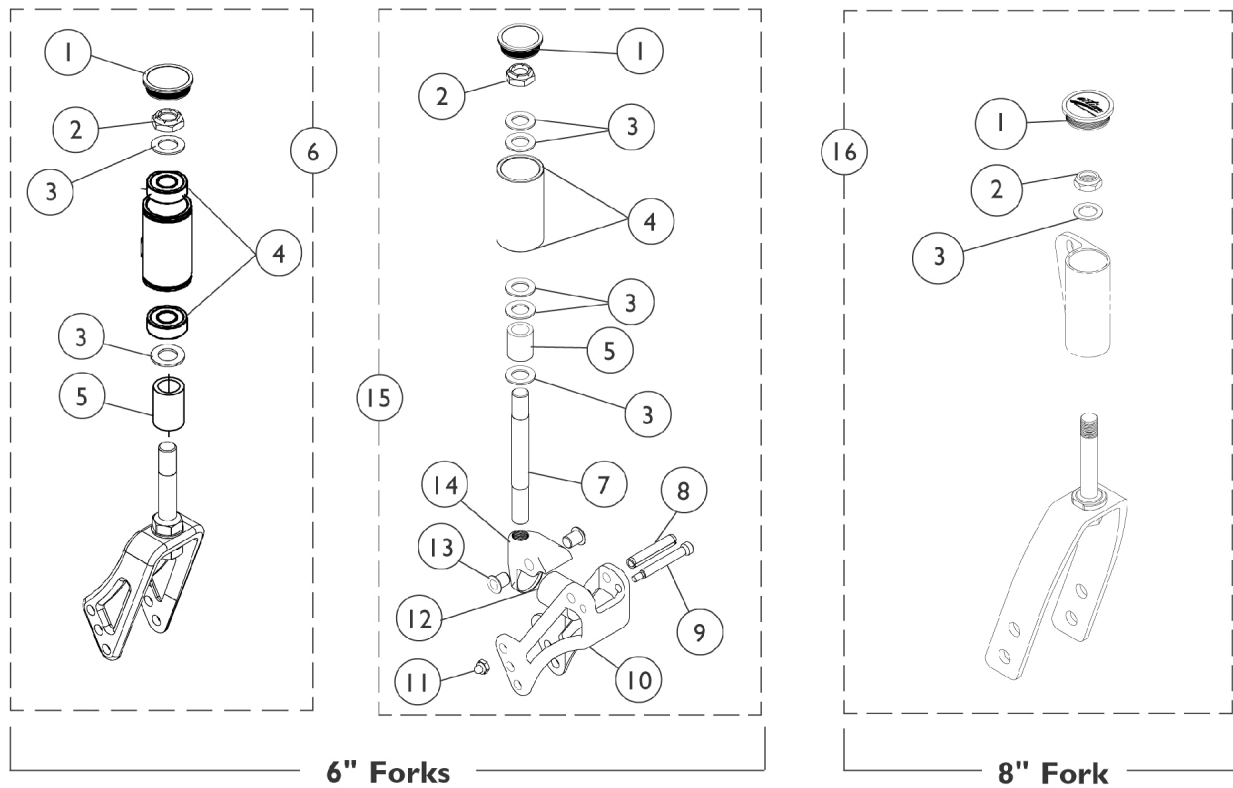

Forks w/ Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1038353	Plug Button, Headtube, Black	1	1
2		1025737	Locknut (1/2-20)	1	1
3		1026174	Washer (1/2 x 7/8 x 3/64")	1	6
4	A	40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	2
5	B	1046495	Spacer (1/2 x 3/4 x 1")	1	1
6	C	1097039-NLA	NLA - Fork with Hardware (6")	1	1
6	D	1097040-NLA	NLA - Fork, Extended with Hardware (6")	1	1
7		1024682-NLA	NLA - Stem, Fork, Black	1	1
7		1046446-NLA	NLA - Fork Stem (1" Extension)	1	1
8		1088647-NLA	NLA - Pin, Slotted (5/16 x 1-3/4")	1	1
9		1030957-NLA	NLA - Screw, Shoulder Hex Socket Head (1/4 x 1-3/4", #10-24)	1	1
10		1088570-NLA	NLA - Fork, Suspension, Front	1	1
11		1100857-NLA	NLA - Nut, Hex with Nylon Cap (#10-24)	1	1
12		1100507-NLA	NLA - Kit, Polyurethane Spacer	2	1
13		1093978-NLA	NLA - Flange (5/16" I.D.)	1	2
14		1088571-NLA	NLA - Mount, Suspension Fork	1	1
15	E	1098173-NLA	NLA - Fork, Suspension with Hardware (6")	1	1
15	F	1098174-NLA	NLA - Fork, Extended Suspension w/ Hardware (6")	1	1
16	G	1065629	Kit, Fork w/ Hardware (6 x 2" and 8" Casters)	1	1
NOTE: A - Item 4 Bearings are Not Included in Fork Assemblies. B - Spacer required for Forks w/ extended stems only (Part nos. 1097040 & 1098174). C - Includes Fork and items 1-3. D - Includes Fork and items 1-3, 5. E - Includes items 1-3,7-14. F - Includes items 1-3,5,7-14. G - Includes Fork and items 1-3.					